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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a mobility aid.

Distributor: This manual must be given to users of this product.

PRODUCT DESTINATION

The Classic steel 2 Wheels Rollator are walking aids for people with reduced mobility. These walking frames are intended for use indoors and outdoors and can be folded up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Do not use the walking frame on stairs or escalators.

Only use accessories and parts supplied by the manufacturer.

Check carefully all parts of the product's component parts (screws, tubes, tips, grips).

Check regularly if all screws are tightened.

Hold firmly and completely the handle when using. In case of falling: let the product fall down on the side, so that you don't fall on it.

Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.

Use only as a walking aid and in proper way : don't use as a lever or to move furniture.

After unfolding or assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.

DO NOT hang anything on the Walking aid.

ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.

No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.

In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.

Be careful when using: beware holes and deformations in the floor.

Be careful: risk of crushing and squeezing fingers when folding/ unfolding.

Make sure that the knobs are tightened to the same level and the handles are securely fastened.

Do not use the rollator to walk backwards.

Do not make any adjustments to the rollator while it is in use.

All wheels should always be in contact with the ground.

Make sure the seat is clipped flat before use.

Do not use the seat to carry objects.

Do not sit on the seat when the rollator is on a slope.

When sitting, do not try to reach objects by leaning forward. Do not lean backwards as this can cause the rollator to tip over.

Be careful when climbing pavements or walking on sloping, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the rollator on a wet, icy, or snowy slope.


If any parts are damaged, broken or worn, contact the dealer.

MOUNTING & USE

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient.

The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

	x2 Hand push tube		x2 screw
	x2 Curved gasket		x2 knob

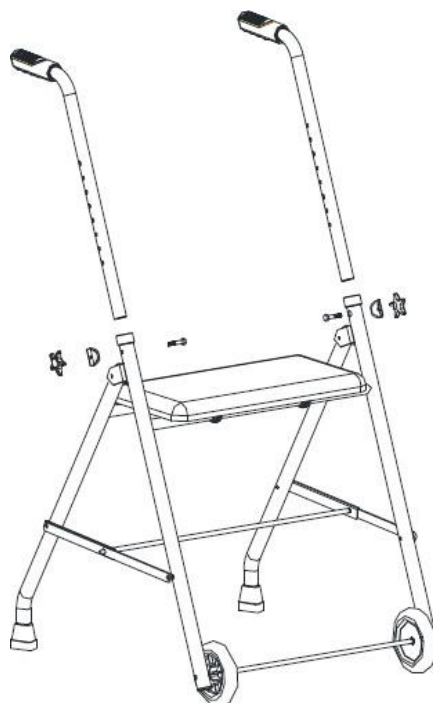CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products or detergents.

- Dry with a clean, dry soft cloth.
- Ensure that there is no build up of dirt or debris around the axle or wheels.
- Do not use cleaning products containing phenol or chlorine and all solvent-based products, as this may damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Regularly check that all screws are properly fixed and tightened.
- If a malfunction occurs, do not repair the rollator yourself. Consult an authorised dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment	X			
Structural weld at wheels		X		
Cleanliness				X
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat			X	
Condition of the wheels and pads		X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for extended periods of time near a source of heat or sunlight (e.g. behind a window or near a radiator or in a car) or near a source of cold.

Keep away from open flames and sparks.

Respect the conditions for storing your rollators.

- In a dry and temperate place
- Protect your rollator by wrapping it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air)
- Store all removed parts together, in the same place (or mark them if necessary) to avoid mixing them with other products during reassembly.
- All components should be stored without any load (do not place heavy objects on the rollator parts, do not jam anything).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

TECHNICAL SPECIFICATIONS

Overall dimensions	Length 61 cm x Width 53 cm x Height 76-92 cm
Width between handles	45 cm
Seat dimensions	Length 41 cm x Width 21 cm x Height 35 cm
Dimensions folded	Length 52 cm x Width 16 cm x Height 69 cm
Weight	4,5 kg
Maximum weight used	136 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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