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IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions, or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a physician before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

The bed rail guard is a Class I medical device that complements the use of the bed rail. This product is attached to the bed rail with fasteners and protects the agitated patient from being knocked into the bed rail. It creates a reassuring cocoon for the patient. The choice to use a bed barrier protection is made by the multidisciplinary care team. It is intended for patients with senility pathology to complement their safety. Patient monitoring is important to ensure patient acceptance.

SAFETY PRECAUTIONS

Failure to observe the precautions for use may result in serious injury.

Check the condition of the seams and fasteners each time you use the bedrail. Do not use a damaged bedrail. Do not use if straps are torn or buckles are broken.

Check that the guard is accepted by the patient and that they will not remove it. If the patient tries to remove it, do not use it and tell the referring doctor, as the patient could be injured by the straps. Choose the right length for the bed rail and the type of bed.

USE

The bed rail protector can only be fitted by the carers. The bed rail protector is attached to the bed rail. It is fastened with straps around the bed rail using the straps and quick-release fasteners.

Each time it is used, check the condition of the seams and the condition of the straps and buckles. It should be checked that it fits properly and that it holds well to the gate. If it does not fit properly and is not suitable for the gate, do not use it.

• **Checklist :**

In order to maintain the product in good condition, check regularly as follows:

Type of control	Method	Frequency
Presence of tears, untied product	Visual checking	Each use
Checks on fasteners (straps, buckles)	Visual checking	Each use

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the bed barrier regularly with a non-chlorinated bactericide. (Spray, wipe)
- Do not use abrasives, detergents or chlorinated products.

- Dry with a clean, dry soft cloth.
- In case of wear or tear, do not attempt to repair the protector: discard it or contact an authorised dealer to see if it can be taken care of by IDENTITES after-sales service.
- The cover can be washed at 95°C and dried at 60°C with standard non-chlorinated detergents

CHANGE OF PATIENT OR BED

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury
The product can be stored between 0 and 50 °C. Keep away from flames and sparks.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

TECHNICAL SPECIFICATIONS

Assembly: by stitching polyester - cotton thread

Filling: polyether foam 29 kg / M³ (± 10 %)

Cover: 25 % Polyester - 75 % PVC (This article contains biocides according to EU 528/2012 / protection against microbiological deterioration. Active ingredient : zinc-pyrithione)

Accessories: Polyacetal (POM) buckle - Polyoxymethylene - Strap 100% polypropylene

Overall Dimensions	823009 – 130 x 35 x 2 cm
Weight	823009 - 0,480 kg

DURATION OF USE

The expected lifetime of this product is 2 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 2 years from the date of purchase by the user. IDENTITÉS will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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