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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using an assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Java bath boards are assistive devices. These products are intended to be used by elderly or disabled people to wash and shower comfortably sitting at the edge of the bathtub. For ease of transfer and safety, this plastic bath board model comes with a handle. It provides a comfortable, contoured seating area for sitting while bathing, and has well-spaced holes for better drainage.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Please read carefully before use. Failure to comply may result in severe injury.
- Do not exceed the maximum weight limit of 100 Kg.
- Inspect all parts for shipping damage before assembly. IF there is shipping damage, DO NOT USE. Contact your dealer for further instructions.
- Ensure all fittings are tight & properly engaged in position prior to use.
- Before use check to make sure screws are tight and product is stable.
- Always, sit in the centre of the swivel bath seat, and do not lean on the edges of the bath seat as this can cause instability. DO NOT stand on the bath seat.
- If any parts are damaged, broken, worn or missing, contact the dealer.

USE

1. Loosen the lower mounting brackets on the underside of the product by turning the leg counter-clockwise. This frees the adjustment bracket to allow adjustment to a different size of bath. Reverse this action to tighten.

2. Slide the brackets until they point towards the centre of the board, then place the product on the bath ensuring there is sufficient width to rest safely on the edge of the bath (see FIG 1).

WARNING

To avoid the risk of tipping over, ensure that the anti-slip screws are securely fastened for height adjustment (see FIG 2).

3. Move the hooks outwards until they are firmly positioned on the edge of the bath on both sides. Tighten the bolts to secure.

4. Check that the board is secure and stable before using it.

5. The above assembly procedure must be carried out each time the bath board is placed on the bath to ensure positive positioning.

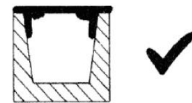
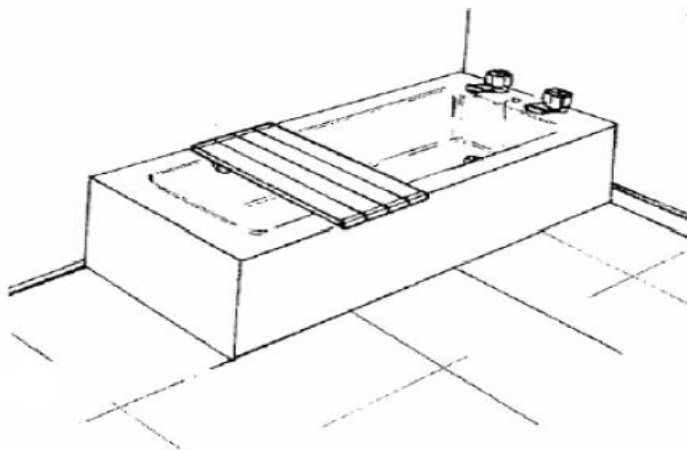
FIG.1

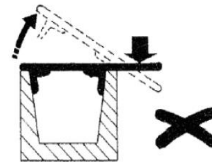
FIG. 2

Note: The 812275 turntable can be installed on Java and Java Plus bathtub tops, making it easier to get in and out of the tub.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry or shower and dry it after each use.
- Periodically clean the product with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant may be used. Rinse with warm water after cleaning. Do not use abrasive powders or scouring pads.
- Check the product for cracks and damage...
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :
To keep the product in good condition, check before each use, and check regularly as follows:

Check the product	Anytime	Every week
Check that the anti-slip screws are securely fastened for height adjustment	X	
Check the condition of the board and handle		X
Check that the brackets are fully attached to the edges of the cabinet	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

812176 :

Dimensions	Length 70 cm x Width 26/30 cm
Lenght adjustment	40.5/65 cm
Weight of the product	2.1 kg
maximum user weight	100 kg

812274 :

Dimensions	Length 75,5 cm x Width 26,5/35 cm
Lenght adjustment	45,5/68 cm
Weight of the product	2,3 kg
maximum user weight	100 kg

812275: Diameter 31.5 cm

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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