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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The black wheeled toilet seat is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, the disabled, the injured or those who can easily lose their balance.

PRODUCT DESCRIPTION

Number	Designation	Number	Designation
1	Right side frame	9	Potty and lid
2	Left side frame	10	Seat cushion
3	Connection tubes	11	Left footrest
4	Right armrest	12	Right footrest
5	Left armrest	13	Key
6	Backrest	14	Allen key
7	Wheels	15	Wheel Allen key
8	Seat assembly	16	E-clips

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 165 kg.
- Make sure all components are secure before use.
- Ensure that the product is in good working order.
- Visually inspect the product.
- Brakes and castors must be checked before each use.
- Check the stability of the chair before use.

- Use only on level ground.
- When stationary, remember to apply the castor brakes.
- Be sure to release the brakes before moving the chair again.
- For transfers or when the chair is stationary, also apply the brakes.
- Raise the footrests before sitting down or leaving the chair.
- Move the footrests out of the way before getting in or out of the chair. Always remove your feet from the footrest before folding it down.
- Never bend over while in the chair.
- Be careful not to pinch your fingers when moving or removing the potty.
- Make sure the potty rests securely on the two rails under the seat.
- Check castor bearings, brakes, footrest locking mechanism, seat and backrest cushions, armrest and backrest pads, potty and cover.
- If any parts are damaged, broken, worn or missing, contact your dealer.

ASSEMBLY AND USE

Assembling the chair frame

1. Take one of the side frames (items 1 and 2) with the disconnecting tubes (item 3).
2. Push the longer tube over the shorter of the two tubes protruding from the side frame, aligning the hole with the spring-loaded button.
3. Press the spring button to allow the disconnecting tube to lock fully in place.
4. Repeat for the short disconnect tube on the longer tube of the side frame.
5. Repeat the process with the second side frame.
6. When complete, the frame should appear as shown below.

Use of armrests

1. The armrests (items 4 & 5) can be removed for lateral transfer and replaced if necessary by the user.
2. To replace the armrests, push both ends of the armrest tube into the receivers on the frame.
3. Press the spring-loaded button at each end to lock the armrest into the frame.
4. To remove the armrests, simply reverse the previous instruction.

Installing the backrest

1. The backrest will slide easily into the tubes at the rear of the set. Once the holes are aligned, push the e-clips (item 16) through the holes until the clip locks onto the outside of the tube and the pin is visible on the opposite side of the tube.
2. Ensure that the backrest handle is facing away from the frame when installed.

Installing the wheels

1. Each castor (item 7) is supplied with a screw and nut in the heel of the castor.
2. Before attempting to install the castor to the frame, remove the nut and screw.
3. Once the nut and screw are removed, place the end of the castor into the upside down frame, ensuring that the holes in the heel and frame are aligned.

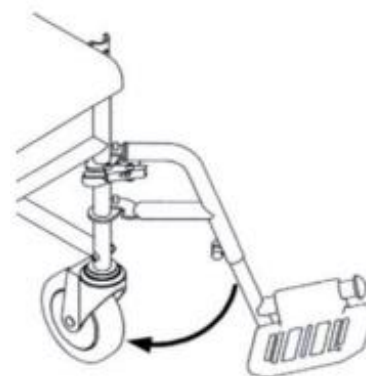
4. Push the screw through the frame and heel, then hand-tighten. Once finger tight, tighten the nut securely with the Allen Key (item 15) and spanner (item 13) supplied.
5. Repeat the above steps for the remaining three castors.

Footrest installation

1. To fit the footrests, insert the 2 tubing holes into the lugs when the footrest is turned outwards. Then rotate the footrest as shown in the diagram to lock it in use position at the front of the wheelchair.
2. To swivel and retract the footrest, press the chrome lever while pushing back the leg rest. The leg rest can then be lifted out of the way.

Wheel brake adjustment

Press the lever above the wheel to engage the parking brake (as shown). To release the lock, press the protruding tab above the wheel.

Seat installation

This product has a blow-moulded seat that clips easily onto the frame. The seat also has guide rails fitted for mounting the pot and cover underneath the seat.

Align the seat on the frame, so that the clips match the positions of the frame tubes.

Lower the seat onto the frame, ensuring that all the clips are secured and tightened onto the frame tubes. Once all the clips are secure, the pot can be slid onto the guides under the seat.

Using the pot and lid

- 1.The potty and lid are housed under the plastic seat. To add the potty, slide it under the seat from the back of the chair. Make sure the potty is securely fastened to the two rails below the seat.
- 2.To remove the potty, slide it under the seat. Add the lid and carry the potty securely using the chrome handle.

Beware of the risk of pinching when using

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry after each use.
- Periodically clean the frame, castors, joints and fixings with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant can be used.
- Dry with a soft, clean, dry cloth.
- Check the product for cracks and damage.
- The chair must be completely overhauled every year, with compulsory lubrication.
- We recommend servicing the product at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend more frequent servicing.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check condition of seat (no cracks)		X
Check brake operation	X	
Check folding function of armrests	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	width 54cm x depth 93cm x height 98cm
Seat Dimensions	width 44 cm x depth 46 cm x height 50 cm
Backrest height	42 cm
Armrest height	74 cm
Pot capacity	5 litres
Length between armrest	43 cm
Weight	14 kg
Maximum weight	165 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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