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## IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a healthcare professional before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

## PRODUCT DESTINATION

Hot and cold compresses are medical devices for the treatment and relief of knuckle and muscle pain. They are applied directly to the painful area. Cold packs provide long-term relief from moderate to severe pain. Warm compresses provide short-term relief for mild and chronic pain.

For cold therapy: Aches, pains and itching, before/after training, joint pain and sports injuries, swelling and sprains, fever, bruising or swelling, minor cuts and burns, after-sun care, headaches and migraines, nosebleeds, muscle aches and headaches, tired eyes, muscle pain and stiffness.

For heat therapy: Relief from fatigue, headaches and migraine, muscle pain and stiffness. Tired eyes, stomach aches or cramps, pain and itching. Before/after training, joint pain and sports injuries.

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Each time you use it, check the condition of the compress, especially if the envelope is not pierced.
- People who are allergic to the skin of this product and do not use it on damaged skin.
- Do not use on sensitive parts.
- This product is for external use, please do not swallow.
- Children need to be under adult supervision when using the product.
- Avoid use for those with terminal circulatory dysfunction and those without autonomy.
- Ask a doctor before use if you have diabetes, poor circulation or heart disease, rheumatoid arthritis or are pregnant.
- Take care when removing from the microwave, the pack may be very hot. If it is too hot for your skin, set it aside until cooled sufficiently for comfortable application.
- Always test the pack temperature before any application.
- To avoid frostbite, which could happen if the pack is too cold for your skin, set it aside until it reaches a temperature your skin can tolerate. If the pack is not cold enough, place in the freezer for an additional 20 minutes.
- Apply to affected area for not more than 20 minutes at one time, with 20 minutes breaks between applications.
- Notice heating longer than recommended may result in physical injury or damage to your microwave.
- Do NOT wear it while sleeping. Check skin frequently during use. If you find irritation or a burn, remove the product immediately.
- DO NOT use if cold hot pack's contents leak and/or wrap is damaged or torn. If cold hot pack's contents come in contact with your eyes, rinse affected area with clean water and seek professional assistance immediately.
- DO NOT apply with pain rubs, medicated lotions, creams or ointments.
- DO NOT use on unhealthy, damaged or broken skin or use hot on areas of bruising or swelling that have occurred within 48 hours or on areas of the body where you can't feel heat or cold or with other forms of heat or cold.
- DO NOT use on people unable to remove the product, including children, infants, and some elderly or on people unable to follow all use instructions.
- DO NOT place extra pressure over the product.
- DO NOT overheat.
- DO NOT swallow. Keep out reach of children and pets.

- Stop use and ask a doctor if pain worsens or persists for more than 7 days.
- If you have any discomfort, burning swelling, rash or other changes in your skin.
- Ask a doctor before use if you have diabetes, poor circulation or heart disease, rheumatoid arthritis or are pregnant.

## USE

### Cooling Instructions

1. Store the pack in the freezer with storage bag so that it is always ready for cold application. Refrigerate the pack for at least 2 hours before use.
2. Check to make sure the temperature of the pack is suitable before application.
3. If it is too cold, better wrap it in towel or any soft cloth before applying on skin.

### Hot Water Method

1. Bring water to boil.
2. Remove water from heat source.
3. Immerse the pack in water for approx. 5-7 minutes.
4. Check to make sure the temperature of the pack is suitable before application.
5. If it is too hot, better wrap it in towel or any soft cloth before applying on skin.

Note: For 835048, tap the tablet to diffuse the heat into the product. The product can be used for 30 minutes to 1 hour. To reuse the product, heat it for 1 hour in a water bath.

### Microwave Method

As microwaves vary in power, you need to find the correct heat time carefully. Optimum microwave is 800 watts.

DO NOT use microwave more than 1400 watt.

1. Flatten the pack and ensure the gel is evenly distributed, make sure it is in the center and not touching the inner sides of the microwave.
2. If the pack is at room temperature, heat it for 30 seconds on medium level. If more heat is needed, try a further 10 seconds at one time with equal standing until correct temperature is reached.
3. If the pack is frozen, heat it for 60 seconds on medium level. If more heat is needed, try a further 10 seconds at one time with equal standing until correct temperature is reached.
4. Check to make sure the temperature of the pack is suitable before application.
5. If it is too hot, better wrap it in towel or any soft cloth before applying on skin.
6. Record the heating time for future use.

To ensure a long service life, check the condition of the product, including the presence of holes, before each use.

## CARE

Failure to care for your product may result in the risk of serious injury.

- If the product is worn or torn, do not attempt to repair it: discard it or contact an authorised dealer to find out if it can be repaired by IDENTITES.
- The covers are machine washable at 40°C.
- The covers are hand washable with a mild detergent.
- Do not use alcohol or other solvents to wash the pad.

## CHANGE OF PATIENT

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

Keep away from flames and sparks.

Seal the package in a bag and store it in the freezer to extend its shelf life.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

## FEATURES TECHNICAL

Cold Hot Pack is made of outer pack with gel inside. The outer pack is plastic, polyester, or plush fabric. The inner gel is mixed with Water, Glycerol, CMC (Carboxymethyl Cellulose) or SAP (Super Absorbent Polymer), Preservative.

Overall Dimensions	835038 – 40 x 30 cm 835048 – 42 x 20 cm 835120 – 30 x 18 cm 835127 – 15 x 25 cm 835128 – 20 x 30 cm 835129 – 30 x 18 cm 835147 – 23 x 13 cm 835166 – 45 x 19 cm 835168 – 36 x 13,5 cm
Weight	835038 – 1,015 kg 835048 – 1 kg 835120 – 0.750 kg 835127 – 0.350 kg 835128 – 0.450 kg 835129 – 0.745 kg 835147 – 0.400 kg 835166 – 0.120 kg 835168 – 1.350 kg

## PERIOD OF USE

The expected lifetime of this product is 1 year under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for 1 year from the date of purchase by the user. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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