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INSTRUCTIONS MANUAL
EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Confort XXL shower chair is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, disabled, injured or who can easily lose their balance.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 200 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Check the stability of the chair before use.
- Only use on a level ground.
- Check the chair for wear and tear
- Do not lean out of the device to avoid rocking.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING

Height Adjustment

1. To adjust the stool to the desired height, firstly remove the plastic clips connecting the stools' legs and leg extensions.
2. Slide the leg extensions to the required position ensuring that all legs are equal in height.
3. Replace the plastic clips ensuring that each clip is fully connected.

USE

The Confort XXL shower chair slopes forward to allow the user to achieve a relaxed "perching" position when sitting and also to assist the user in rising. The chair may be used in the shower.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The product can be cleaned in an aqueous decontaminant machine at 80°C.
- The chair must be completely overhauled every year, with compulsory lubrication.
- It is recommended that the device is serviced at least annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist

To keep the product in good condition, check before each use, and check regularly as follows:

Type of control	Method	Frequency
Seat and backrest control	Adjustment/visual control	Before every use
Rubber feet control	Adjustment/visual control	Before every use

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 62-69 cm x Depth 46-52 cm x Height 80-95 cm
Seat Dimensions	Width 49 cm x Depth 42 cm x Height 47-61 cm
Width between Arms	60 cm
Backrest height	35 cm
Maximum Weight authorized	220 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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