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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Tobago folding shower chair with wheels is a medical device. It is a technical toilet aid for elderly, people with reduced mobility, disabled, injured or who lose their balance easily.

SAFETY PRECAUTIONS

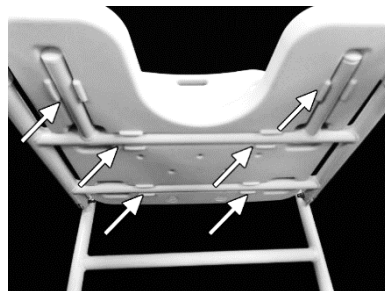
Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 110 kg
- Ensure that all components are secured before use.
- Ensure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and castors should be checked
- Check the stability of the chair before use.
- Use only on level ground.
- When stopped, remember to apply the wheel brakes.
- Be sure to release the brakes before moving the chair again.
- When transferring or when the chair is stationary, also apply the brakes.
- Never bend over while in the chair.
- Check wheel bearings and brakes
- If any parts are damaged, broken, worn or missing, contact your dealer.

MOUNTING & USE

MOUNTING

- 1 - Remove all components from the packaging and check the contents: frame (x1), seat (x1), adjustable wheels (x4).
- 2 - Unfold the chair frame.
- 3 - Place the seat on the frame and press firmly so that the seat tubes fit into the seat housings.

USE :

Before each use, make sure the chair is properly assembled, especially the locks on the seat and the 4 wheels.

When using with wheels, the chair is locked by pressing the levers as shown opposite. Unlocking is done by pulling up the levers.

Non-braked wheel

Braked wheel

WARNING : always lock the brakes (rear wheels) for static use and when moving from a sitting to a standing position. The Tobago Chair should always be used under the supervision and assistance of a caregiver when used by an invalid.

After use, the Tobago Chair can be stored folded. To do this, disengage the seat and fold it down, then fold the frame as shown opposite. **Beware of pinching risk when folding.**

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair should be serviced every year completely, with mandatory lubrication.
- Lubricate the wheels with a little oil if they squeak.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- In case of malfunction, do not repair the product yourself. Consult an authorized dealer.
- Checklist :

To ensure optimal product life, check before each use and verify as follows:

| Type of control | Method | Frequency |
|---------------------------|---------------------------|------------------|
| Seat and Backrest control | Adjustment/Visual Control | Before every use |
| Fastenings control | Adjustment | Once a month |
| Wheels Control | Adjustment | Once a month |
| Brakes control | Adjustment | Before every use |

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).

- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

| | |
|------------------------|--|
| Overall Dimensions | Width 53 cm x Depth 50 cm |
| Seat Dimensions | Width 41 cm x Depth 40 x Height 46 / 53 cm |
| Width between armrests | 48 cm |
| Weight | 7,5 kg |
| Max. user weight | 110 kg |

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from who you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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