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INSTRUCTIONS MANUAL
EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The enveloping shower chair is a medical device. It is a technical aid designed for showering, intended for elderly people, those with reduced mobility, disabilities, injuries, or who are prone to losing their balance. The chair can also be used for daily activities. It features a sturdy plastic seat with perforations to allow for quick water drainage. The armrests provide additional support. Designed as a temporary seating solution, this chair should not be used as a step stool or transfer device. It is suitable for use in showers, bathrooms, or other wet rooms, but is not suitable for use in swimming pools.

PRODUCT DESCRIPTION

PRECAUTIONS FOR USE

Failure to follow these precautions may result in serious injury.

- Never exceed the maximum weight capacity of 160 kg.
- Ensure the product is in proper working condition.
- Visually inspect the product before each use.
- Check the stability of the chair before use.
- Use only on a flat surface to prevent tipping.
- Check for wear before each use (fastening components, tubes, backrest, seat, etc.).
- Do not lean while seated on the chair.
- If any parts are damaged, broken, worn, or missing, contact the retailer.
- When moving the chair, always lift it by holding the frame located under the seat.
- Do not allow children or unauthorized persons to play with or use the chair without supervision.
- Avoid rocking forward, backward, or side to side while in use, as this may cause excessive stress, structural damage, or injury.
- Do not use the chair outdoors or on uneven surfaces (such as in a garden) to avoid instability.
- Never place your fingers or objects inside exposed tubular sections, as this may cause serious injury.

USE AND ADJUSTMENT

Safety Instructions :

- Unpack and carefully inspect the product to ensure that all components are present.
- Before each use, make sure there are no signs of damage, such as a cracked weld, chipped paint, damaged seat, or

deformed tube.

- The seat must be securely attached to the frame with the supplied screws before use.
- The frame is marked “front” and “back” to indicate the correct orientation (the highest crossbar is at the front).
- Do not attempt to use the product if it shows any damage.

Height Adjustment :

1. To adjust the height, remove the E-clips from the extension legs, then extend each leg evenly to the desired height, ensuring that you do not exceed the last adjustment hole.
2. Once the correct height is set, reinsert the E-clips ensuring they pass through the extension leg and are securely positioned in the outer tube

Do not attempt to level the seat by adjusting the legs individually, as this would place excessive stress on the structure. This could compromise user safety and cause irreversible damage to the chair, including cracking of the tubular sections or other components.

Use :

Ensure the product is properly assembled and that all components are securely fastened before use. Sit down in a stable manner on the seat and use the product only on a flat, non-slip surface. Avoid sudden movements that may cause loss of balance. If you feel discomfort or notice any instability, stop using the product immediately.

CARE - MAINTENANCE

Failure to properly maintain and care for your product may result in a risk of serious injury.

- Clean the product regularly (frame, fittings, pads, seat) with a mild detergent and warm water, then wipe the frame with a soft, clean, and dry cloth at least once a week, or more often if necessary.
- For disinfection, use a chlorine-based solution, taking care to rinse and dry thoroughly afterwards.
- Do not use abrasive or solvent-based cleaning products.
- Before cleaning the chair, please remove all E-clips as well as the four legs. Clean these parts separately from the main frame.
- The legs and fixing clips can be removed to facilitate cleaning. This feature offers an advantage over fixed shower chairs, as it allows access to all surfaces for thorough cleaning.
- If the product is subject to heavy use and/or use in a multi-user environment, we recommend having it serviced periodically. In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.
- Checklist:

To keep the product in good condition, check before each use, and carry out regular inspections as follows:

Type of Inspection	Method	Frequency
Seat and backrest inspection	Visual check	Before each use
Legs and “E” clips inspection	Adjustment/Visual check	Before each use
Pads inspection	Visual check	Once a week

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene guidelines. When transferring the product, make sure to provide all the necessary technical documents to the new user. Before reuse, the product must be inspected by an authorized specialist.

RECYCLING

When the product becomes unusable and needs to be discarded, please contact your specialized retailer. If you wish to handle the recycling yourself, contact your local waste management company for disposal instructions.

STORAGE

Failure to comply with storage conditions may cause product deterioration and result in serious injury.

Do not store the product for a long period near a heat source or in direct sunlight (e.g., behind a window or near a radiator), or near a source of cold.

Keep away from flames and sparks.

Observe the following conditions when storing your product.

- Store in a dry, temperate location.
- Protect the product from dust and corrosion (e.g., abrasive elements, sand, seawater, salty air).
- Store all removed parts together in the same place (or label them if necessary) to avoid mixing them with other products during reassembly.
- All components must be stored without any load — do not place heavy objects on product parts or wedge anything between them

SERVICE LIFE

The expected service life of this product is 5 years under normal conditions of use, safety, and maintenance for a single user. Beyond this period, the product may continue to be used as long as it remains in good condition. If the product is used in a multi-user environment (such as a nursing home or hospital), its service life may be reduced.

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 62 × depth 46 × height 77 to 91 cm
Seat dimensions	Width 44 × depth 39 × height 44.5 to 59 cm
Width between armrests	50 cm
Maximum user weight	160 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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