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---

## **IMPORTANT**

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

## **PRODUCT DESTINATION**

Travelite transfer chairs are walking aids for people with reduced mobility. These products are designed for both indoor and outdoor use and fold up for easy storage or transportation.

## **PRODUCT DESCRIPTION**

The transfer chair consists of a frame, seat, backrest, armrests, footrests, front and rear wheels and brakes.

**Figure 1**

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 125kg.
- It is strictly forbidden for the user to modify the transfer chair without the agreement of the technician. For maximum stability, the rear wheel axle must be fixed in the rearmost position. Any movement of the wheel axle will affect the stability of the transfer chair when moving. If the transfer chair's rear wheel axle is moved forward, please check that it is stable before using the transfer chair. Before tilting the transfer chair seat cushion to the limit or moving the rear wheel axle forwards, the stability of the transfer chair must be tested. To ensure safety, users are advised to check the stability of the transfer chair carefully before using it.
- Do not use both hands to exert pressure on the sides of the frame or handle when folding. Improper folding or opening of the transfer chair can easily cause damage.
- The brakes on both sides of the transfer chair are for braking only when parked. They are not braking devices. They must not be used to decelerate or brake when travelling. If you are unaccompanied, you can hold the steering wheel and brake gradually.
- Do not rely on inertia to get the transfer chair up the steps, as this could cause an accident or damage the transfer chair.
- Do not go up or down stairs unaided. When going up and down stairs, the person accompanying you must not move the occupant at the same time as the transfer chair, as this could endanger the occupant and damage the transfer chair.
- If any parts are damaged, broken or worn, contact your dealer.
- Unless specifically instructed otherwise, all transfer chair adjustment operations must be carried out with no-one in the transfer chair.
- When you are sitting in a transfer chair and you want to pick up objects other than the transfer chair (such as the front, sides or back of the transfer chair), make sure that your body is not tilted too much to prevent the transfer chair from losing weight and tipping over.
- Before using the transfer chair, check that the armrests, handles, fixing screws, etc. are not loose. If this is the case, please contact a professional technician to rectify the problem before using the transfer chair normally. Before using the transfer chair, please practise using it with an accompanying person.
- It is strictly forbidden for the user to perform a balancing act in which the front wheel is lifted off the ground or both front wheels are lifted off the ground, so as not to endanger others or themselves. Do not bend below the knees to pick up objects from the ground.
- Do not change your sitting position when leaning back, otherwise the transfer chair will tip over.
- The transfer chair must not be used on the escalator or stairs, as this could cause serious injury.
- When a person in a transfer chair wishes to transfer from one transfer chair to another, they must first park it in the nearest and most suitable place, then turn the front wheel of the transfer chair towards the place where they are to transfer. Please check that the transfer chair is braked before using it.
- When transporting the transfer chair, please grasp the fixed part of the transfer chair and not the soft parts, such as the armrests and footrests.
- Do not step on the pedal to get in and out of the transfer chair; be sure to fold or roll the pedal to the side first. The user must not stand on the footrest during use to prevent the transfer chair from tipping over.
- When using a transfer chair on a sloping road, do not tip the transfer chair over or change direction abruptly. Do not brake suddenly when going downhill to avoid the risk of falling forwards.
- It is advisable to drive on level ground when using a transfer chair, and not to drive on uneven roads. When you encounter obstacles, you should take a diversion to avoid the risk of the transfer chair tipping over. When the user drives the transfer chair themselves, they must maintain a constant speed, and the driving speed must be kept at 3~5km/h.
- A slope of less than 5° is the safe driving area for the transfer chair, and steeper slopes require the assistance of other people. When getting on and off the transfer chair, maintain the corresponding front and rear positions and keep the backrest upright on the slope. Long-term users should wear protective gloves to prevent hand injuries.
- Do not put your fingers in the wheel to avoid pinching your fingers.

- When used for patients, routine cleaning is carried out once a day. When replacing infected patients, disinfectants must be used for complete disinfection.

- Driving restrictions:

1) When the transfer chair is driven on sloping ground, it should be driven when the user is in good physical condition and the angle of inclination of the ground is less than 10°. When the angle of inclination is greater than 10°, other people must help to drive behind the wheel.


2) Do not drive up or down steps or stairs unassisted.

3) It cannot be used in bad weather, such as heavy rain, sandstorm, strong wind, etc.

4) This transfer chair is suitable for use indoors and outdoors, on grass or flat ground. It is not suitable for long-distance driving on uneven and irregular roads. If you wish to travel long distances, you should be accompanied by someone. If you have to negotiate an obstacle no higher than 12 cm, those around you should hold the handles firmly with both hands and step on the transfer chair's lower rear tube to lift the front wheel over the obstacle. If you are forced over the obstacle, the transfer chair will be damaged.


## MOUNTING & USE

- When you open the transfer chair, you can press down on both sides or just one side of the seat, making it easy to open; when you fold the transfer chair, please raise the footrest first, then hold the cushion and lift it, making it easy to fold. (See figure 2)

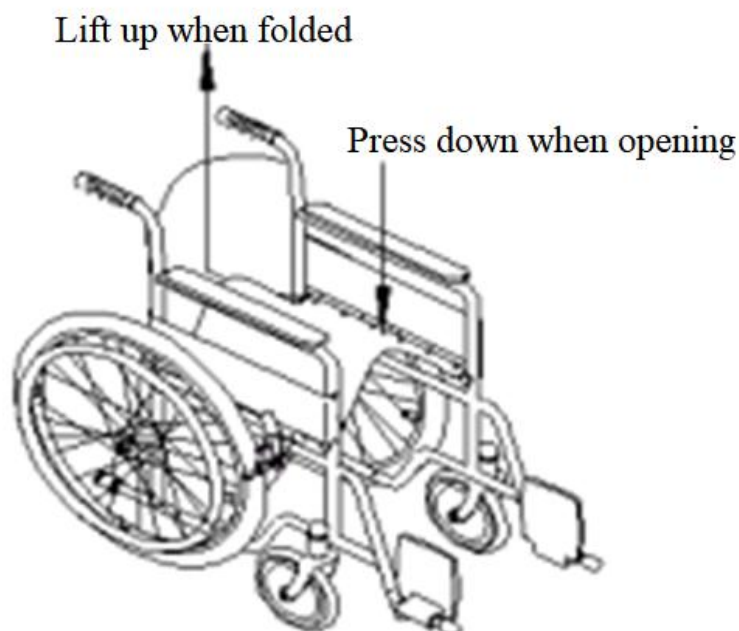
**Figure 2**

- There are brakes on both sides of the seat. Be sure to use the brakes when parking the transfer chair. (See figure 3)

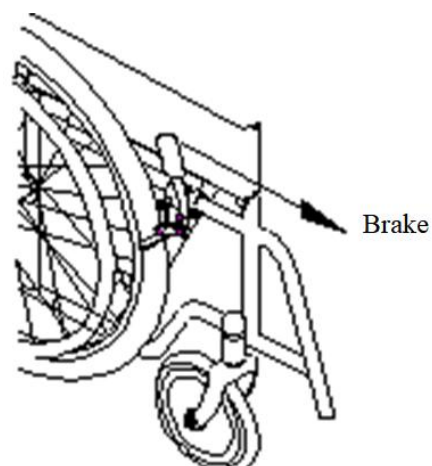**Figure 3**

- There is a rear-tilt bar with a plastic sleeve at the bottom of the transfer chair. The follower steps on the tipper bar to tilt the transfer chair backwards - the front wheel first goes up the steps, then easily lifts the transfer chair with the passenger up the stairs. (See figure 4)

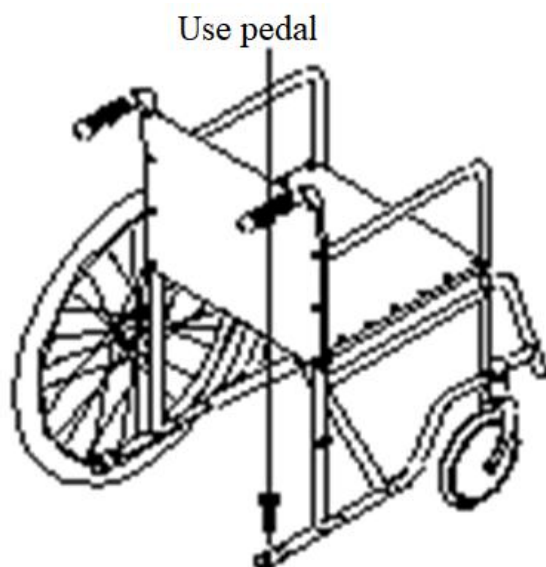
**Figure 4**

- When you have to go up and down steps, please help the occupants up and down the steps first, then move the transfer chair onto a flat road before driving.
- If you have purchased a transfer chair with the following features, please follow the instructions below.
- If your transfer chair is fitted with a removable leg support, pull the locking handle on the leg support when removing it, the leg support can pivot outwards, and after a certain angle, it can be lifted up and out.
- When installing, align the positioning hole of the leg support with the positioning pin of the frame and suspend it, then gently rotate it inwards until the handle is automatically locked and secured. (See Figure 5)


**Figure 5**

- If your transfer chair is fitted with a removable armrest frame, press down on the steel ball of the armrest frame when disassembling to release the steel ball from the hole in the handrail, and the armrest frame can be removed; insert the armrest frame when installing on the chassis, the steel ball may be jammed in the hole in the armrest frame. (See figure 6)

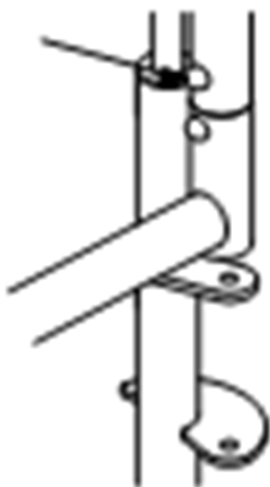
**Figure 6**

- If you purchase a transfer chair with a type A flexible leg support, simply lift the leg support to a suitable position when raising the leg support, then secure it by itself; when lowering, support the leg support slightly with one hand, the other hand moves the lifting handle forward, the leg support can be lowered, and once it is in a suitable position, the lifting handle can be released to secure it by itself. (See Figure 7)

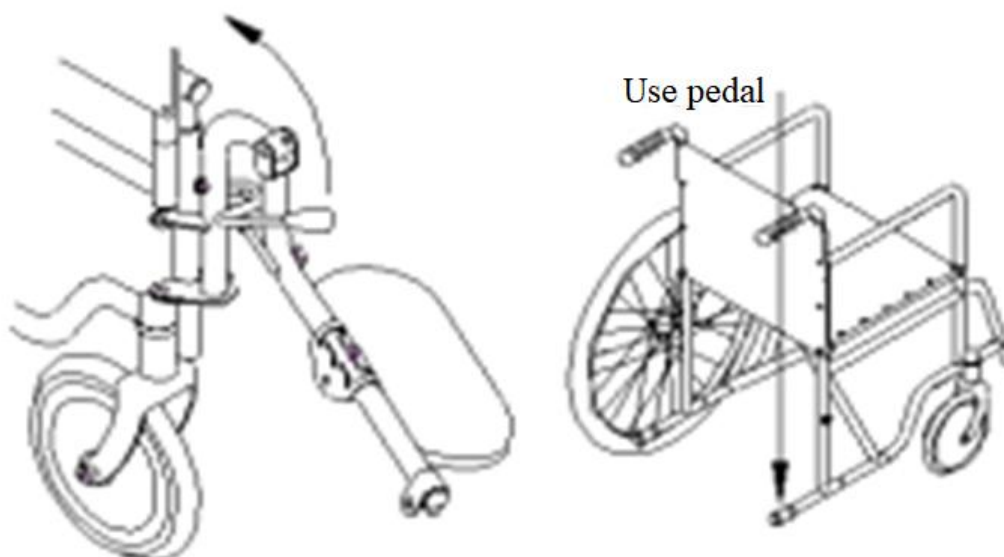
**Figure 7**

- If you buy a transfer chair with an E-shaped folding rear armrest, support the rear handle lightly with one hand when folding it, and pull the folding handle backwards with the other hand until the pin is released from the frame and the rear handle can be folded down; when refitting, simply rotate the handle upwards to the refitted state to lock and secure it. (See figure 8).

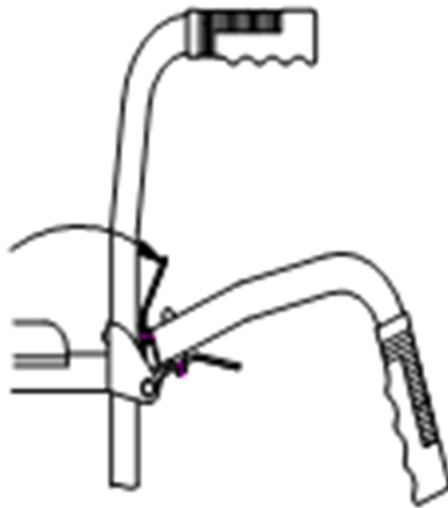
**Figure 8**

- If you choose a transfer chair fitted with a rear tilt armrest, hold the front end of the armrest as you tilt it backwards, and lightly push the armrest locking handle with your index finger, and the armrest can be tilted backwards; when the armrest is restored, tilt it forwards, the armrest lock is inserted on the jack in the frame, and the handle is automatically reset and locked (see Figure 9).

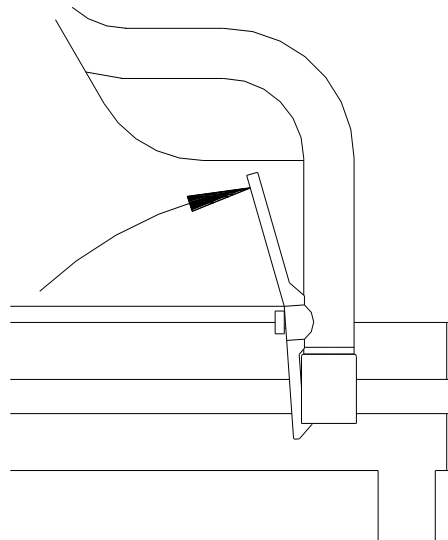**Figure 9**

- If you are buying a transfer chair with a detachable aluminium alloy leg support, turn the leg support to lock the handle when dismantling, the leg support can pivot outwards and can be lifted up after pivoting through an angle; when inserting, align the upper and lower locating holes of the leg support with the locating pins on the frame and hook it in, then gently pivot it inwards until it automatically snaps back into place and secures. (See Figure 10)

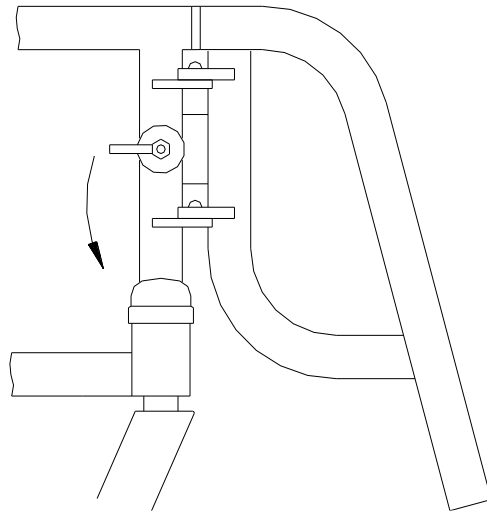**Figure 10**

## CARE AND MAINTENANCE

**Failure to care for and monitor your product may result in the risk of serious injury.**

- Clean the chair regularly using lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Make sure there is no build-up of dirt or debris around the axle or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check all screws and bolts for wear and strength. Replace all broken, damaged or worn parts immediately.
- In the event of a malfunction, do not repair the transfer chair yourself. Consult an authorized dealer.

- **Checklist :**

To keep the product in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly
Setting elements		X	
Structural weld at wheels		X	
Cleanliness	X		
Screws and fasteners		X	
Stability of frame and backrest			X
Condition of seat			X
Brakes	X		

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

**Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.**

Do not store your product for a prolonged period near a source of heat or in direct sunlight (e.g. behind a window or near a radiator) or near a source of cold.

The chair should be stored in a ventilated and clean room, at a temperature between -10 °C and 40 °C and at a relative humidity not exceeding 80%.

Keep away from flames and sources of sparks.

Observe the conditions for storing and storing your product:

- In a dry, temperate place
- Protected by packaging, from dust and corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together, in the same place (or mark them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place heavy objects on product parts, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 84 cm x Depth 57 cm x Height 95 cm
Folded dimensions	Width 77 cm x Depth 29 cm x Height 29 cm
Seat dimensions	Width 45 cm x Depth 45 cm x Height 48 cm
Weight	9.3 kg
Maximum load	125 kg

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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