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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT DESTINATION

Swivel bath chairs are technical aids. They are intended to be used to facilitate access to the bathtub for people with reduced mobility. This rotating shower seat is designed to enable a user of limited mobility to rotate, whilst seated, into the bath to allow seated washing. It is suitable for most standard baths but is not suitable for corner baths, baths which have uneven/ profiled top edges or some baths with deeply molded side panels. The frame can be adjusted to fit most common bath sizes.

## PRODUCT DESCRIPTION

1. Backrest
2. Armrest
3. Lever of seat rotating
4. Adjustable screw of fixing width in bathtub
5. Tip
6. Seat

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 130 Kg.
- Inspect all parts before assembly to ensure that they are not damaged during shipping. If there is shipping damage - DO NOT USE. Contact your dealer for further instructions.
- Always sit in the centre of the swivel bath seat and do not lean on the edge of the bath seat as this may cause instability. Do NOT stand on the bath seat.
- If any parts are damaged, broken, worn or missing, contact your dealer.
- Do not overtighten the adjustment screws
- Ensure that the anti-slip supports are securely attached to the bath

## ASSEMBLY

### Assembly and Installation

Remove all parts from the packaging and check they are not damaged. If you see, or suspect a fault, do not use, contact your dealer immediately.

### Product Assembly

Slide each armrest into the locating tubes on the seat assembly until the spring button clips “snap” into position. Slide the mounting frame into the locating tubes on the seat assembly, note the frame is width adjustable, and choose the correct width to suit your bath. Please refer to the instruction for correct fitting.

### Assembling to a bath

Place the seat onto flat bath ledges.

\*NOTE: All the mounting frame feet must rest on the flat sides of the bath.

If necessary extend the mounting frame by removing the fixing e-clips, adjusting the width of the frame so that it is closest to the external width of the bath and replacing the clips, check they are all replaced and secured correctly.

Secure the mounting frame using the clamps under the seat by turning the hand wheels until secure against the walls of the bath on both sides.

Check the frame is secure, but do not over tighten as this could damage the bath.

\*NOTE: If the seat is removed and replaced over the bath, it is important the fixing clamps are re-adjusted and tightened as detailed above.

## USE

1. Raise the locking handle and rotate until the seat is facing the user, lower the handle and lock into position.
2. Transfer user into seat, raise locking handle and rotate slowly into bath, taking care when raising legs over the side of the bath.
3. Ensure seat is locked into position and no longer rotates freely before starting to wash.

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry after each use.
- Periodically clean the bath seat with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant can be used.
- The bath seat can also be cleaned in an aqueous system up to 80°C.
- Check that the bath seat swivels easily, smoothly and locks into place. If the locking mechanism does not seem positive or if there is excessive movement in the unit, remove it from service and contact your dealer.
- Check the bather for cracks and damage, particularly to the seat and plastic tube.
- Do not use in environments above 60°C (e.g. saunas).
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check the operation of the spring loaded knob to allow 90° rotation of the product	X	
Check the condition of the seat and the armrests		X
Check the function of the anti-slip screws	X	
Check that the brackets are fully attached to the edges of the bathtub	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Fits standards baths with	Maximum interior width 66 cm
Overall dimensions	Width 71 x depth 54 cm x height 57 cm
Seat Dimensions	Width 41 cm x depth 34 cm
Structure	Aluminium
Weight	4,9 kg
Maximum weight	130 kg

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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