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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Aston shower chair is a medical device. It is a technical toilet aid for elderly, people with reduced mobility, disabled, injured or who lose their balance easily.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 160 kg for the references 822165 – 822253, 260 kg for the reference 822178 and 300 kg for the reference 822272.
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and wheels should be checked.
- Check the stability of the chair before use.
- Only use on level ground.
- When stationary, apply the brakes on the wheels.
- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary and the brakes are applied when transferring to and from the device.
- Do not lean out of the device.

- Check the wheels bearings the brakes
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

1. Insert the backrest into the frame tubes
2. Lock the backrest in place with the knobs on the back of the seat

3. Position the footrests in the footrest socket on the top of the base frame. Turn the footrests towards the front of the chair until they lock into place. You can adjust the height and angle of these footrests according to the instructions below.

4. If the wheels are not mounted on the frame, install them according to the adjustment instructions below.
5. You can also install the seat by fixing it with the clips on the frame and also position the bucket and the lid.

➤ Footrest Operation and Adjustment :

Swinging footrests In and Out

The footrests can swing in and out if the user requires this. If required, first flip up the footplates. When required, rotate the footrests to the forward position until they automatically lock in position. Flip down the footplates if required.

Footrest installation and removal

Follow the assembly instructions for details on how install the footrests. To remove the footrests, swing out the footrest and lift up to remove.

➤ Footplate adjustment :

1. To adjust the height of the footplate, slacken the bolt at the end of the white tubing of the legrest, using an allen key and turning the bolt anticlockwise. Slide the footrest to the required height and secure the footrest by tightening the bolt. Tighten the bolt by turning clockwise with an allen key.

2. To adjust the footplate angle, slacken the bolt near the bottom of the inner tube (chrome). Adjust the footplate to the required angle and resecure the bolt.

➤ **Castor operation :**

1. The castors can be locked so that the wheel cannot move forwards or reverse nor will the castor fork swivel. To lock the castor press down on the locking tab with a foot (as shown).

2. To release press the other side of the locking tab with a foot. Unless the chair is fitted with 24" wheels, it is recommended that the castors are always locked when the chair is stationary.

➤ **Castor adjustment :**

The height of the chair and seat can be altered by adjusting the castor height. To adjust the castor height, remove the 'E' clip or nut and bolt (arrowed). Reposition the castor stem to the required position and reattach the 'E' clip or nut and bolt. Repeat for the 3 castors ensuring they are all set at the same height.

Note : If the chair is fitted with the 24" rear wheels, the axle bracket height should also be altered so that the rear castors are nearly touching the ground when the 24" wheels are fitted. Models will be supplied with either 'E' clips or a nut and bolt to secure the castor. 'E' clips do not require any tools for height adjustment, whereas the nut and bolt solution requires a spanner and allen key.

➤ **Potty installation and removal :**

The potty is installed and removed by sliding the potty in to the two rails mounted to the underneath of the seat. To install, slide the potty underneath the seat from the rear of the chair, ensuring the potty sits on both rails of the seat. Ensure nobody is sat on the chair when installing or removing the potty to prevent any tapping risk.

Cover the potty with lid to reduce the risk of spillages and carefully dispose of the potty contents in a sluice or WC. Do not dispose of in an open drain or sink as this can cause serious health risks.

The potty can be cleaned and disinfected manually or in an autoclave/aqueous decontamination machine. It should be cleaned after each use.

➤ **Seat operation en removal :**

The seat is a horseshoe shape with removable cover for the aperture. When used as a shower chair, it is recommended that the aperture cover is removed to allow better access to the occupant for cleaning. Ensure weight is not applied directly to the aperture cover as this can fall through the seat. The seat clips on and off the frame. To remove the seat for cleaning, first remove the potty and aperture cover. Then lift up the front of the seat to release and slide the seat away from the chair. To attach the seat first locate the seat by sliding the back of the seat into the frame, then press down on the whole of the seat until secure.

➤ **Arm operation :**

The arms can flip up to aid side transfer. Simply lift the front of the arms to flip up, and then push down to lower. The brakes should always be applied before transferring.

➤ **24" Rear wheel kit - Installation :**

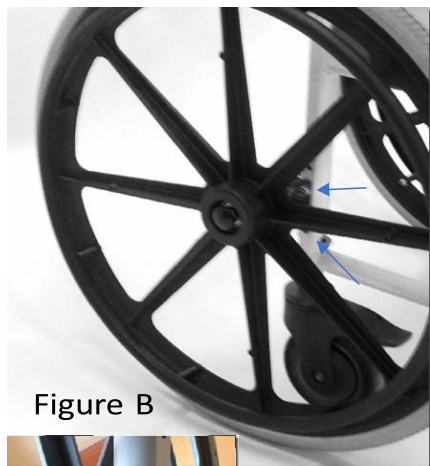
Figure B

Place the axle support on two of the holes at the required height, ensuring that the axle points downwards (as shown in Figure B). The brackets can be forward or rearward facing, depending on the user's needs. Insert the two screws into the holes in the bracket and frame. Secure by attaching the nuts shown in figure B.

➤ **Installation of 24" rear wheel brakes :**


Figure C

Before installation, loosen the two screws securing the brake clamp (shown in Figure C.) Position the brake clamp on the top horizontal bar of the frame. Slide the brake assembly into the approximate position shown in Figure C. Tighten the two fixing screws to secure the brake. Check that the brake is in the correct position by the following checks:

- i. When released, the brake does not interfere with the tyre
- ii. When applied, the brake stops the wheel
- iii. The user can easily apply the brake
- iv. When applied, the brake is lower than the seat.

This is important to avoid injury to the user during lateral transfer. If the position is incorrect, loosen the two screws on the brake collar, reposition, retighten the two screws and recheck.

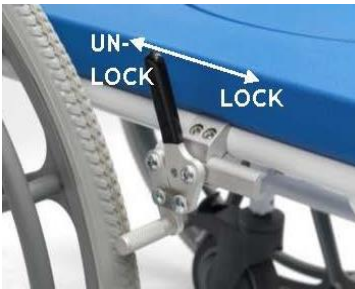➤ **Brake operation :**

Push the brakes forward to lock the wheel. Pull the brakes back to release the brake. Ensure both brakes are applied when the chair is stationary and when the user is side transferring.

➤ **Operation – 24" rear wheels :**

When the 24" rear wheels are installed, the chair can be propelled by either the attendant or the chair occupant. If the attendant is propelling the chair, they should propel the chair using the push handle at the rear and apply the castor brakes when the chair is stationary. If the occupant is propelling the chair, they should use the handrims mounted on the side of the wheels to go, manoeuvre and stop and then apply the brakes (above) when the chair is stationary.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
 - Worn tyres, wheel bearings, wheels, brakes, leg rest locking mechanism, seat and back cushions, armrest cushions, quick release pin, pot and lid, "E" clips) should be repaired, replaced, adjusted and/or lubricated if necessary.
 - In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
 - Checklist :

To ensure optimal product life, check before each use and verify as follows:

Type of Control	Method	Frequency
Seat and backrest control	Adjustment/Visual Control	Before every use
Fastenings control	Adjustment	Once a month
Wheels control	Adjustment	Once a month
Brakes control	Adjustment	Before every use
Armrests and legrests control	Adjustment	Once a month

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Aston shower chair - 822165	
Overall dimensions	Width 56 cm x Depth 87 cm x Height 99 / 104 cm
Seat dimensions	Width 46 cm x Depth 43 cm x Height 51 / 54cm
Backrest height	47 cm
Seat height	43 / 48 cm
Dimensions of the aperture	27 x 17 cm
Weight	13,4 kg
Maximum supported weight	160 kg
Aston shower chair 24''- 822253	
Overall dimensions	Width 74 cm x Depth 112 cm x Height 99 cm
Seat dimensions	Width 46 cm x Depth 43 cm x Height 50 cm
Backrest height	47 cm
Seat height	44,5/47/49,5 cm
Dimensions of the aperture	27 x 17 cm
Weight	16 kg
Maximum supported weight	160 kg
Aston XL shower chair - 822178	
Overall dimensions	Width 66 cm x Depth 87 cm x Height 99 / 104 cm
Seat dimensions	Width 55 cm x Depth 43 cm x Height 51 / 56 cm
Backrest height	47 cm
Seat height	43 / 48 cm

Dimensions of the aperture	27 x 17 cm
Weight	17 kg
Maximum supported weight	260 kg
Aston XXL shower chair - 822272	
Overall dimensions	Width 80 cm x Depth 87 cm x Height 99 / 104 cm
Seat dimensions	Width 60 cm x Depth 43 cm x Height 51 / 56 cm
Backrest height	62 cm
Seat height	43 / 48 cm
Dimensions of the aperture	27 x 17 cm
Weight	19 kg
Maximum supported weight	300 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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