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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Comfort XXL toilet chair is a medical device. It is a technical aid for toileting that is designed for people who are elderly, have reduced mobility, are disabled or injured, or who can easily lose their balance. It allows comfortable positioning and WC use for strong people. It has removable armrests for easy transfers and a wide seat. Removable 14-litre capacity potty. Height-adjustable seat.

PRODUCT DESCRIPTION

Check that all parts are present.

- A) 1 x PU seat and seat frame
- B) 1 x Backrest c/w PU back pad
- C) 1 x pair of arm rests
- D) 4 x Adjustable height legs
- E) 1 x Large Potty
- F) 1 x Potty Lid

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 280 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Check the stability of the chair before use.
- Only use on level ground.
- Before each use, check that the 4 legs are at the same height and that the clips are fully extended.
- Never bend over when using the chair.
- Beware of the risk of pinching when using the chair.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

INSTALLATION :

Step 1 :

Place the seat & frame on a flat surface with the two tubes facing backwards.

Take the backrest & place the tubes into the tube holes in the seat frame, pushing each tube in firmly **ensuring that the spring pins click into the holes securely.**

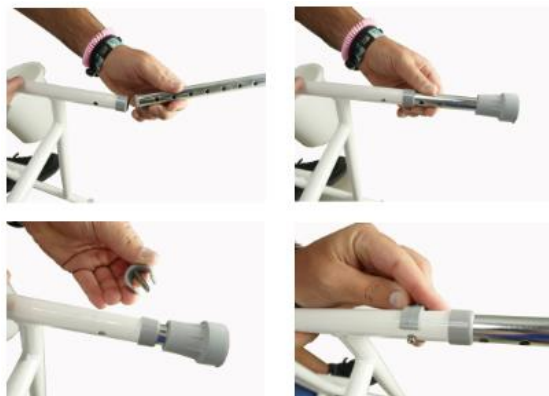
Step 2 :

Fit the arms into each side of the stool. Press the spring pin in at each end & push the tubes into the sockets on the frame. Ensure spring pins fit securely as with the back rest.

To remove the arms, press the spring pin inside the tube & gently lift the armrest until the spring pin disappears. Do the same at the other end & then pull the armrest out of the sockets.

Step 3 :

Insert one of the legs into one of the tubes on the lower section of the frame.

When the leg is seated properly the holes will line up on both tubes. Once you have the height you require, push the E-Clip through both tubes, locking the leg tube in place.

Fit the other three legs ensuring that they are all set at the same height and stand the stool upright

Step 4 :

Push the potty under the chair from the front. There are some guide bars fitted to the frame that will fit underneath the rim of the potty. Push the potty as far back as the guide bars will allow. Place the lid of the potty into the rack that is fitted to the back of the chair.

USE :

- When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.
- Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.
- Do not use the perching stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with a damp cloth and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Before cleaning the chair, remove the two armrests, the backrest and the four extension feet with the end caps. Clean these parts separately from the main frame.
- The chair must be completely serviced every year, with compulsory lubrication.
- It is recommended that the product be serviced at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it be serviced more frequently.
- Before each use, check the condition of the rubber feet and seat.
- In the event of a malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 66 x depth 54 cm. Width between armrests 61 cm
Seat dimensions	Width 61 cm x Depth 43 cm x Height 43/58 cm
Weight	13 kg
Maximum user weight	280 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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