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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.


PRODUCT DESTINATION

The Resist bed support is a medical device (MD) designed to secure a bed and prevent falls, particularly for elderly or disoriented individuals. It is fixed between the mattress and the bed base to facilitate getting in and out of bed. This MD is ideal for use at home or in care facilities, providing a practical and safe solution.

PRODUCT DESCRIPTION

- Carefully unpack the product.
- Check that all parts are present (see contents list below).
- Check for any damage.
- If you notice or suspect a fault, do not assemble or use the product. Contact your supplier immediately.

Number	Item	Quantity
1	E-clip	2
2	Bed rail board	1
3	Hand rail	1
4	Bed straps	2

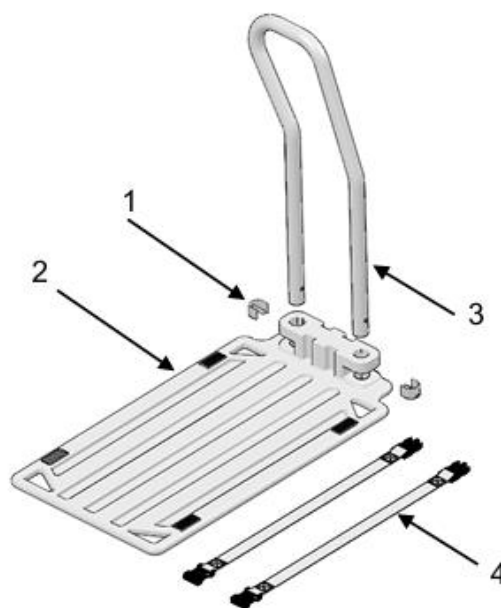
SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Make sure all parts are securely fixed and in good working condition. Use only the accessories and parts supplied by the manufacturer. Do not make any adjustments to the hand rail while it is in use.
- If any parts are damaged, broken, or worn, contact the supplier.
- Check the condition of the hand rail before each use: make sure it is not damaged or worn.
- Correct fixing: the hand rail must be firmly secured to the bed structure before use. Use the 2 straps provided to secure it in place before use.
- Avoid entrapment risks: take care when folding or moving the hand rail to avoid finger entrapment. Handle the rail carefully and ensure your fingers are not in the moving area.
- Intended use: this product is designed to secure a bed and must only be used for this purpose. Do not use it for any other application.

- This product is not suitable for use by children.
- Regular inspection: inspect the hand rail regularly to ensure it functions correctly and shows no signs of failure.
- Avoid incorrect assembly: follow the assembly instructions carefully to prevent incorrect installation, which could compromise safety.
- Do not force: do not apply excessive pressure on the hand rail.
- Always ensure that the mattress is as close as possible to the hand rail. Any gap could allow the user to become trapped between the two.
- Always use the securing straps provided when fitting to a bed.
- Regularly check that the straps are tight and that the device is stable and firmly secured.
- Do not fit this product to profiling beds or sprung divan bases.
- To avoid any risk of head entrapment, position the hand rail at least 320 mm (12.5") away from both the headboard and the footboard. See diagram: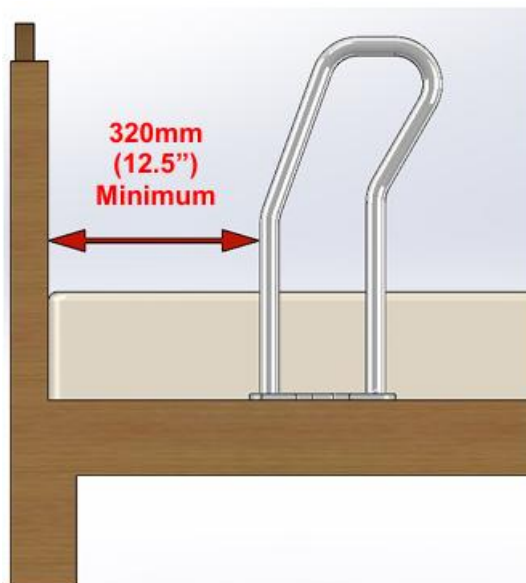

INSTALLATION

1. Fixed slatted bed

a. Remove the mattress and place the bed rail board inside the wooden bed frame. Make sure the hand rail guide is facing upwards (see diagram).

b. Insert the hand rail into the guide.



Note: the hand rail can be installed so that the angled part of the handle faces towards or away from the user, depending on their needs.

Secure it in place with the two E-clips (see diagram).

c. Position the hand rail so that the user can reach the handle from a relaxed lying position without having to stretch.

For the user's safety, it must be placed at least 320 mm from both the headboard and the footboard.

Then secure the hand rail to the bed base using the straps provided.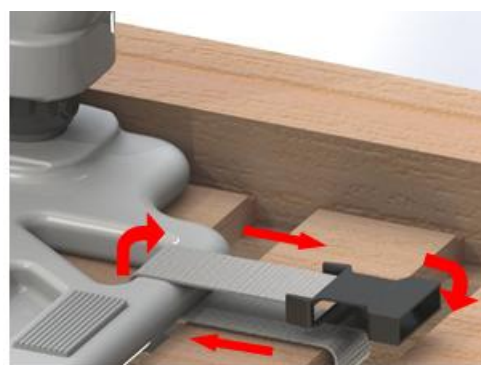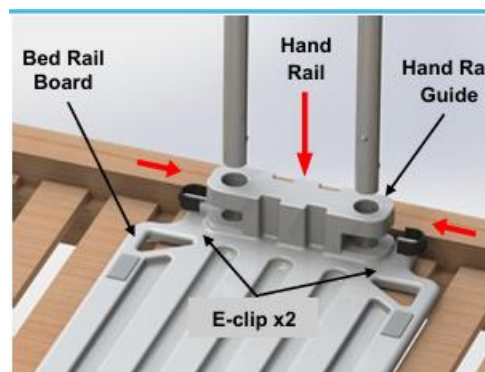

d. Pass one end of the strap (with the female buckle) under the bed, around the slat, through the triangular opening on side "A" of the bed rail board, and back under the slat (see diagram).


e. Repeat step d with the other end of the strap (with the male buckle), passing it through the opposite triangular opening on side "B" of the bed rail board (see diagram).

- f. Bring the two buckles together under the bed and clip them together (see below). Pull on the free end of the strap to tighten. Tuck away any excess strap to avoid a tripping hazard.
- g. Repeat the same procedure (step e) for the second strap, at the bottom of the bed rail board.
- h. Replace the mattress. Check that the hand rail is stable and securely fixed before use.


2. Loose slatted bed

- a. For beds where the slats are not individually fixed to the bed frame, do not use the securing straps.
- b. Assemble the hand rail as shown in diagram.
- c. For added safety, we recommend placing a non-slip mat between the hand rail and the slats.
- d. Replace the mattress. Check that the hand rail is stable and securely fixed before use.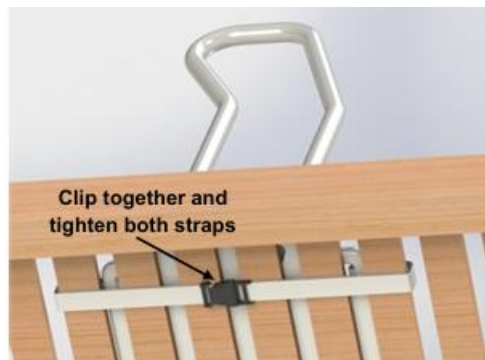

3. Divan bed

- a. Remove the mattress and place the bed rail board on the divan base at the desired position. Make sure the hand rail guide is facing downwards, over the edge of the divan (see diagram).
- b. Insert the hand rail into the guide.
Note: the hand rail can be rotated so that the angled part of the handle faces towards or away from the user, depending on their needs. Secure it in place with the two E-clips.
- c. Position the hand rail so that the user can reach the handle from a relaxed lying position without having to stretch.
For the user's safety, it must be placed at least 320 mm from both the headboard and the footboard.

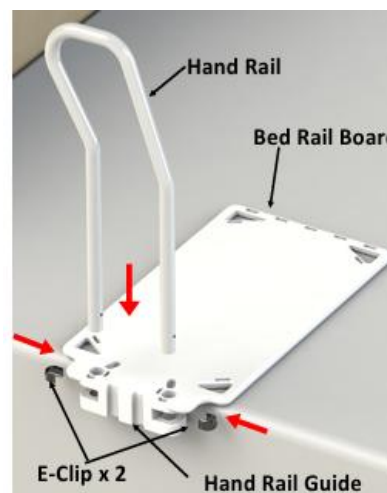
To secure the hand rail to the divan using the straps provided, proceed as follows :

- d. Take the first strap. Thread the female buckle through one of the triangular slots near the hand rail (A) and tie a secure knot. The buckle should hang down the side of the bed by approximately 50–75 mm (2–3 inches) (see diagram).
- e. Pass the strap under the bed rail board and out through the rear triangular slot (B). Take it around the end of the divan and back through slot (B) (see diagram). The strap should be pulled tight under the board, and the board must not slide on the strap.

- f. Pass the strap over the bed and back under the base again (see diagram). To secure the strap beneath the divan, take it to the end of the divan base, lift it slightly, and feed the strap underneath the feet/castors.
- g. Once the strap has been passed under the divan base, bring the male buckle up along the side of the bed to connect it to the hanging female buckle.
- h. Clip the two ends of the strap together.
- i. Pull the free end of the strap to tighten. The bed rail board must be pulled firmly against the divan base. Tuck the free strap ends under the bed to avoid any tripping hazard.
- j. Repeat the procedure with the second strap, fixing it on the other side of the bed rail board. The hand rail should now be fully secured to the divan base (see diagram).
- k. Replace the mattress. Check that the hand rail is stable and securely fixed before use.

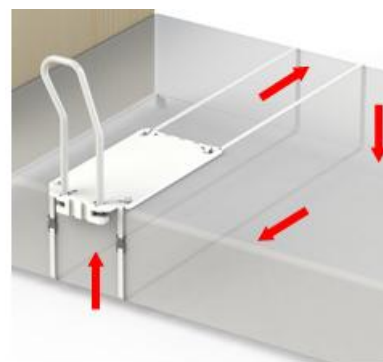If required, the hand rail may be removed from the bed rail board when changing the bedding.

Simply remove the two E-clips and lift out the hand rail.

To replace, re-fit the hand rail and secure it with the E-clips (see section 1.b or 3.b).

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regular Cleaning: Clean the product regularly with warm water and a soft cloth. Avoid abrasive products, detergents, or solvents.
- Drying: Dry the product with a clean, dry cloth.
- A diluted bleach solution may be used to disinfect the bed rail board and the hand rail, but not the straps
- Cleaning Products: Do not use cleaning products containing phenol or chlorine, as they may damage the product.
- Inspection: Periodically check the wear and stability of all screws. Immediately replace any broken, damaged, or worn parts.
- Repair: In case of malfunction, do not attempt to repair the product yourself. Consult an authorized dealer.
- Checklist :**
To keep the product in good condition, check before each use, and check regularly as follows:

Check the item	Anytime	Weekly	Monthly
Condition and functioning of the straps and E-clips	X		
Straps Condition and Functionality	X		
Cleanliness			X
Screws and Fastening Elements		X	
Frame Stability	X		

- Maintenance:**
- Regularly check for any signs of wear or damage.
- Remove the product from use immediately if there are signs of wear or damage, or if the hand rail is excessively loose in the bed rail board.
- Check that all securing straps are tight and that the hand rail is correctly positioned.
- Check that there is no gap between the hand rail and the mattress.
- Check that both E-clips are correctly fitted and securely in place.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product: in a dry and temperate place, protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Handle height	55.5 cm
Platform length	62 cm
Platform width	33.5 cm
Weight	3,8 kg
Maximum User Weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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