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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Caraïbes toilet chair is a medical device. It is a technical aid for toileting that is designed for people who are elderly, have reduced mobility, are disabled or injured, or who can easily lose their balance. It enables comfortable positioning on the toilet. The seat height is adjustable on the 811146.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum weight capacity of 130 kg.
- This product is not suitable for patients with cognitive impairment or who are unable to remain seated.
- Ensure that all components are secure before use.
- Ensure that the product is in good working order.
- Visually check the product
- Check that the chair is stable before use.
- Use only on level ground.
- Before each use, check that the 4 legs are at the same height and that the clips are fully extended.
- Never bend over when using the chair.
- Beware of the risk of pinching when using the chair.
- If any parts are damaged, broken, worn or missing, contact your retailer.
- Do not use in extreme temperatures (above approximately 38°C or 100°F or below 0°C or 32°F).
- Regularly check all the parts making up this product (screws, tubes, frame, etc.).
- Be careful not to sit down without first installing the seat.

- Check that all screw connections are tightened correctly. Do not sit down without checking that the seat is correctly in place.
- Do not use the chair for any purpose other than that for which it is intended: for example, do not stand on it.
- ALWAYS comply with the weight limits shown on the label or in the instructions.
- Do not modify this product. If necessary, use only original parts to repair it. Only qualified personnel should repair this product.
- Once you have unfolded or assembled the product, make sure that it is locked in the OPEN position and that it is stable on the floor before using it. Some models are delivered assembled.
- Do not hang anything from your commode. This could unbalance you and cause you to fall.
- If you are not going to be using it for a long time, have your appliance checked by your specialist retailer.
- Do not put on or remove the bucket when the patient is sitting in the chair: there is a serious risk of injury to the genital area!
- On models with lift-up armrests, take great care to avoid pinching fingers.
- If in doubt, do not use it and ask your specialist dealer for advice.
- Only use the chair on a level, stable floor.
- Do not sit on the chair's armrests.
- Do not bend over to avoid tipping over.

MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

INSTALLATION :

This product should only be commissioned by qualified personnel. If in doubt, ask your dealer.

Installing the potty :

Lift the seat towards the backrest and remove the potty from the top [B]. It is advisable to put the lid on before transporting the potty.

Installing the seat :

The seat can be easily removed for cleaning or replacement as it is only attached to the frame by two clips, photo [A].

To remove the seat, unclip the two clips, photo [B]. To replace it, fit the two clips onto the back tube of the chair, photo [C].

By fitting the plug back over the hole, the chair can be used as a normal chair.

USE :

- Do not allow children or unauthorised persons to play with or use the equipment without adequate supervision.
- Do not rock backwards and forwards or sideways when using the product, as this will cause excessive stress and may result in damage and failure which could cause injury.
- Do not use the product outside the house or on uneven surfaces (such as in the garden), as this will put stress on the feet and could damage them.
- Do not put your fingers or foreign objects into exposed tubular sections, as this could cause serious injury.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with a damp cloth and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Before cleaning the chair, remove the two armrests, the backrest and the four extension feet with the end caps. Clean these parts separately from the main frame.
- The chair must be completely serviced every year, with compulsory lubrication.
- It is recommended that the product be serviced at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it be serviced more frequently.
- Before each use, check the condition of the rubber feet and seat.
- Be careful, as urine is very corrosive and attacks steel tubes very quickly. It is therefore important to clean and dry the chair daily.
- In the event of a malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).
- Do store outdoors, do not leave in the sun (Be careful not to leave inside a car in direct sunlight).
- The chair is intended for indoor use only and must not be stored outdoors.

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	811125 - Width 60.5 x depth 44 x height 86 cm 811146 - W 60.5 x D 44 x H 86 to 97 cm/ Seat height adjustable from 46 to 62 cm
Weight	811125 : 6,6 kg 811146 : 6,8 kg
Maximum user weight	130 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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