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## INSTRUCTIONS MANUAL EN

### IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise on the most appropriate use for you.

Distributor: this leaflet must be given to the users of this product.

### PRODUCT DESTINATION

The Zero metal wheelchairs are walking aids for people with reduced mobility. These products are intended for indoor and outdoor use. The design allows for long-term use in the shower, in water, including swimming pools and spas. Without any magnetism, it can be used in an MRI room. It can also be used in airport security checkpoints and secure companies. It has all the features of a regular chair including removable armrests and footrests.

### PRODUCT DESCRIPTION

- 1 Rear wheel
- 2 Rear wheel axle
- 3 Brake (both sides)
- 4 Guide roller with cover
- 5 Track roller
- 6 Front swivel wheels
- 7 Handles
- 8 Backrest
- 9 Armrest
- 10 Seat
- 11 Frame crossbar
- 12 Removable footrest

### SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum supported weight of 100 kg.

Ensure that all parts are secure and in good working order.

Do not use the product on stairs or escalators.

Only use accessories and parts supplied by the manufacturer.

Check that the brakes work properly before use.

This wheelchair has been designed for indoor use in vehicles of any kind. Always transfer the user of the wheelchair into a seat approved by the vehicle manufacturer and use all available safety restraints.

Do not use this wheelchair on roads, streets or other surface traffic.

Be sure to fully activate the wheel brakes when getting in and out of the wheelchair, or when riding in a lift or wheelchair lift. The wheelchair should be on a flat, stable and level surface.

When getting in or out of the wheelchair, do not climb or put weight on the foot. When trying to extend the arms on the back, do so only until the arm can be extended without changing your sitting position.

Do not stand in the wheelchair

If you have to lean or lean on the wheelchair, make sure that the front wheels point forward, so that the base of the wheel is extended, to stabilise the wheelchair.

Seek professional advice when going up or down slopes greater than 10 cm. If you have to stop on a slope, avoid sudden and rapid movements when moving. Movement can cause instability due to the angle of the chair. Use of the brakes can lead to accidental braking, uncontrolled movement or sudden stalling. Try not to change direction on a ramp, as this may cause instability and the chair may tip over.

During use, lock the brakes for all actions.

All wheels must always be in contact with the ground.

When the product is in a stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.

Ensure that the footrests are engaged.

Ensure that the dismantling axle is securely engaged in the rear wheels and the wheelchair frame.

Be careful when climbing kerbs or when using sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the product on wet, icy or snowy slopes.

Be aware of the risk of pinching when using the product.

If any parts are damaged, broken, worn or missing, contact your dealer.


## ASSEMBLY AND USE

It is important that these assembly instructions are read and understood before assembling the wheelchair. Before using the wheelchair, it is necessary to carry out the following steps.

Note: you will only need the 2 special spanners that are in the assembly kit.

Operate the brake from position 1 to position 3: unlock the 2 side brakes from the vertical position to the forward position.

The wheel brake is locked when the wheelchair is packed in the box. When you start assembly, it must be unlocked, otherwise the wheel will not be assembled correctly.

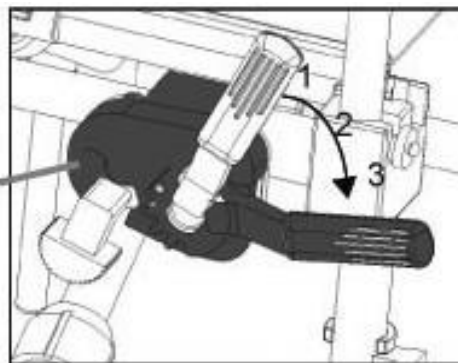### LIST OF PARTS

#### a) Parts of the chair

①

Rear wheel x 2

②

Left footrest x 1

③

Right footrest x 1

④

Front roller x 2

⑤

Main Frame x 1

Armrest x 2

⑥

Backrest x 1


Handle x 2

⑦

Push Bar x1

**b) Tools**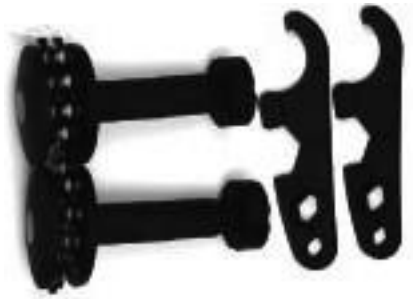

Rear wheel axle x 2

Spanner x 2

Front bearing x 2

**c) Repair kit**


Rear Bearing x 1

Brake handle x 1

Dust cover x

**Installation of the rear wheel parts. Install the axle bearing in position first.**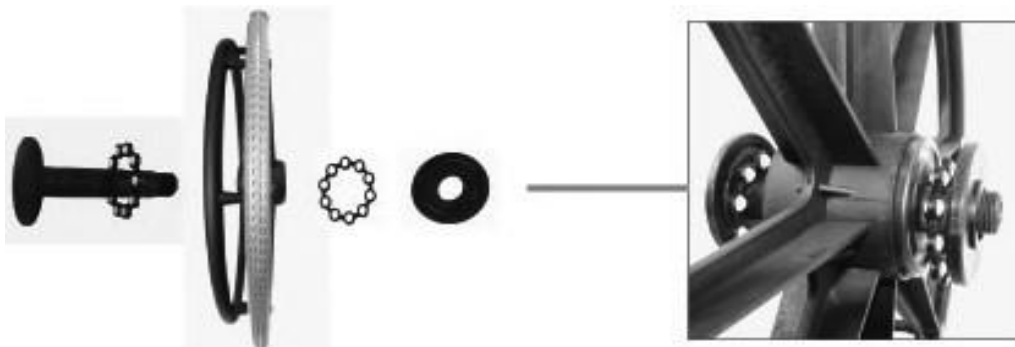

Put the rear wheel kit in the hole N°2 see below.  
There are 2 holes on the rear frame, make sure to put the rear wheel in hole 2.

- If you install the rear wheel kit in hole 1, the brake will not work and the rear wheel cannot be mounted properly.
- If the brake does not work, check this step.

Install the plastic wheel handle on the rear wheel and use the special spanner to tighten the axle.

Wheel handle

Tightening spanner

\* Rotate the wheelchair after installing the 4 wheels

Install the footrests. Check the orientation against the illustration. Release the lock to allow the footrests to be adjusted. Lock the lock when the footrests are in position.

\* There are 4 levels of adjustment for the footrests.

Install the backrest. Insert the handles. Please make sure that the backrest is correctly inserted into the frame and lock it. Then insert the push bar and lock it.

\* Raise the lock to assemble the backrest.  
Lower the lock to secure the backrest.

Insert the push bar  
\* The push bar has 2 fixing rings. Rotate them for left and right attachment

Check the attachment of all components for correct and safe assembly.


Check the attachment of the armrests, the attachment of the footrests, the attachment of the handles and the attachment of the push bar.

Check that the 2 side brakes can be used correctly.

If there is a seat belt and leg support, use them.


### **To fold and unfold the chair**

1. Raise the footrests to the upright position.
2. The push bar must be lowered or removed.
3. Grasp the seat cover with both hands in the middle and lift it up.
4. Place your hand on top of the seat bar. Press down on the bar until the wheelchair is fully open and ready to sit down.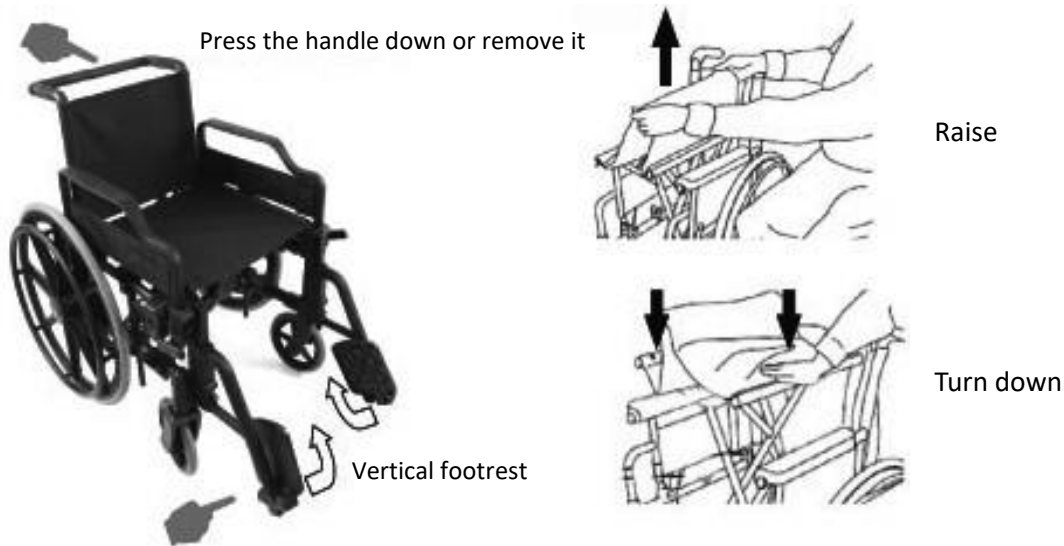

## Lock


1. Armrests, footrests, backrest handles, push bar all have the same lock.
2. Raise the lock, insert the armrests, footrests, backrest, push bar correctly. Make sure they are fully inserted.
3. Lower all locks to use the chair safely. Check before using the chair.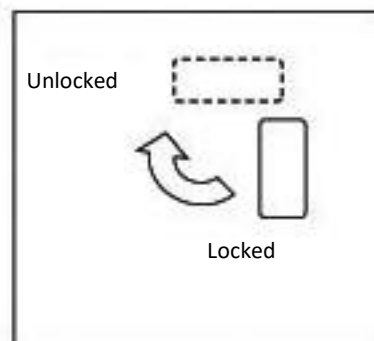

## Rear wheel brake

1. Place the chair on a flat surface to perform this procedure.
2. There are 3 brake positions: unlocked, slow motion, locked/stopped.
3. Position 1, unlocked (horizontal position), the rear wheel can turn freely. Position 2, slow position (handle position in intermediate).

Position 3, press to lock and park the wheelchair, the brake handle must be in the end position.

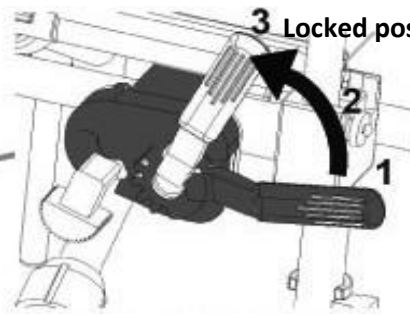**1. Position unlocked**  
**Press to block and stop**

\* Do not attempt to sit or stand

up from the chair until the brakes are securely locked.

## Footrest adjustment

1. The footrests are removable and adjustable. There are 4 footrest height positions, allowing different positions to be chosen for different uses.
2. Positions 1 to 4 determine the height of the footrests, from highest to lowest.
3. To select the footrest level, the user must open the lock and then lock it.

### Switching

Caution: Do not attempt to catch and/or lift distant objects. Do not lean forward, as the chair may tip over.

Make sure that the wheels or swivel castors are always in the direction of travel. Stand as close as possible to the objects to be picked up and apply the brakes.

### Back support

Caution: Do not try to catch objects behind you. The wheelchair could tip over.

Get as close as possible to the object with the wheelchair, without changing your sitting position. Move the front castors forward so that the 2 wheel axles are as far apart as possible. Tilt back as far as your arm length will allow without changing your sitting position.

Position and tilt at the rear

Both brakes must be locked to get on or off the wheelchair.

Unfold the main structure with your hands, pressing on the 2 side rails of the seat (do not try to open or separate the 2 armrests, they will not resist and will break).


Seat rails

Install the swivel castors at the front. Rotate the main frame to make it easier to fit the wheels. Fit the 2 front bearings, the fork through the tube and use the spanner to tighten the castors.

Front bearing kit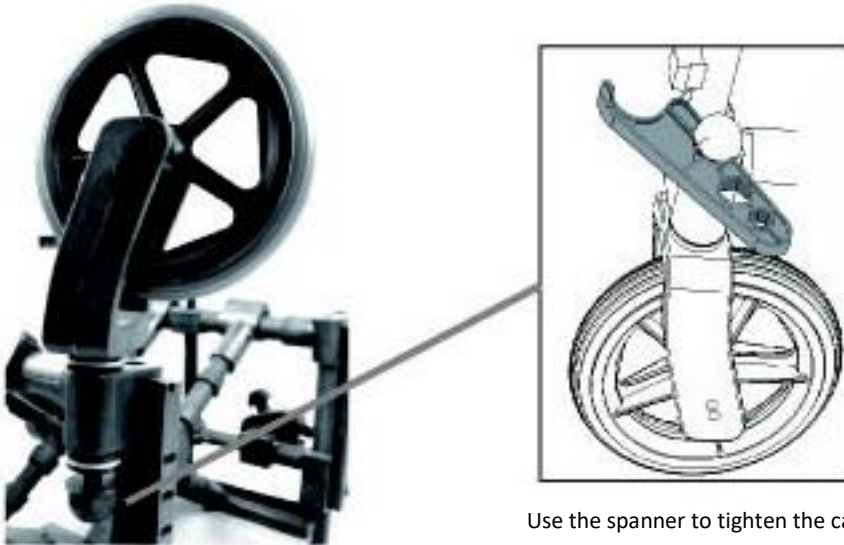

Use the spanner to tighten the caster screw

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Ensure that there is no build-up of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this may damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check all screws and bolts for wear and strength. Replace any broken, damaged or worn parts immediately.
- Check that the brakes work before each use.
- Check the parking brakes by pushing the handles into the locked position and then attempting to push the product.
- Check the padding for cuts, tears or heavy use. Replace if necessary.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Heavy duty rubber tyres require limited maintenance but do require regular cleaning with a soft cloth.
- Heavy duty rubber tyres should be replaced if they are cracked or torn.
- Checklist :  
To keep the product in good condition, check before each use, and check regularly as follows:

Check the product	Anytime	Every week	Every month
Screws and fasteners		X	
Stability of frame and backrest			X
Condition of the seat and wheels		X	
Functioning of brakes	X		

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 65 x height 87 x depth 109 cm
Seat dimensions	Width 45 x height 52 x depth 40 cm
Folded width: 36 cm	
Front wheel size	20 cm
Rear wheel size	59 cm
Weight	18 kg
Maximum supported weight	100 kg

MR SAFE Device

## **PERIOD OF USE**

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## **WARRANTY**

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

### **Exclusion of warranty**

- Footrest plates (wearing parts not guaranteed).
- Tyres (wearing parts not guaranteed).
- Parts damaged by an accident
- Damage due to improper use, maintenance and/or storage

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