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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case. Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Mini walker is a walking aid for people with reduced mobility. This rollator is designed for indoor use and folds up for easy storage or transport. It can be used outdoors on flat surfaces.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 Kg.
- Make sure that all parts are secure and in good working order.
- Do not use the walker on stairs or escalators.
- Only use accessories and parts supplied by the manufacturer.
- Do not use the walker to walk backwards.
- The walking frame is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may cause damage to the feet/wheels and the structure of the walking frame itself
- Do not make any adjustments to the walker while it is in use.
- All tips must always be in contact with the ground.
- Be sure to put your hands on each handle so that the walker can be in the parallel force position.
- Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on a wet, icy, or snowy slope.
- If any parts are damaged, broken or worn, contact the dealer.

USE

UNFOLDING :

1. Swing the side frames outward until a distinct click is heard.
2. Before using, test walker by grasping the side frames and, with some force, attempt to fold them inward. The walker will remain locked in the OPEN position if it is engaged properly.

FOLDING :

3. Pull up the lever and pivot the corresponding side frame inward toward the front cross brace.
4. Repeat the same procedure for the remaining side frame.

HEIGHT ADJUSTMENT :

5. Depress knob and slide leg up and down.
6. Tighten the knob into hole that provides correct height.
7. Repeat for each leg, making sure the walker is level.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the walker regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris under the tips.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly	Yearly
Check whether the product is locked in the unfolded state	X			
Checking the condition of feet and handles		X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width. 52 x Depth. 58 x Height 81.5 / 91.5 cm
Width between handles	38,5 cm

Weight	2,6 kg
Maximum supported weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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