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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a medical device.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Simplex bed table is a medical device. This product allows you to read and eat in bed in complete comfort. The height and tilt of the tabletop can be adjusted to suit different times of day.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

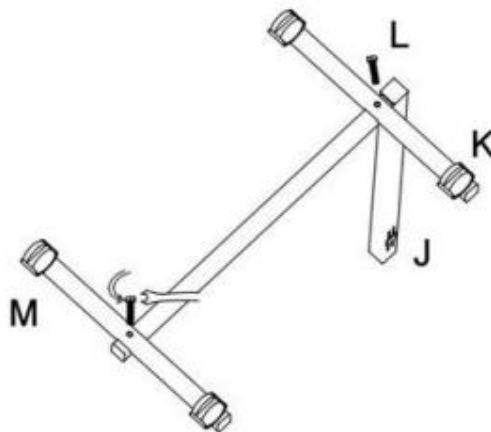
Failure to observe the precautions for use may result in the risk of serious injury.

- Use the bed table only as described in this manual.
- Do not exceed the maximum weight of 25 kg
- Install the bed table on a flat surface.
- Store the bed table indoors, away from mould and dust.
- Check that the adjustment knobs work properly before use.
- Before use, check that all parts are present and in good condition. In the event of damage, please contact your dealer.
- Carefully follow the assembly instructions and ensure that all components are correctly secured before use.
- Before use, apply the brakes to the castors.
- If any parts are damaged, broken or worn, please contact your dealer.

MOUNTING AND USE

ASSEMBLY AND USE :


Assemble crossbars K and M to frame J using bolts L. Observe the position of wheel lock crossbar K as shown in the diagram. Tighten bolts L. Ensure that all 4 wheels are in contact with the ground.

Insert the sliding tube C (square section) into the frame J and adjust it to the required height, then tighten the clamping knob F. Insert the clamping rod B into the tube C (cylindrical part) until the clamping nut makes contact with the tube. The two oblique parts are now in contact.

Place clamping washer E (small diameter in tube C) and loosely tighten clamping knob D.

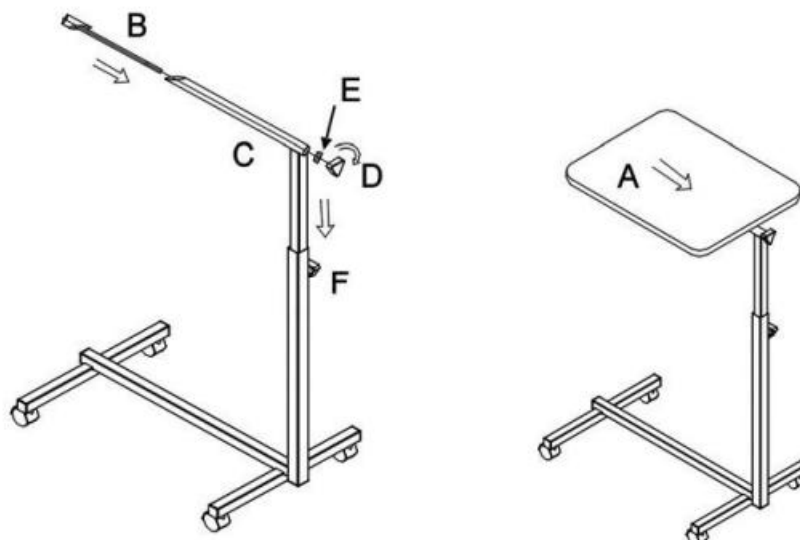Note: Clamping washer E must be fitted to lock plate A in the inclined position.

Slide the table top A into the sliding tube C and tighten the clamping knob D.

The height of the table can be adjusted using the clamping knob F.

The tilt of the table can be adjusted using clamping knob D.

When in use, make sure that clamping knobs F and D are correctly tightened.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water, a soft cloth and a neutral detergent.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Dry with a soft, clean, dry cloth.
- Make sure there is no accumulation of dirt on the product.
- Periodically check all screws for wear and strength. Replace all broken, damaged or worn parts immediately.
- Regularly check the operation of the brakes on the castors and the operation of the cylinder.
- In the event of a malfunction, do not repair the table yourself. Consult an authorised dealer.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 61 x depth 41 cm. Passage width 51 cm. Height adjustable from 70 to 118 cm. Tray 60 x 40 cm
Steel frame Paint: powder coating Table: Polystyrene high density board	
Wheels	Diameter: 1.50" Rotation: 150°
Weight (without accessories)	6,1 kg
Maximum weight supported	25 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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