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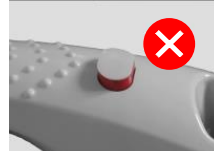
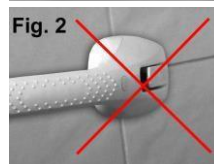
CAUTION

1. The suction cup grab bar must be used as a support aid and must under no circumstances support the total weight of the body. The maximum pull-out weight is 40kg.
2. As the suction cups may lose adhesion over time, it is recommended to reattach the support bar to ensure that it adheres perfectly to the surface.
3. The locking indicators are designed to ensure a successful installation. They are not intended to guarantee good adhesion to the surface in subsequent uses. Re-installing and testing the support bar regularly is the best guarantee of a perfect installation.
4. If you leave the grab bar mounted for a long period of time without reattaching it from time to time, the suction cups may lose their grip. In this case the support bar may be less adhesive when it is reassembled. It will then be necessary to leave it aside for 3 or 4 days to allow the suction cups to return to their original shape. You can then reuse the grab bar in its original shape conditions of maximum efficiency. In any event, check the assembly before each use.

IMPORTANT :

THIS ARTICLE CAN BE USED TO SUPPORT HEAVY LOADS. UMISUSE CAN IMPAIR ITS PERFORMANCE AND PERMANENTLY DEGRADE IT.

USE

INSTALLATION :

Place the grab bar at the desired position near the bathtub, in the shower, near the toilet... The suction cups must be on a flat, uniform, smooth and non-porous surface. Placing the suction cups on a crack or tile joint will prevent it from adhering properly and will make the grab bar unstable.

1. Clean the mounting surface cleanly. Make sure it is dry and dust-free. Check that the locking levers are released and place the grab bar, making sure that the suction cups are not placed on a crack or tile joint. Slightly moisten the suction cups for better adhesion.
2. Press the grab bar firmly to release air from the suction cups (Figure 1) and lock by firmly pushing the locking levers (Figure 3).

IMPORTANT: Make sure the LOCKING WARNINGS are fully depressed (Figure 4). If RED still appears, release the locking lever and lock it again. The LOCKING PADS (near the suction cups) must not be placed on waterdrops, or a surface different from that on which the suction cups rest. Re-test the adhesion of the grab bar a few minutes after installation.

PLEASE NOTE:

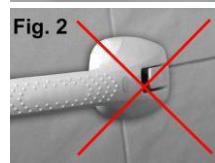
THE LOCK INDICATORS ARE ONLY AIDS TO INDICATE THE QUALITY OF THE INSTALLATION. YOU MUST ALWAYS TEST THE GRAB BAR MANUALLY BEFORE EACH USE.

DISMANTLING :

1. Release the locking levers.
2. Slowly slide the grab bar to the edge of its support surface. You can also slide your finger under the grab bar to slightly lift the suction cup.

DO NOT PULL THE GRAB BAR OUT, AS THIS COULD CAUSE PERMANENT DAMAGE.

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