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## USER MANUAL EN

You must carefully read this instruction manual before use and follow all instructions. Keep these instructions for future reference. You must check the functions of the device in accordance with this manual before each use and only use this device if it is in proper working condition.

### INTENDED USE

This device is an infrared thermometer intended to measure the temperature of the forehead of infants and adults without contact with the human body. It can be used by individuals in a home environment and used as a reference device in healthcare facilities.

### WARNINGS AND PRECAUTIONS

- Use this thermometer only after reading and understanding the instructions for use.
- Use this thermometer only for the specific purpose described in this manual. Any misuse will void the warranty.
- This thermometer can be used for professional or personal use at home.
- The use of this thermometer does not replace consultation with a physician. Do not perform self-diagnosis or self-medication based on the measurements without consulting your doctor.
- In particular, do not start taking new medication or change the type and/or dosage of existing treatment without medical advice.
- This thermometer must be used by adults and kept out of reach of infants.
- Children must not use this device without supervision; some parts are small enough to present a choking hazard if swallowed.
- If the thermometer is stored in a place colder or warmer than the place of use, allow it to adjust to ambient temperature for approximately 30 minutes before use.
- After physical activity, bathing, or staying outdoors, wait approximately 30 minutes before measuring forehead temperature.
- To ensure accurate measurements, use this thermometer only within an ambient temperature range of 10°C (50°F) to 40°C (104°F).
- Before each use, clean and disinfect the thermometer according to the manual to avoid bacterial infection.
- Do not expose this thermometer to electrical shock.
- Do not expose this thermometer to extreme temperatures (above 55°C / 131°F or below -20°C / -4°F).
- Do not use this thermometer in relative humidity above 85%.
- Do not place this product in direct sunlight, in areas of high heat, excessive humidity, dust, or near flammable gases or liquids.
- Do not use this product in the environments mentioned above.
- Do not drop the thermometer.
- DO NOT TOUCH THE INFRARED SENSOR LENS WITH YOUR FINGERS, as it is extremely fragile and may cause injury. Do not use this thermometer if it is damaged.
- Do not open this thermometer. It contains small parts that may be swallowed by children.
- Never immerse the device in water or other liquids, and do not allow liquids to enter the device.
- Remove the batteries if you do not intend to use the device for a long period, as there is a risk of battery leakage.
- Keep the thermometer away from electromagnetic fields generated by devices such as microwave ovens, welding equipment, or induction heaters.

- The device must not be used beyond its expected service life (5 years from the date of manufacture).
- Electrical waste and batteries have a significant impact on the environment; please comply with local laws and regulations for their disposal.
- Contact the manufacturer or your dealer if you have any questions.
- In case of malfunction or if the thermometer is damaged, DO NOT ATTEMPT TO REPAIR IT YOURSELF. Any repair by unauthorized personnel will void the warranty. Only authorized service centers may carry out repairs. The manufacturer can provide, upon request, diagrams, parts lists, descriptions, calibration instructions, and other information necessary for qualified personnel.
- If you need assistance with installation, use, or maintenance, or to report an unexpected event, please contact service personnel.

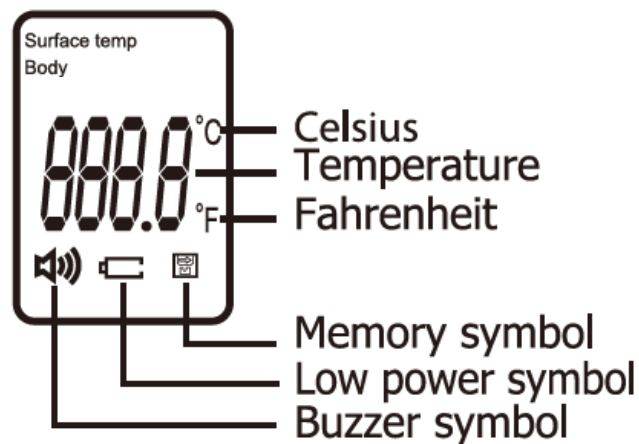
## OPERATING PRINCIPLE OF THE THERMOMETER

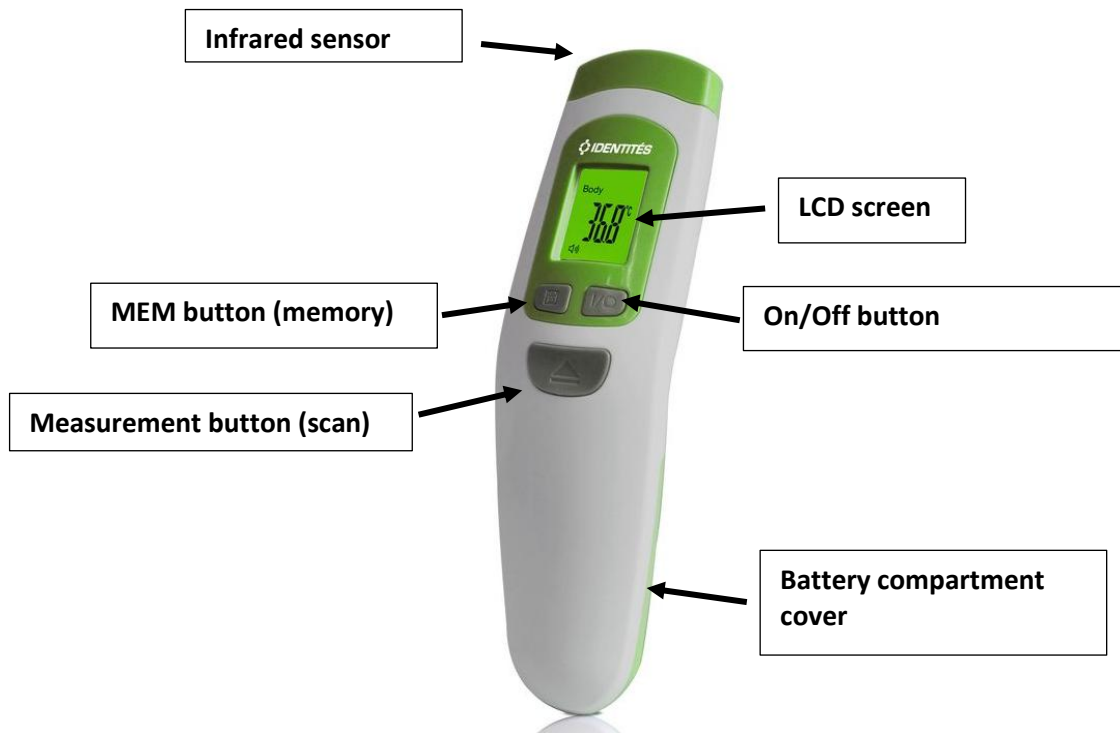
All objects—solid, liquid, or gaseous—emit energy in the form of infrared radiation, which depends on the temperature of the object. The infrared thermometer therefore measures a person's temperature based on the energy emitted by the body.

This measurement is made possible by an external temperature sensor integrated into the thermometer, which detects this energy. The operator holds the thermometer close to the body, and the measurement is obtained by detecting the arterial blood flow near the surface of the forehead. In this way, body temperature can be measured without interference from ambient heat.

*Note: The reference body temperature for non-contact infrared thermometers is the axillary temperature.*

## PRODUCT DESCRIPTION

## HOW TO USE YOUR NON-CONTACT INFRARED THERMOMETER

1. Install two batteries according to the manual provided in the “TECHNICAL SPECIFICATIONS” section.
2. When using the device for the first time after installing the batteries, wait 10 to 15 minutes for the device to stabilize and adapt to the ambient temperature.
3. Press the On/Off button «  », position the thermometer at a distance of 3 to 5 cm (1.2 to 2 inches) from the forehead (as shown in Fig. 1), then aim the infrared sensor at the forehead. Ensure that the measurement mode is set to body temperature “BODY”, press the measurement button “”, and the temperature will be displayed instantly.

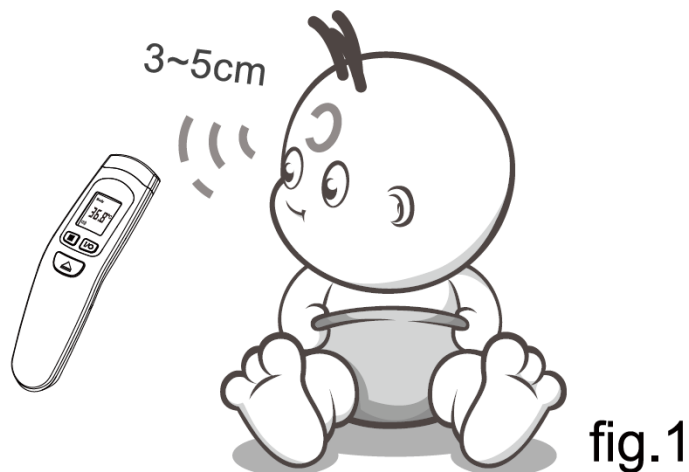
fig.1

4. Before taking a person's temperature, ensure that any hair, dirt, or dust is removed from the forehead and that it is dry. Wait 10 minutes after cleaning before performing the measurement.

For reliable results:

Do NOT attempt to measure temperature on any part of the body other than the forehead (e.g., arms, legs, torso, etc.).

Do NOT move the thermometer before the final result is displayed.

Do NOT take measurements near air drafts (e.g., wind, air conditioning, fans).

Do NOT take a person's temperature immediately after vigorous physical activity, bathing, taking medication, eating or drinking, or after applying a cold or warm compress to the forehead.

Do NOT take a person's temperature when there has been a significant change in ambient conditions. Allow the device to adapt to the new ambient temperature for at least 30 minutes before taking a measurement.

Wait at least 3 to 5 seconds between measurements.

We recommend taking three measurements on the forehead. If the results differ, use the highest value.

## TURNING ON AND SETTING THE THERMOMETER

### 1. Turning on the device

Press the «  » button. The display will illuminate, enter standby mode, and show “---°C” or “---°F”.

When you press the “” button, the measurement result will be displayed in 1 second.

If no action is taken, the device will automatically turn off within 30 seconds.

### 2. Setting the MODE

When the device is off, press the «  » button for two seconds : the screen will display “°C” or “°F”.

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Confirm by pressing the «  » button.

Then press the «  » button once to exit the settings.

Note: Body temperature is the default mode.

#### NOTE

SURFACE TEMPERATURE IS NOT INTENDED FOR MEDICAL USE.

Surface temperature is different from body temperature. To obtain body temperature, you must use the “Body temperature mode”.

Use the surface temperature mode to measure the temperature of a baby bottle or bath water.

### 3. Selecting the temperature unit

When the device is off, press the «  » button for two seconds: the screen will display “°C” or “°F”.

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button.

Then press the «  » button three times to exit the settings.

#### 4. Data Memory

When the device is turned on, press the «  » button: the last measured temperature will be displayed, and you can review the last 32 measurements.

#### 5. Memory Deletion

When the device is turned on, press and hold the «  » button for 5 seconds: all stored data will be deleted. Press the «  » button again: the screen will display “CLR”.

#### 6. High Temperature Alert Function

In body temperature mode, if the measured temperature is greater than or equal to 38°C, the device will emit a continuous beep.

#### 7. Buzzer ON/OFF

When the device is off, press the «  » button for two seconds: the screen will display “°C” or “°F”.

Press the «  » button once to access the buzzer setting (ON/OFF).

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Press the «  » button to select the language: French, English, Spanish, or turn off the voice function.

Confirm by pressing the «  » button.

Then press the «  » button twice to exit the settings.

### TECHNICAL SPECIFICATIONS

|                        |                                                                                                                                                                                                                                                                                                                                                                                                      |
|------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Power supply           | DC 3 V (2 AAA batteries)                                                                                                                                                                                                                                                                                                                                                                             |
| Temperature sensor     | Infrared sensor                                                                                                                                                                                                                                                                                                                                                                                      |
| Measuring distance     | 3 to 5 cm (1.2 to 2 inches)                                                                                                                                                                                                                                                                                                                                                                          |
| Measurement ranges     | Body temperature mode:<br>32.0°C to 43.0°C (89.6°F to 109.4°F)<br>In body mode, three backlight colors: <ul style="list-style-type: none"><li>• Green: ≤ 37.3°C (99.1°F) → normal temperature</li><li>• Orange: 37.4°C to 37.9°C (99.3°F to 100.2°F) → possible low fever</li><li>• Red: ≥ 38.0°C (100.4°F) → possible high fever</li></ul> Surface temperature mode:<br>0°C to 60°C (32°F to 140°F) |
| Measurement accuracy   | Body mode:<br>32.0°C to 34.9°C (89.6°F to 94.8°F): ±0.3°C (±0.6°F)<br>35.0°C to 42.0°C (95.0°F to 107.6°F): ±0.2°C (±0.4°F)<br>42.1°C to 43.0°C (107.8°F to 109.4°F): ±0.3°C (±0.6°F)                                                                                                                                                                                                                |
| Clinical repeatability | ≤ 0.2°C (0.4°F)                                                                                                                                                                                                                                                                                                                                                                                      |
| Display resolution     | 0.1°C (0.1°F)                                                                                                                                                                                                                                                                                                                                                                                        |
| Measurement time       | 1 second                                                                                                                                                                                                                                                                                                                                                                                             |

|                                       |                                                                                                                                |
|---------------------------------------|--------------------------------------------------------------------------------------------------------------------------------|
| Memory                                | 32 records                                                                                                                     |
| Operating conditions                  | Ambient temperature: 10°C to 40°C (50°F to 104°F)<br>Relative humidity: ≤ 85% RH<br>Atmospheric pressure: 700 hPa to 1060 hPa  |
| Storage and transportation conditions | Ambient temperature: -20°C to 55°C (-4°F to 131°F)<br>Relative humidity: ≤ 95% RH<br>Atmospheric pressure: 700 hPa to 1060 hPa |
| Weight                                | Approx. 75 g (excluding batteries)                                                                                             |
| Product dimensions                    | Approx. 170 × 48 × 40 mm (L × W × H)                                                                                           |
| Expected service life                 | 5 years                                                                                                                        |
| Automatic power-off                   | ≤ 30 seconds                                                                                                                   |
| Measurement site                      | Forehead                                                                                                                       |
| Reference body site                   | Axillary                                                                                                                       |
| Mode of operation                     | Adjusted mode                                                                                                                  |

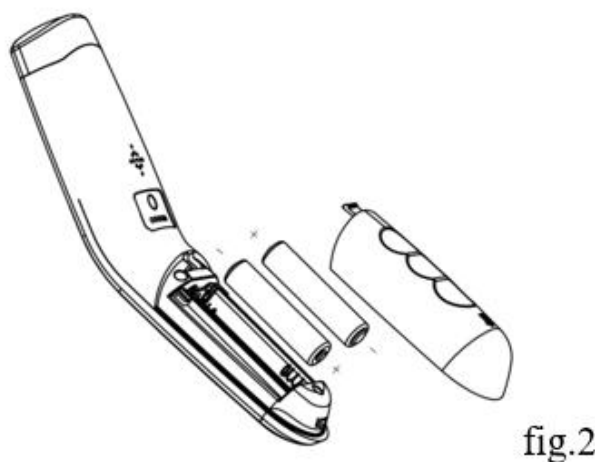
## BATTERIES

### 1. Battery Replacement

When the battery symbol «  » flashes on the display, the batteries need to be replaced.

To replace the batteries, open the cover on the back of the device (see Fig. 2) and ensure that the batteries are correctly installed. Incorrect installation may damage the device.

Do NOT use rechargeable batteries. Use only non-rechargeable (single use) batteries.

### 2. Battery Disposal

To protect the environment, dispose of used batteries at appropriate collection points in accordance with national or local regulations. The disposal method is described in the DISPOSAL section.

Battery Handling Instructions

In case of contact of battery fluid with skin or eyes, rinse the affected area with water and seek medical advice.

Choking hazard! Small children may swallow batteries. Keep batteries out of reach of children.

If a battery has leaked, wear protective gloves and clean the battery compartment with a dry cloth.

Protect batteries from excessive heat.

Risk of explosion! Do not throw batteries into fire.

Use only identical or equivalent battery types.

Always replace all batteries at the same time.

Do not use rechargeable batteries.

Do not disassemble, open, or crush batteries.

## **PACKAGE CONTENTS**

1 non-contact infrared thermometer, 1 instruction manual and 2 AAA alkaline batteries.

## **CLEANING AND DISINFECTION**

The thermometer is a reusable device intended for multiple patients. The cleaning and disinfection instructions have been validated. Please follow these instructions before each use.

### **Cleaning:**

1. Use a cotton swab moistened with warm water or a neutral detergent to thoroughly clean the thermometer casing and the measuring probe in order to remove stains or grease.
2. Visually inspect the thermometer after the cleaning process and ensure that no dirt or stains are visible. If the device is not considered visually clean at the end of the cleaning step, you must either repeat the previous cleaning steps or safely dispose of the device so that a visibly contaminated device is not reused.

### **Disinfection:**

1. Disinfect the thermometer casing and probe for 30 seconds using a cotton swab or a non-linting cloth soaked in 75% ethanol.
2. Repeat the above steps at least 3 times, then wipe the casing and probe with a clean cloth. Allow approximately 1 minute for drying after the disinfection process.

### **Note:**

- Ensure that no liquid enters the interior of the thermometer.
- Never use abrasive cleaning agents, thinners, or benzene, and never immerse the thermometer in water or any other cleaning liquid.
- Take care not to scratch the surface of the probe lens or the display screen.
- If visual inspection reveals unacceptable deterioration such as corrosion, discoloration, pitting, or cracks, the device must no longer be used and should be discarded.
- If the thermometer has been used on patients, visually inspect it after each use to ensure there are no stains or residues on the casing or probe, and follow the cleaning and disinfection instructions.

## CARE AND MAINTENANCE

- Clean the thermometer regularly, generally once a month.
- If the probe tip is damaged, please contact the maintenance center.
- If necessary, clean the device according to its level of contamination. If you do not use this product for a long period (more than 2 months), remove the batteries.
- Store the thermometer in a dry place, free from dust, contaminants, and direct sunlight.
- Do not use batteries other than those specified, do not use rechargeable batteries, and do not dispose of them in fire.
- Do not expose the thermometer to sunlight or water.
- Handle the thermometer with care. Dropping, impacts, or strong shocks may damage the product.

### Calibration

This thermometer is calibrated at the time of manufacture. If it is used according to the instructions, periodic recalibration is not required.

If you have any doubts about the accuracy of the measurements, please contact your dealer immediately.

Do not attempt to modify or disassemble the thermometer.

Calibration of this product requires appropriate calibration equipment; it cannot be performed by the user.

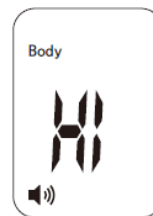
## TROUBLESHOOTING

If you encounter any problem while using your thermometer, please refer to this guide to help resolve it. If the problem cannot be resolved, please contact our customer service.

### THE SCREEN DISPLAYS THE MESSAGE « HI »

In body temperature mode, the thermometer will display “HI” if the measured temperature is higher than 43°C (109.4°F).


In surface temperature mode, the thermometer will display “HI” if the measured temperature is higher than 60°C (140°F).

### THE SCREEN DISPLAYS THE MESSAGE « LO »

In body temperature mode, the thermometer will display “Lo” if the measured temperature is lower than 32°C (89.6°F).

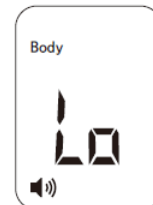In surface temperature mode, the thermometer will display “Lo” if the measured temperature is lower than 0°C (32°F).


This message may be displayed for various reasons. Please refer to the list of main causes below:

| Causes of the “LO” message display                                                | Solutions                                                                                 |
|-----------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------|
| Temperature measurement is affected by hair or perspiration                       | Ensure there is no obstruction or moisture before taking the temperature                  |
| Measurement is affected by airflow or a significant change in ambient temperature | Ensure there is no airflow in the measurement area that could affect the infrared reading |
| Measurements are taken too quickly, without allowing the thermometer to stabilize | Wait at least 3 to 5 seconds between measurements (15 seconds recommended)                |
| The measuring distance is too great                                               | Take measurements at the recommended distance (approximately 3 to 5 cm)                   |

## SIGNS AND SYMBOLS
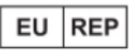
| Symbols                                                                             | Description of symbols                                                                                                                                                                                                                                                                                            |
|-------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | Indicates the need for the user to consult the instructions for use in order to obtain important safety information such as warnings and precautions that cannot, for various reasons, be presented directly on the medical device itself.                                                                        |
|    | Please read the instructions carefully.                                                                                                                                                                                                                                                                           |
|    | Applied part, type BF                                                                                                                                                                                                                                                                                             |
|   | Direct current                                                                                                                                                                                                                                                                                                    |
| <b>IP22</b>                                                                         | Degree of protection against ingress of water and solid particles: <ul style="list-style-type: none"> <li>• 2: Protected against solid foreign objects with a diameter <math>\geq 12.5</math> mm</li> <li>• 2: Protected against vertically falling water drops when the enclosure is tilted up to 15°</li> </ul> |
|  | DISPOSAL: Do not dispose of this product with unsorted municipal waste. Separate collection of this type of waste is required for proper treatment                                                                                                                                                                |
|  | Manufacturer                                                                                                                                                                                                                                                                                                      |
|  | Serial number                                                                                                                                                                                                                                                                                                     |
|  | Date of manufacture                                                                                                                                                                                                                                                                                               |
|  | Model number                                                                                                                                                                                                                                                                                                      |
|  | Made in China                                                                                                                                                                                                                                                                                                     |

|                                                                                   |                                                                                                                                                               |
|-----------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Medical Device                                                                                                                                                |
|  | Unique device identifier                                                                                                                                      |
|  | Symbol of the European Authorized Representative: indicates that the name and address of the European Authorized Representative are provided after the symbol |
|  | Symbol of the European Authorized Representative: indicates that the name and address of the European Authorized Representative are provided after the symbol |
|  | Importer                                                                                                                                                      |
|  | Consult electronic instructions for use.                                                                                                                      |
|  | Dispose of packaging in an environmentally responsible way.                                                                                                   |

## EMC DECLARATION

The ME EQUIPMENT or ME SYSTEM is suitable for use in home healthcare environments as well as clinical environments.

**Statement:** The Non-contact Infrared Thermometer model JXB-190 was tested according to the recommendations of Technical Report IEC TS 60601-4-2: Medical electrical equipment – Part 4-2: Guidance and interpretation – Electromagnetic immunity; performance of medical electrical equipment and medical electrical systems.

**Warning:** Do not use near active HF surgical equipment or in the RF-shielded room of an ME system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.

**Warning:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

**Warning:** Use of accessories, transducers, and cables other than those specified or provided by the manufacturer of this equipment may result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and could lead to improper operation.

**Warning:** Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the JXB-190 equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

1. All instructions necessary to maintain **BASIC SAFETY** and **ESSENTIAL PERFORMANCE** about electromagnetic disturbances throughout the expected service life.

Portable and mobile RF communications equipment may affect the performance of the JXB-190. Avoid strong electromagnetic interference during use, such as near mobile phones, microwave ovens, etc.

2. Guidance and manufacturer's declaration – electromagnetic emissions and immunity

Table 1

| Guidance and manufacturer's declaration – electromagnetic emissions |                |
|---------------------------------------------------------------------|----------------|
| Emissions test                                                      | Compliance     |
| RF emissions – CISPR 11                                             | Group 1        |
| RF emissions – CISPR 11                                             | Class B        |
| Harmonic emissions – IEC 61000-3-2                                  | Not applicable |
| Voltage fluctuations / flicker emissions – IEC 61000-3-3            | Not applicable |

Table 2

| Guidance and manufacturer's declaration – electromagnetic immunity                                      |                                                  |                                                  |
|---------------------------------------------------------------------------------------------------------|--------------------------------------------------|--------------------------------------------------|
| Immunity test                                                                                           | IEC 60601-1-2 test level                         | Compliance level                                 |
| Electrostatic discharge (ESD)<br>IEC 61000-4-2                                                          | ±8 kV contact<br>±2 kV, ±4 kV, ±8 kV, ±15 kV air | ±8 kV contact<br>±2 kV, ±4 kV, ±8 kV, ±15 kV air |
| Electrical fast transients / bursts<br>IEC 61000-4-4                                                    | Not applicable                                   | Not applicable                                   |
| Surges<br>IEC 61000-4-5                                                                                 | Not applicable                                   | Not applicable                                   |
| Voltage dips, short interruptions, and voltage variations on power supply input lines<br>IEC 61000-4-11 | Not applicable                                   | Not applicable                                   |
| Power frequency magnetic field<br>IEC 61000-4-8                                                         | 30 A/m<br>50Hz/60Hz                              | 30 A/m<br>50Hz/60Hz                              |
| Conducted RF disturbances<br>IEC 61000-4-6                                                              | Not applicable                                   | Not applicable                                   |

|                                                                                   |                                                 |                                                 |
|-----------------------------------------------------------------------------------|-------------------------------------------------|-------------------------------------------------|
| <b>Radiated RF disturbances</b><br>IEC 61000-4-3                                  | 10 V/m<br>80 MHz – 2, 7 GHz<br>80 % AM at 1 kHz | 10 V/m<br>80 MHz – 2, 7 GHz<br>80 % AM at 1 kHz |
| <b>NOTE:</b> UT is the a.c. mains voltage prior to application of the test level. |                                                 |                                                 |

Table 3

| Guidance and manufacturer's declaration – electromagnetic immunity                                                                     |                      |            |                                                                                |                                                               |                 |              |                           |
|----------------------------------------------------------------------------------------------------------------------------------------|----------------------|------------|--------------------------------------------------------------------------------|---------------------------------------------------------------|-----------------|--------------|---------------------------|
| Radiated RF disturbances<br>IEC 61000-4-3<br>(Test specifications for enclosure port immunity to RF wireless communications equipment) | Test frequency (MHz) | Band (MHz) | Service                                                                        | Modulation                                                    | Modulation type | Distance (m) | Immunity test level (V/m) |
|                                                                                                                                        | 385                  | 380-390    | TETRA 400                                                                      | Pulse modulation 18 Hz                                        | 1.8             | 0.3          | 27                        |
|                                                                                                                                        | 450                  | 430-470    | GMRS 460<br>FRS 460                                                            | FM (frequency modulation) ±5 kHz deviation<br>1 kHz sine wave | 2               | 0.3          | 28                        |
|                                                                                                                                        | 710                  | 704-787    | LTE Band 13, 17                                                                | Pulse modulation 217 Hz                                       | 0.2             | 0.3          | 9                         |
|                                                                                                                                        | 745                  |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 780                  |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 810                  | 800-960    | GSM800/900,<br>TETRA800,<br>CDMA850,<br>LTE Band 5                             | Pulse modulation 18 Hz                                        | 2               | 0.3          | 28                        |
|                                                                                                                                        | 870                  |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 930                  |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 1720                 | 1700-1990  | GSM 1800;<br>CDMA 1900;<br>GSM 1900;<br>DECT;<br>LTE Bands 1, 3, 4, 25<br>UMTS | Pulse modulation 217 Hz                                       | 2               | 0.3          | 28                        |
|                                                                                                                                        | 1845                 |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 1970                 |            |                                                                                |                                                               |                 |              |                           |
|                                                                                                                                        | 2450                 | 2400-2570  | Bluetooth<br>WLAN (Wi-Fi)<br>802.11 b/g/n<br>RFID 2450<br>LTE Band 7           | Pulse modulation 217 Hz                                       | 2               | 0.3          | 28                        |
|                                                                                                                                        | 5240                 | 5100-5800  | WLAN (Wi-Fi)<br>802.11 a/n                                                     | Pulse modulation 217 Hz                                       | 0.2             | 0.3          | 9                         |

**WARRANTY**

In accordance with our policy of continuous product improvement, we reserve the right to make technical and aesthetic changes without prior notice.

#### **Warranty and Repair Terms**

Warranty : 12 months for professional resellers. 24 months for private consumers.

Please contact your dealer in the event of a warranty claim. If you need to return the device, please include a copy of your proof of purchase and indicate the nature of the defect. Batteries are not covered by the warranty.

#### **The following warranty terms apply :**

1. In the event of a claim, the date of purchase must be proven by means of the invoice.
2. Defects in materials or workmanship will be corrected free of charge during the warranty period.
3. Repairs carried out under warranty do not extend the warranty period, either for the device or for the replaced parts.
4. The following are excluded from the warranty:
  - a. Any damage resulting from improper use, e.g., failure to follow the instructions for use.
  - b. Any damage resulting from repairs or modifications carried out by the customer or unauthorized third parties.
  - c. Any damage occurring during transport from the manufacturer to the consumer, or during transport to the service center.
  - d. Accessories subject to normal wear and tear.
5. Any liability for direct or indirect consequential losses caused by the device is excluded, even if the damage to the device is accepted as a warranty claim.

Under no circumstances shall IDENTITES 49 be liable for any loss of profit or any consequential, incidental, indirect, special, punitive, or contingent damages whatsoever in connection with claims made by any customer, consumer, or other user of IDENTITES 49 products.

In the event that any applicable laws impose warranties, conditions, or obligations that cannot be excluded or modified, this paragraph shall apply to the fullest extent permitted by such laws.

## **DISPOSAL**

It is prohibited to dispose of WEEE (Waste Electrical and Electronic Equipment) as unsorted municipal waste. Such waste must be collected separately.

In accordance with Directive 2012/19/EU and national regulations regarding the disposal of waste electrical and electronic equipment, these devices must be disposed of in a specific manner within the European Union.

These regulations require environmentally friendly recycling and disposal of waste electrical and electronic equipment at the end of their life. These products must not be disposed of with household waste.

This requirement is indicated by the symbol of the « crossed-out wheeled bin ».

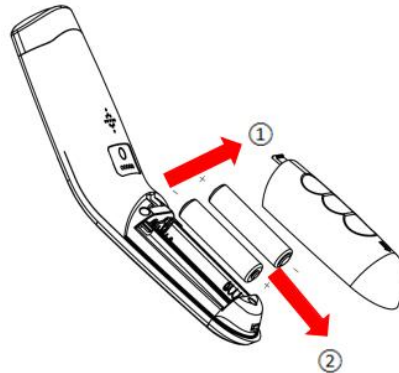
The non-contact thermometer (Model: JXB-190) contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it with unsorted household waste, but collect it separately and take it to an appropriate local collection point.

The non-contact infrared thermometer (Model: JXB-190) is classified as small equipment (Category 5) in accordance with Annex IV of Directive 2012/19/EU.

Before disposing of your non-contact infrared thermometer (Model: JXB-190), please remove the batteries from the device and dispose of the batteries and the thermometer separately.

You can remove the batteries as follows:

1. Open the cover located on the back of the device
2. Remove the batteries

Batteries pose a risk to health and the environment! Never open, damage, or swallow batteries, and do not allow them to pollute the environment. They may contain toxic heavy metals that are hazardous to the environment.

Removed batteries must be returned to the point of sale or disposed of in the appropriate collection containers provided at collection points by local public waste authorities, and handled in accordance with Directive 2006/66/EC.

Batteries and recyclable electronic waste must be disposed of in separate collection containers.

Packaging materials must be disposed of in accordance with local regulations. For more information, please consult authorized collection points.

#### In Germany :

For the disposal of recyclable electronic waste, you can search for authorized collection points at the following website:

<https://www.ear-system.de/ear-verzeichnis/sammel-und-ruecknahmestellen>

For batteries, you can search for authorized collection points at the following website: <https://www.ear-system.de/ear-verzeichnis/battghersteller#no-back>

Search method: use your postal code or the name of the city and federal state where you live to find nearby collection points.

#### Other countries :

For country-specific information on disposal, please contact your local dealer or importer.

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Guangzhou Berrcom Medical Device Co., Ltd.
Shanghai International Holding Corp. GmbH (Europe)