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**INSTRUCTIONS MANUAL**  
**EN**

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT DESTINATION

The Modulolight transfer chair are walking aids for people with reduced mobility. These products are intended for use indoors and can be folded up for easy storage or transport.

## PRODUCT DESCRIPTION

Coussin dossier	Back cushion
Accoudoir	Armrest
Barre transversale double	Double crossbar
Plaque de pied	Footplate
Poignée de main	Handgrip
Frein auxiliaire	Attendant brake
Dossier demi-pliant	Backrest half holding
Frein à main	Hand brake
Roue arrière	Rear Wheel

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 135 Kg.

Make sure that all parts are properly secured and in good working order.

Do not use the product on stairs or escalators.

Only use the accessories and parts supplied by the manufacturer.

Check the correct operation of the brakes before use.

The transfer chair may be used outdoors only for moving from point A to point B, on a flat surface, and for a limited duration.

Always transfer the wheelchair user to a seat approved by the vehicle manufacturer and use all available safety fasteners.

Do not use this wheelchair in roads, streets or other surface traffic.

Be sure to fully activate the wheel brakes when getting out of the wheelchair, or when you are in a lift or wheelchair lift. The wheelchair must be on a flat, stable and level surface.

When getting in or out of the wheelchair, do not climb or put any weight on your foot. When you try to extend your arms over your back, do so only until your arm can be extended without changing your sitting position.

Do not stand in the wheelchair

If you have to lean or lean on the chair, make sure that the front wheels are pointing forward, so that the base of the wheel extends to stabilise the chair.

Seek professional advice when going up or down slopes of more than 10 cm. If you have to stop on a slope, avoid abrupt and rapid movements when moving. Movement can cause instability due to the angle of the chair. Using the brakes may result in accidental braking, uncontrolled movement, or sudden locking. Try not to change direction on a ramp, this could cause instability and cause the chair to tip over.

During use, lock the brakes for all actions.

All wheels must always be in contact with the ground.

When the product is in a stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.

Make sure that the footrests are properly engaged.

Make sure the removal pin is securely engaged in the rear wheels and frame of the chair.

Be careful when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the product on a wet, icy or snowy slope.

Beware of the risk of pinching when folding and unfolding.

If any parts are damaged, broken, worn, or missing, contact the dealer.

## **MOUNTING & USE**

### **1. UNFOLD THE WHEELCHAIR**

Hold the handles with both hands to separate the frame slightly. Push both sides of the seat upholstery flatness slightly by hand, so that the wheelchair has been unfolded. (Figure 1)

Figure 1

### **2. FOLDING THE WHEELCHAIR**

Figure 2

First turn over both the footplates, lift up both sides of the seat upholstery lightly with hands, and then it can be folded. (Figure 2)

### **3. USE OF BRAKE**

The user holds the brake handles with both hands when the wheelchair stops, and then moves the brake handles forward to selflocking condition, to make the rear wheels stop completely to keep still. The process for braking is completed (Figure 3).

Figure 3

### **4 . USE OF ARMREST**

Figure 4

When detaching the armrest, first press the armrest fastener with one hand to separate it from the front bracket of seat tube, then you can flip it back. When assembling the armrest, make the armrest fastener aim at the position of the front bracket of the seat tube and press lightly to self-locking fastener. (Figure 4)

## 5. USE OF LEGREST

To mount the footrest, positioning holes to fit with the sides of the footplate. Fit the footrests and rotated inwards until the closing perfectly. To remove the footrests, squeeze the handle rotate footrest, outward and lift them up. (Figure 5)

Figure 5

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the product.
- If there is a malfunction, do not repair the product yourself. Consult an authorized dealer.
- Checklist :  
To keep the product in good condition, check before each use, and check regularly as follows:

Check the product	Anytime	Every week	Every month
Screws and fasteners		X	
Stability of the frame and backrest			X
State of the seat and wheels			X
Operation of the brakes and folding armrests	X		
Folding and unfolding operation		X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

DESCRIPTION	874001.R 874001.G	874006.R 874006.G	874007.R 874007.G	874009.R 874009.G	874012G
OVERALL DIMENSIONS	Width 63.5 x Depth 97 x Height 92 cm	Width 58.5 x Depth 97 x Height 92 cm	Width 61 x Depth 97 x Height 92 cm	Width 66 x Depth 97 x Height 92 cm	Width 56 x Depth 97 x Height 92 cm
SEAT DIMENSIONS	Width 47.5 x Depth 41 x Height 49 cm	Width 42.5 x Depth 41 x Height 49 cm	Width 45 x Depth 41 x Height 49 cm	Width 50 x Depth 41 x Height 49 cm	Width 40 x Depth 41 x Height 49 cm
FOLDED DIMENSIONS	Width 72 x Depth 27 x Height 65 cm	Width 72 x Depth 27 x Height 65 cm	Width 72 x Depth 27 x Height 65 cm	Width 72 x Depth 27 x Height 65 cm	Width 72 x Depth 27 x Height 65 cm
FRONT CASTER SIZE	20.32 cm	20.32 cm	20.32 cm	20.32 cm	20.32 cm
REAR CASTER SIZE	30.48 cm	30.48 cm	30.48 cm	30.48 cm	30.48 cm
FULL WEIGHT	14.6 kg	14.2 kg	14.4 kg	14.8 kg	14 kg
MAXIMUM WEIGHT AUTHORIZED	135 kg	135 kg	135 kg	135 kg	135 kg
Double parking brakes 12" wheels - Polyurethane tyres Nylon backrest with foam Aluminium frame - liquid coated					

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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