PROTECTION DE BARRIERE DE LIT EN MOUSSE ZIPPEE

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IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a physician before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product

PRODUCT DESTINATION

The bed rail guard is a Class I medical device that complements the use of the bed rail. This product is attached to the bed rail with fasteners and protects the agitated patient from being knocked into the bed rail. It creates a reassuring cocoon for the patient. The choice to use a bed barrier protection is made by the multidisciplinary care team. It is intended for patients with senility pathology to complement their safety. Patient monitoring is important to ensure patient acceptance.

SAFETY PRECAUTIONS

Failure to observe the precautions for use may result in serious injury.

Check the condition of the seams and fasteners each time you use the bedrail. Do not use a damaged bedrail.

Do not use if straps are torn or buckles are broken.

Check that the guard is accepted by the patient and that he will not remove it. If the patient tries to remove it, do not use it and tell the referring doctor. Choose the right length for the bed rail and the type of bed.

Available in 2 lengths: 130 cm and 180 cm.

If you need to remove the protection for washing, do not forget to put it back with the foam.

Patient monitoring is important to ensure patient acceptance.

USE

The bed rail protector can only be fitted by carers. The bed rail protector is placed around the bed rail and attached with a ZIP. The foam side inside the bed. Each time it is used, check the condition of the seams and the ZIP. It should be checked that it fits well, that it is not too tight and that it fits the bed rail well. If the cover does not fit the height of the bed rail, do not use it, as it may be damaged.

• **Checklist :**

In order to maintain the product in good condition, check regularly as follows:

Type of control	Method	Frequency
Presence of tears, untied product	Visual checking	Each use
Checks on fastener (ZIP)	Visual checking	Each use

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the bed barrier regularly with a non-chlorinated bactericide. (Spray, wipe)

- Do not use abrasives, detergents or chlorinated products.

- Dry with a clean, dry soft cloth.

- In case of wear or tear, do not attempt to repair the protector: discard it or contact an authorised dealer to see if it can be taken care of by IDENTITES after-sales service.

- The cover can be washed at 95°C and dried at 60°C with standard non-chlorinated detergents

CHANGE OF PATIENT OR BED

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact the service in charge of waste disposal for this type of product. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

The product can be stored between 0 and 50 °C. Keep away from flames and sparks.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

TECHNICAL SPECIFICATIONS

Assembly : by stitching polyester - cotton thread

Filling: polyether foam 30 kg / M³ (± 10 %)

Cover: 41 % Polyester - 59 % Polyurethane - (This article contains biocides according to EU 528/2012 / protection against microbiological deterioration. Active ingredient : zinc-pyrithione)

Accessories: polyester zip and slider

Overall dimensions	823086 – 140 x 35 x 2 cm 823099 – 180 x 35 x 2 cm
Weight	823086 – 0,600 kg 823099 – 0,800 kg

DURATION OF USE

The expected lifetime of this product is 2 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 2 years from the date of purchase by the user. IDENTITÉS will provide replacement parts for any part that has broken due to a manufacturing, design or material defects. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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