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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Air transfer chair is a medical device. It is a mobility aid for people with reduced mobility. This transfer chair is designed for daily use both indoors and outdoors. It can be used on different types of surfaces such as concrete, asphalt, indoor surfaces or carpets. Please note that the Air transfer chair cannot be used on uneven or soft ground such as sand as this may damage the wheels, frame or other parts of the product.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 100 kg.
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Check the stability of the transfer chair before use.
- Do not use the product on stairs or escalators.
- Check that the brakes work properly before use.
- Take care when climbing kerbs or when using inclined, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the product on wet, icy or snowy slopes.
- Do not stand on the product's footrests as there is a risk of the product tipping over.
- Be sure to release the brakes before moving the wheelchair again.
- When transferring or when the wheelchair is stationary, apply the brakes.
- Never bend over while in the chair.
- Beware of the risk of pinching when folding.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

The Air transfer wheelchair does not require any installation, it is delivered fully assembled.

UNFOLDING :

- 1 - Wheelchair folded in the box
- 2 - Raise the backrest to the vertical position
- 3 - Lower the footrest to the horizontal position
- 4 - Spread the seat completely and press down with the palm of your hand.
- 5 - Lower the armrests to the horizontal position

FOLDING :

To fold the Air transfer wheelchair after use, follow the above steps in reverse order.

- 1 - Raise the armrests to the vertical position
- 2 - Fold in the seat
- 3 - Fold down the footrest
- 4 - Lower the backrest

BRAKING :

To brake, squeeze the left and right (black) brake handles at the same time.

To lock the brakes (parking brake), squeeze the black handles and then pull up the red triggers to lock them.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
 - Dry with a clean, dry soft cloth.
 - The walker must be completely reviewed every year, with compulsory lubrication.
 - It is recommended that the device is serviced annually.
 - If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
 - In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
 - **Checklist :**
To keep the product in good condition, check before each use, and check regularly as follows :

Check the product	Anytime	Every week	Every month
Screws and fasteners		X	
Stability of the frame and backrest			X
State of the seat and wheels			X
Brake and folding operation	X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 46 x Depth 77 x Height 90 cm
Seat dimensions	Width 36 x Depth 39 x Height 45,5 cm
Folded width	21 cm
Weight	7,14 kg
Maximum supported weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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