**COUSSIN RELEVÉ JAMBES POSITPRO
COUDIERE DE PROTECTION POSITPRO
TALONNIERE DE PROTECTION POSITPRO**

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DISPOSITIF MEDICAL

**POSITPRO LEG SUPPORT CUSHION
POSITPRO PROTECTIVE ELBOW PAD
POSITPRO PROTECTIVE HEEL PAD**

Instructions manual

MEDICAL DEVICE

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them in case of need.

User: If you are unable to read or understand the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause risks and damage. They will be able to advise and inform you on the most appropriate use.

Distributor: this leaflet must be given to users of this product.

PRODUCT DESTINATION

Positpro Leg Elevation Cushion:

POSITIONING aid for recumbent patients. This product is designed for patients at risk of heel pressure sores. It prevents bedsores in immobilised patients lying down. It acts by reducing pressure points at support points: heels, calves, malleoli, calves. It is available in a simple version and a version with a removable abduction pad to be placed at the crotch. The abduction block holds and wedges the legs in abduction. This device is intended for use by carers and is subject to a medical prescription.

Positpro Elbow Protector:

This positioning aid is designed for patients at risk of elbow pressure sores. It serves as a preventive measure against bedsores for immobilized patients. This device is intended for use by healthcare professionals and requires a medical prescription.

Positpro Heel Protector:

This positioning aid is intended for patients at risk of heel pressure sores. It serves as a preventive measure against bedsores for immobilized patients by offloading pressure from the heel and pressure points. This device is intended for use by healthcare professionals and requires a medical prescription.

PRECAUTIONS FOR USE

This product must be used on the advice of a healthcare professional following a medical prescription.

If the patient does not accept the product, do not use it. If the patient does not tolerate the product, do not use it.

If the patient experiences pain when using this product, do not use it.

Follow the instructions in the medical prescription and the planned treatment protocol.

Do not use a soiled or undecontaminated cushion.

Do not use the product if it is damaged - Check the seams before use.

Failure to observe the precautions for use may result in discomfort and injury to the hemiplegic patient.

USE

Positpro Leg Elevation Cushion:

The cushion is used for patients at risk of heel pressure sores. Be sure to use it while checking that the heel is relieved and that the calves are wedged in the grooves of the cushion foam. The abduction pad is placed in the crotch area and helps to position the legs and make the patient more comfortable. For optimum positioning, it is advisable to position the patient every 2 hours in a supine position at 30° (semi-fowler) and in a left and right lateral position at 30°. The POSITPRO range of cushions allows this positioning. Ask a healthcare professional for advice. Clinical monitoring involves ensuring that the patient is properly hydrated and fed.

Positpro Elbow Protector:

Place it on the patient's elbow and secure it with Velcro straps. It is recommended to reposition the patient every 2 hours in a 30° dorsal decubitus (semi-Fowler) and in left and right lateral positions at 30°. The POSITPRO cushion range aids in patient positioning. Review each cushion's characteristics and possible combinations. Consult a healthcare professional for guidance. The product is available in gray and aubergine colors.

Positpro Heel Protector:

Place it on the patient's foot and secure it with Velcro straps, ensuring heel offloading. It is recommended to reposition the patient every 2 hours in a 30° dorsal decubitus (semi-Fowler) and in left and right lateral positions at 30°. The POSITPRO cushion range aids in patient positioning. Review each cushion's characteristics and possible combinations. Consult a healthcare professional for guidance.

CHANGE OF PATIENT

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions. When changing patient, remember to give this leaflet to the new user.

CARE AND MAINTENANCE

Positpro Leg Elevation Cushion:

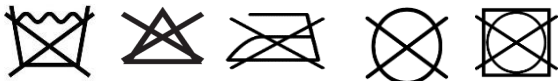
The STERITEX cover for the POSITPRO cushion can be decontaminated with a non-chlorinated disinfectant (spray, wipe). The cover can be removed and machine-washed at 95°C with the zip closed. Make sure you put the cover back on after washing. Disinfect the cushion after each use. Dry with a soft, clean, dry cloth. Do not use abrasive products, detergents or chlorinated products. In the event of wear or tear, do not attempt to repair the product. Dispose of it or contact your retailer to find out if it is covered by the after-sales service.

STERITEX Cover

Positpro Elbow Protector / Positpro Heel Protector:

The POSITPRO elbow and heel protectors can be decontaminated using a non-chlorine disinfectant (spray or wipes). Disinfect the product after each use. Dry it with a clean, soft, and dry cloth. Do not use abrasive products, detergents, or chlorine-based substances. If the product is worn or torn, do not attempt to repair it. Dispose of it properly or contact your retailer to inquire about warranty coverage.

RECYCLING

When the product has become unusable and you need to dispose of it, contact a local waste disposal company. If you wish to recycle the product, please enquire locally.

STORAGE

Failure to comply with storage conditions may lead to deterioration of the product and therefore a risk of injury. The product can be stored between 0°C and 50°C. Keep away from sources of ignition. Observe the following conditions when storing your products

- In a dry, temperate place
- Protect your product with dust-proof packaging
- Do not place heavy objects on the cushion: the foam could settle and deteriorate.

TECHNICAL CHARACTERISTICS

Positpro Leg Elevation Cushion:

Assembly: polyester-cotton thread stitching
Filling: Memory-effect visco-elastic polyurethane foam.
 The abduction pad in version 829102 (A) is padded with 100% polyester TECHNI GEL fibre with memory effect.
Cover in contact with the patient: Made from STERITEX technical fabric, polyurethane coating, soft and comfortable to the touch (190g/m2). STERITEX polyurethane is breathable and waterproof thanks to its 100% polyester bi-elastic knitted base. (190g/m2 - Composition: 55% polyester - 45% polyurethane)
 This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.
 Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione
 The covers have a ZIP with flap for better protection of the cushion.
 829034 : 59 x 53 x 10 cm – 1,500 kg
 829102 : 59 x 53 x 12 cm – 1,800 kg

Positpro Elbow Protector / Positpro Heel Protector:

Assembly: Polyester-cotton thread stitching

Filling: Viscoelastic foam flakes.

Coating: Made of polyurethane-coated fabric, flexible and comfortable to the touch (150g/m2).

Polyurethane is breathable and waterproof.

This product contains biocides per EU Standard 528/2012 for protection against microbiological deterioration.

Active ingredients: zinc – pyrethione + silver + +iodo-2propynyl butylcarbamate

829049: 28 x 16 cm - 0.180 kg

829050: 26 x 20 x 14 cm - 0.160 kg

Positpro protective heel cap (microbeads):

Assembly: polyester-cotton thread stitching

Filling: Polystyrene

Textile: 56% Polyester - 44% Polyurethane

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: zinc - pyrethione + silver + +iodo-2propynyl butylcarbamate

829143: One size - 0,100 kg

PERIOD OF USE

The expected life of this product is 2 years under normal use and maintenance. After this period, if the product has lost its memory properties, if the cover is damaged or if the Velcro fasteners are damaged: do not use this product. It can be used as long as it is still in good condition.

The lifespan of the product will vary according to the weight of the patients who use it and the number of times it is washed. The flattening of the product will be an indicator of its age.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user against defects in stitching and workmanship. Contact the distributor from whom you purchased the product should you need to make a claim. Failures resulting from accidental damage, misuse, improper storage, improper unpacking, modifications or wear and tear are not covered by this warranty.

Contact the retailer from whom you purchased the product to apply the guarantee.

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