 **ROLLATOR APPUIS ANTE - BRACHIAUX
NEO SUPPORT**

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESCRIPTION

The Neo Support Rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

CONTENTS

- 1 rollator
- 1 bag
- 1 Shoulder strap

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Do not exceed the maximum load limit in the basket (5kg). Only transport loads in the basket. Do not hang heavy objects on the rollator as this could affect its stability.

Make sure that all parts (wheels, cables and handles) are secure and in good working order.

Make sure that the wheels are tight and at the same height.

Do not use the rollator on stairs or escalators.

Only use accessories and parts supplied by the manufacturer.

The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.

Do not use the rollator to walk backwards.

Do not make any adjustments to the rollator while it is in use.

Check that the brakes work properly before use.

During use, lock the parking brakes for all actions.

All wheels must always be in contact with the ground.

When the rollator is in the stationary position, the brakes must be locked.

Do not attempt to move the rollator if the brakes are locked.

Be sure to put your hands on each handle so that the rollator can be in the parallel force position.

Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken or worn, contact the dealer.

MOUNTING

Unfold the rollator

Remove the protective coating from all four wheels.

Remove the cardboard between the rear wheels and the brake latches. Place the walker on a stable surface.

Unfold by spreading the frame apart while pushing down, making sure that each clip locks properly.

Adjust the height of the handles

Adjust the height of the handles by pressing the adjustment knob and sliding the tube up or down. The height will adjust one hole at a time. Make sure that both handles are adjusted to the same height.

Ideal handle height

A series of holes are placed every 2.5 along the handle tube. They allow the handle height to be adjusted to suit the user.

The ideal handle height is maintained when the top of the handle is aligned with the user's wrist.

Attach the satchel

Attach the side loops of the bag to the rollator handles.

Check braking

Check that the brake cables are running on the outside of the frame and adjust the cable guides to hold the cables neatly against the frame.

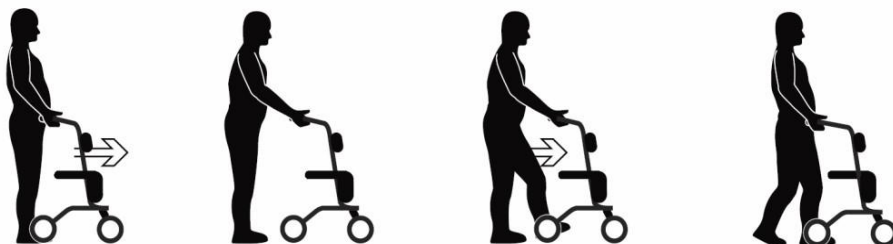
Make sure the brakes are working properly before using the rollator.

USE

Use of the walker

When using the rollator to move around, it is recommended to place both hands firmly on the handles, with fingers wrapped around the brake levers so that the walker can stop immediately in an emergency.

The ideal walking technique is to push the walker forward at a comfortable arm's length.

Use of brakes

To stop the rollator, gently squeeze the 2 brake levers until it comes to a complete stop.

Locking the brakes (parking brake function) When using the rollator as a booster seat, the brakes must be locked.

Freins en marche

To lock the brakes, press the brake levers until they click. Always verify that the brake lock is engaged before sitting in the seat.

To release the brake, pull the brake lever until the lock disengages. Never release the brake lock when sitting on the seat.

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Use as a booster seat

Before using the rollator as a booster seat, make sure it is parked on a safe, level surface. Before sitting down, lock both brake levers to prevent injury.

Transport and storage

Remove the bag

To remove the bag, detach the side buckles.

Fold the rollator To fold up the rollator, pull firmly on the strap on the seat. This will disengage the side braces and collapse the frame. Push the frame together for a compact fold.

New version Neo Support ante-Brachial support

After unfolding the rollator, fasten the belt before use. Make sure the belt is taut and tight. When you need to fold the rollator, you must loosen the belt.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized distributor.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 61.5 x Depth 70 x Height 108.5 to 121 cm (6 positions)
Overall width x depth	61.5 cm x 70 cm
Seat	46 cm x 25 cm x 54 cm
Colour	Grey metal
Width when folded	28 cm
Wheels	Diameter 20 cm
Framework	Aluminium
Maximum load	136 kg

Weight	8,4 kg
--------	--------

DURATION OF USE

The expected life of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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