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IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a physician before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

Technical positioning aid with memory effect for lying patients. These products are intended for patients at risk of pressure sores. They act to prevent pressure sores in patients who are lying down and immobilised. The cushions of the POSITPRO range allow to realize positioning combinations of the lying patient for the prevention of the pressure sore. They act by reducing the pressure points at the level of the head, elbows, sacrum, heels...but also the friction points of the knees, malleoli, calves and the areas at risk.

This device is intended for caregivers and is subject to a medical prescription. Read each property of each cushion before use.

SAFETY PRECAUTIONS

This product must be used upon medical advice and following a medical prescription.

Failure to follow the usage precautions may lead to injuries and deterioration to the pad properties.

Please follow the instructions of the medical prescription and care protocol.

Please follow the doctor's instructions on the type of pad to use, which will vary according to the patient's condition.

Do not use a soiled, contaminated pad.

Do not use a damaged product – Check seams before use.

If the patient does not tolerate the product, do not use it.

USE

It is advisable to position the patient every 2 hours in a 30° supine position (semi-fowler) and in a 30° left and right lateral position. Read the individual characteristics of the cushions and the possible combinations. Seek advice from a health professional.

Clinical monitoring includes proper hydration and dietary management of the patient. The product is available in grey and aubergine.

CARE AND MAINTENANCE

The STERITEX cover of the POSITPRO cushion can be decontaminated with a non-chlorinated disinfectant (spray, wipe).

The cover can be washed in a washing machine at 95°C with the zip closed.

Disinfect the cushion after each use. Dry with a clean and dry soft cloth. Do not use abrasive products, detergents, or chlorinated products. If the product is worn or torn, do not attempt to repair it. Dispose of it or contact your dealer to find out if it is covered by the After Sales Service.

STERITEX cover

Cushion core

CHANGE OF PATIENT OR BED

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing patients, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you need to destroy it, contact a local waste disposal company.

If you want to recycle this product: ask your local authorities.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.
The product can be stored between 0°C and 50°C.
Keep away from flames and sparks.
Respect the conditions for storing your products

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy objects on the cushion: the fiber could settle and degrade.

TECHNICAL SPECIFICATIONS

Assembly : polyester-cotton thread stitching

Filling : Comfortable TECHNI GEL fibre with an exceptionally soft and supple feel. 100% polyester with memory effect.

Core cover: Made of 100% cotton fabric, warp and weft.

Cover in contact with the patient: Made of a technical textile STERITEX, polyurethane coating, soft and comfortable to the touch (190g/m²)

STERITEX polyurethane is breathable and waterproof thanks to its bi-elastic knitted base made of 100% polyester.

(190g/m² - Composition: 55% polyester - 45% polyurethane)

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

Active ingredients: silverglass complex 398477-47-9 + zinc - pyrethione

The covers are equipped with a ZIP with flap for better protection of the cushion.

Overall Dimensions	829033 – 829033.A – 190 x 30 cm 829074 – 829074.A – 190 x 30 cm 829075 – 829075.A – 55 x 40 cm 829076 – 829076.A – 27 x 40 cm 829077 – 829077.A – 55 x 18 cm 829078 – 829078.A – 30 x 18 cm 829079 – 829079.A – 40 x 15 cm 829080 – 829080.A – 50 x 25 cm 829081 – 829081.A – 78 x 35 cm 829082 – 829082.A – 30 x 15 cm 829084 – 829084.A – 50 x 17 cm 829086 – 829086.A – 57 x 23 cm 829098 – 829098.A – 50 x 25 cm 829099 – 829099.A – 50 x 25 cm
Weight	829033 – 829033.A – 2,740 kg 829074 – 829074.A – 2,000 kg 829075 – 829075.A – 0,700 kg 829076 – 829076.A – 0,720 kg 829077 – 829077.A – 0,600 kg 829078 – 829078.A – 0,720 kg 829079 – 829079.A – 0,250 kg 829080 – 829080.A – 1,300 kg 829081 – 829081.A – 0,450 kg 829082 – 829082.A – 0,350 kg 829084 – 829084.A – 0,500 kg 829086 – 829086.A – 0,600 kg 829098 – 829098.A – 1,300 kg 829099 – 829099.A – 1,900 kg

DURATION OF USE

The life expectancy of this product is 2 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

La durée de vie prévue de ce produit est de 2 ans en condition normale d'utilisation et d'entretien. After this period, if the product has lost its shape memory properties, if the cover is damaged or the velcro : do not use this product anymore. It can be used as long as it is still in good condition. The life span will vary according to the weight of the patients who will have used this product and according to the number of washings of the product. The flattening of the product will be the indicator of the product's age.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user, on the defects of sewing and confection. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, incorrect storage, incorrect unpacking, modifications, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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