**TENSIOMETRE DE POIGNET***Notice d'utilisation - DISPOSITIF MEDICAL***WRIST BLOOD PRESSURE MONITOR***User manual - MEDICAL DEVICE***TENSIÓMETRO DE MUÑECA***Manual de instrucciones - DISPOSITIVO MEDICO***TENSIÓMETRO DE PULSO***Manual de instruções - DISPOSITIVO MÉDICO***MISURATORE DI PRESSIONE DA POLSO***Manuale d'uso - DISPOSITIVO MEDICO***HANGLINK-BLUTDRUCKMESSGERÄT***Gebrauchsanweisung - MEDIZINPRODUKT***POLSBLOEDDRUKMETER***Gebruiksaanwijzing - MEDISCH HULPMIDDEL***ΠΙΕΣΟΜΕΤΡΟ ΚΑΡΠΟΥ***Οδηγίες χρήσης - ΙΑΤΡΙΚΗ ΣΥΣΚΕΥΗ***NADGARSTKOWY CIŚNIENIOMIERZ***Instrukcja użytkowania - WYRÓB MEDYCZNY***REF. 861185**

SYMBOLES FIGURANT SUR L'ÉTIQUETTE OU DANS LA NOTICE D'UTILISATION
SYMBOLS APPEARING ON THE LABEL OR IN THE INSTRUCTION MANUAL
SIMBOLOS QUE APARECEN EN LA ETIQUETA Y EN EL MANUAL DE INSTRUCCIONES
SÍMBOLOS PRESENTES NO RÓTULO OU NO MANUAL DE INSTRUÇÕES
SIMBOLI PRESENTI SUL L'ETICHETTA O NEL MANUALE D'USO
SYMBOLE AUF DEM ETIKETT ODER IN DER GEBRAUCHSANWEISUNG
SYMBOLLEN OP HET ETIKET OF IN DE GEBRUIKSAANWIJZING
ΣΥΜΒΟΛΑ ΠΟΥ ΕΜΦΑΝΙΖΟΝΤΑΙ ΣΤΗΝ ΕΤΙΚΕΤΑ Ή ΣΤΙΣ ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
SYMBOLE ZNAJDUJĄCE SIĘ NA ETYKIECIE LUB W INSTRUKCJI UŻYTKOWANIA

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INTRODUCTION AND SAFETY INFORMATION

Thank you for purchasing our Wrist Blood Pressure Monitor.

Before using the device, please read carefully the instructions provided in this user manual. If you have any questions, please contact your distributor.

This device is intended to measure systolic and diastolic blood pressure, as well as pulse rate in adults.

WARNING

- Self-diagnosis and treatments based on measurement results may be dangerous. Follow your doctor's instructions.
- If the cuff does not stop inflating, remove the cuff or turn off the device immediately, as continued use may be hazardous.
- Only take measurements using the cuff provided by the manufacturer to avoid inaccurate results.
- Do not use the monitor near electromagnetic fields or high static electricity, and do not use your phone during a measurement.
- Measurements must be taken in a stable environment; otherwise, results may be inaccurate.
- Do not use the device while using oxygen equipment or in environments where flammable gases may be present.
- Do not use the device simultaneously with another monitoring device, as cuff inflation may temporarily interfere with the other device's operation.
- When replacing batteries, avoid touching both the device and the patient at the same time.
- This product is suitable for both personal use and medical use in a hospital setting.
- Blood pressure measured with this device is equivalent to that obtained by a professional using a cuff and stethoscope.
- Keep out of reach of children.
- Do not take repeated measurements too frequently, as this may cause bruising at the cuff site.
- Do not apply the cuff to a wound or inflamed area.
- Do not use the cuff on an arm where a mastectomy has been performed.
- Applying and inflating the cuff may temporarily alter blood flow but will not cause injury to the patient.

CONTRAINDICATIONS

- Do not use the blood pressure monitor for any purpose other than measuring human blood pressure.
- For individuals with arrhythmia, diabetes, circulatory disorders, or history of stroke, use should be supervised by a physician.
- Do not use on infants, children, or pregnant women.

MAINTENANCE

- Do not expose the main unit or the cuff to extreme temperatures, humidity, or direct sunlight.
- Do not disassemble the device.
- Do not subject the device to excessive shocks or vibrations (e.g., dropping it on the floor).
- Do not use volatile liquids to clean the main unit.
- Use a soft cloth dampened with soapy water to clean the cuff.
- Do not wash or immerse the cuff in water.
- Do not use gasoline, thinner, or other solvents to clean the cuff.
- Do not attempt to repair the device yourself. In case of malfunction, contact the manufacturer or distributor as indicated on the packaging.
- Clean the device with a dry, soft cloth.
- Store the device in its protective case when not in use.
- This blood pressure monitor has been rigorously tested to ensure accuracy and durability. It is generally recommended to have the device inspected every two years to maintain optimal performance and accuracy.


PRECAUTIONS FOR USE

BEFORE TAKING YOUR BLOOD PRESSURE:

- Do not eat, drink alcohol, smoke, take a shower, or exercise within 30 minutes prior to measuring your blood pressure. Do not take any medication that may raise blood pressure.
- Do not attempt to take your blood pressure if you are nervous, upset, or anxious, as this may cause elevated readings.
- Relax for about 5 to 10 minutes before the measurement. Sit comfortably, breathe freely and calmly, and remain still.

- The cuff should cover approximately $\frac{3}{4}$ of your wrist. It adjusts easily using the Velcro strap.
- If possible, always use the same wrist for each measurement.
- Taking your blood pressure at the same time each day helps ensure consistent results (unless affected by external factors such as physical activity).
- Changes in medication or diet may affect your results. Please consult your doctor before discontinuing any medication or dietary supplement.

PRODUCT DESCRIPTION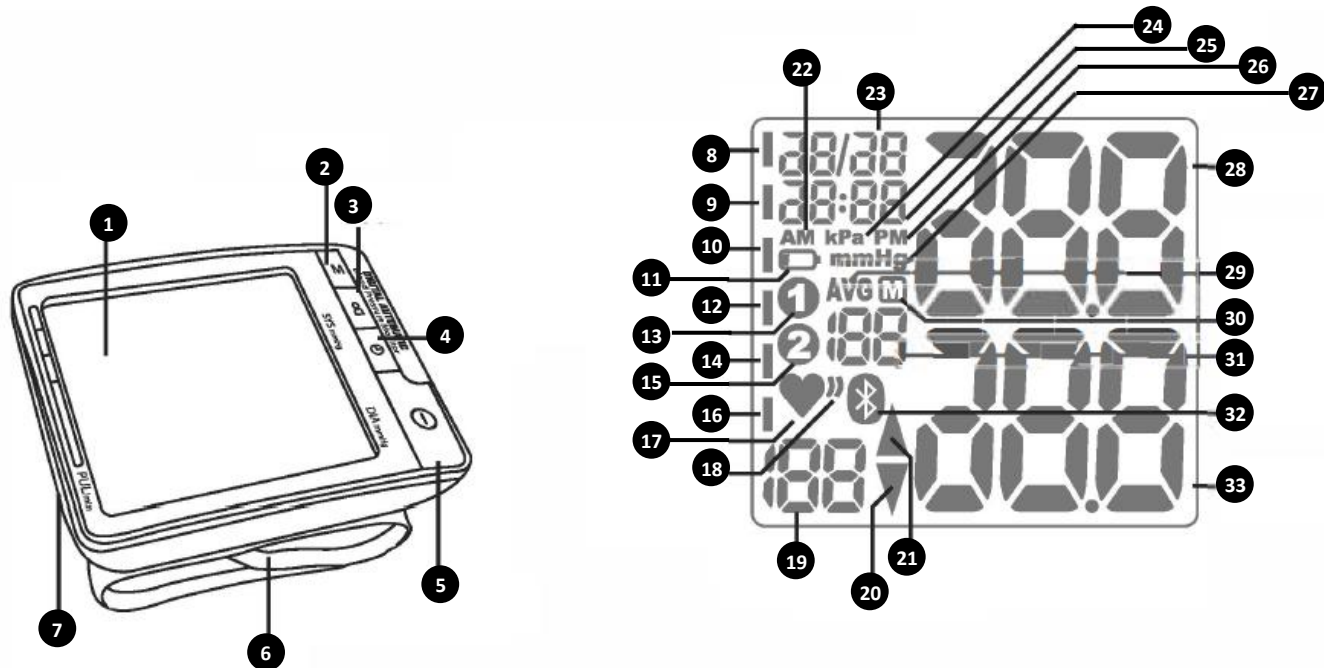

- | | | |
|--------------------------|-------------------------------|--------------------------------------|
| 1. Display | 13. User 1 | 25. Time |
| 2. Memory | 14. Normal | 26. Post Meridien (PM) |
| 3. User | 15. User 2 | 27. Pressure unit |
| 4. Clock | 16. Optimal | 28. Systolic pressure |
| 5. Power On / Off | 17. Heartbeat | 29. Average of previous measurements |
| 6. Cuff | 18. Irregular pulse indicator | 30. Memory |
| 7. Battery compartment | 19. Pulse | 31. Memory record number |
| 8. Severe hypertension | 20. Deflation | 32. Bluetooth |
| 9. Moderate hypertension | 21. Inflation | 33. Diastolic pressure |
| 10. Mild hypertension | 22. Ante Meridien (AM) | |
| 11. Battery indicator | 23. Date | |
| 12. High | 24. Pressure unit | |

USING THE PRODUCT


BATTERY INSTALLATION:

1. Open the battery compartment as shown.


2. Remove the old batteries and insert new ones. Use LR03 or AAA alkaline batteries.

3. Make sure to observe the correct polarity (+) and (-) inside the compartment.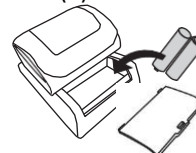

4. Close the battery compartment cover.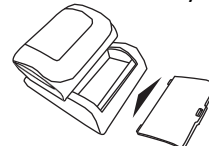


Note:

- Replace the batteries when the battery indicator shows a low level.
- The battery is low when the indicator displays "E6" and the battery icon appears []
- Do not use rechargeable batteries in the wrist blood pressure monitor.
- Battery life may vary depending on usage.

SETTING THE TIME:

(A) When installing the batteries

1. The year will start blinking on the screen.
2. Press the "M" button to advance the year.
3. Press the "Time" button to confirm. The month will then start blinking.
4. Press the "M" button to advance the month.
5. Press the "Time" button to confirm. The day will then start blinking.
6. Repeat the same process to set the hour and minutes.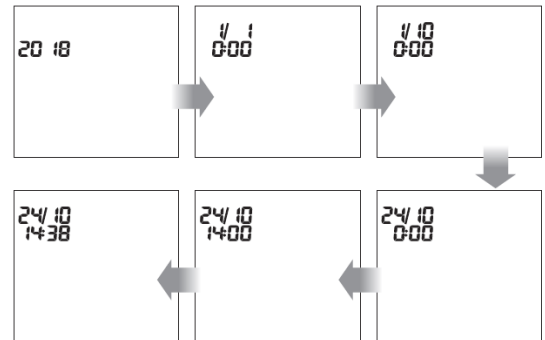


(B) At any time:



Press the "Time" button to display the current time, then press and hold the "Time" button for 3 seconds to access the date and time setting mode.

APPLYING THE CUFF:


1. Roll up your sleeve. Remove your watch, jewelry, and any tight clothing, as these may restrict blood flow.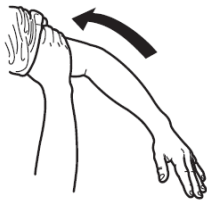

2. Wrap the cuff directly on your bare skin. Do not place it over clothing. With your palm facing upward, attach the monitor to your left wrist and secure it with the Velcro strap.
3. The bottom edge of the cuff should align with the crease of your wrist.
4. Hold the bottom part of the cuff and wrap it snugly around your wrist. Fasten the remaining part of the cuff securely.
5. If you are unable to use your left wrist, the measurement can also be taken on your right wrist.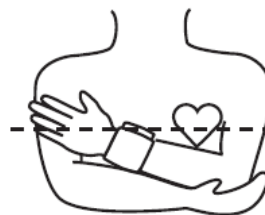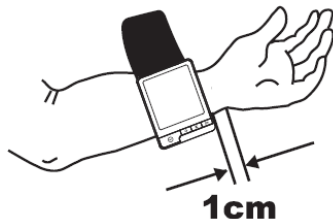

X TOO LOW

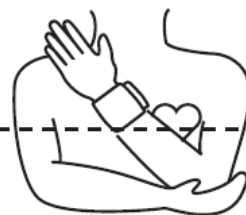X TOO HIGH


OK

TAKING A MEASUREMENT:


1. Place the cuff as described in the APPLYING THE CUFF section.
2. Press the "Power On/Off" button; the cuff will begin to inflate.
3. Once the cuff is fully inflated, it will automatically start deflating. As soon as the pulse is detected, the heart symbol will flash to indicate that the measurement is in progress.
4. When the measurement is complete, the systolic pressure, diastolic pressure, and pulse rate will be displayed on the screen.
5. Press the "Power On/Off" button to turn off the monitor.

Note: You can stop the inflation at any time by pressing the "Power On/Off" button again.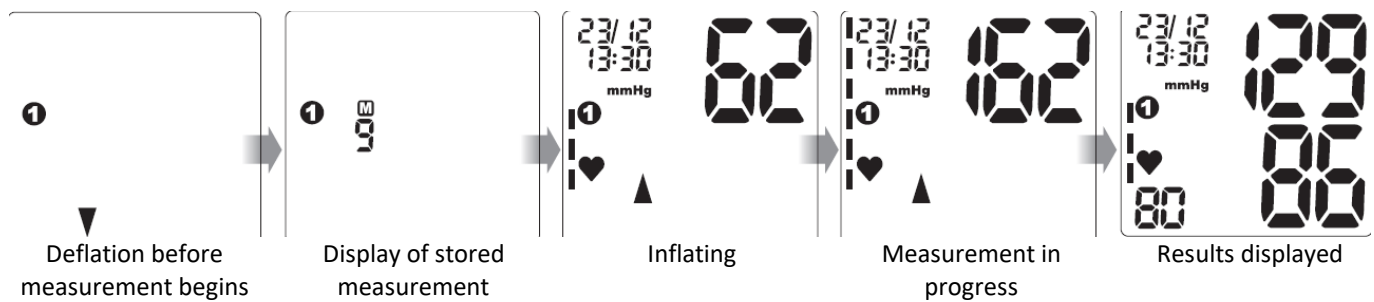


BLOOD PRESSURE LEVEL INDICATOR:

The standards for interpreting blood pressure levels, regardless of age, have been established by the WHO – World Health Organization – and are represented by the following chart: *Journal of Hypertension 1999, Vol. 17 No. 2.*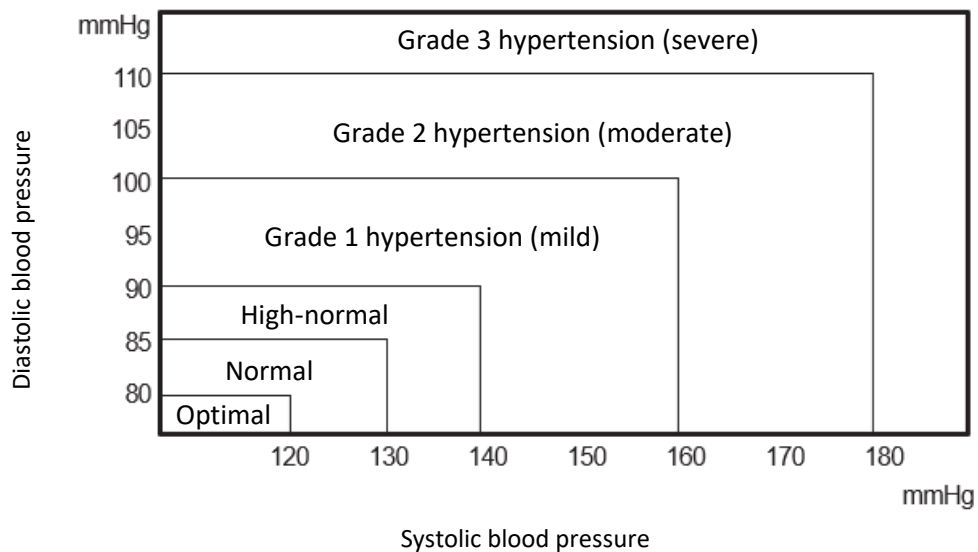

WHO CLASSIFICATION:

Each of the 6 segments of the indicator bar corresponds to the WHO (World Health Organization) blood pressure classification.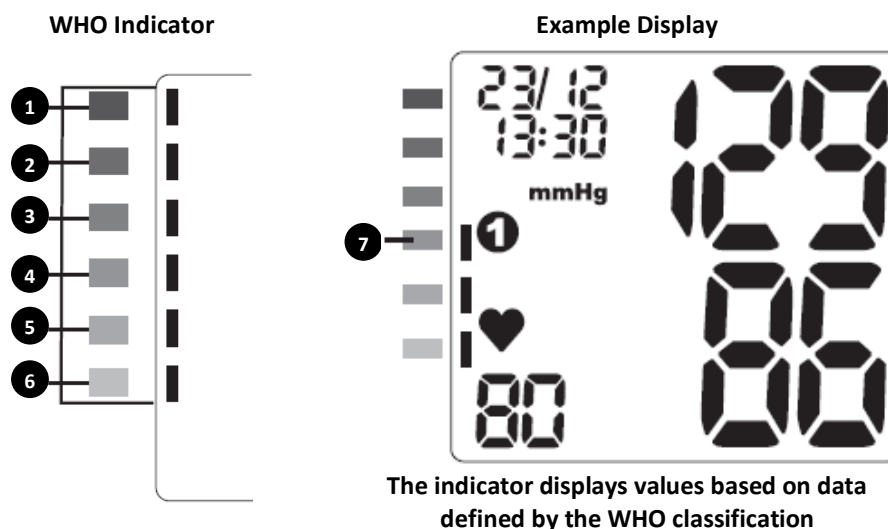

- | | |
|--------------------------|----------------------------|
| 1. Severe hypertension | 5. Normal blood pressure |
| 2. Moderate hypertension | 6. Optimal blood pressure |
| 3. Mild hypertension | 7. Elevated blood pressure |
| 4. High blood pressure | |

STORING / MEMORY / VIEWING PREVIOUS MEASUREMENTS:

The wrist blood pressure monitor can store measurements for each user. Each time a measurement is completed, it is automatically saved.


1. To view the average of the last 3 measurements, press the "Time" button.
2. Press the "M" button to display the data, then press "User" to select the desired user.
3. Press and hold the "M" button for a few seconds to display the most recent measurement.
4. Press the "Power On/Off" button to turn off the wrist blood pressure monitor.

DELETING RECORDS:

Two possible methods:


(A) Delete a single record

1. Navigate to the desired record.
2. Press the "M" button until the screen shows "CL" and "--".
3. Press the "M" button again to confirm. The screen will now show "CL" and "00".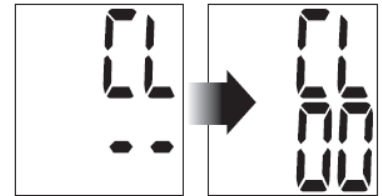

(B) Delete all records

1. Press the "M" button to enter the memory records.
2. Press and hold the "M" button until the screen displays "CL" and "--".
3. Press the "M" button again to confirm. The screen will now show "CL" and "00".

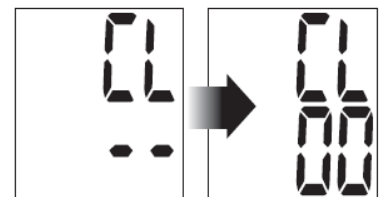BLOOD PRESSURE

What is Blood Pressure?

Blood pressure refers to the force exerted by the blood against the walls of the arteries. The contraction of the heart that pushes blood into the arteries is called systolic pressure. The relaxation of the heart that allows blood to return to it is called diastolic pressure.

What is High Blood Pressure (Hypertension)?

Hypertension (high blood pressure) is diagnosed when blood pressure readings are consistently above normal levels. It is widely recognized that untreated hypertension can lead to serious health risks such as stroke, heart attack, and other diseases.

Often referred to as the "silent killer" because it usually shows no symptoms to warn us of the problem, hypertension can be treated effectively if detected early.

To help prevent high blood pressure, it is recommended to:

- Avoid smoking
- Exercise regularly
- Reduce salt and fat intake
- See a doctor regularly
- Maintain a healthy weight

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Nothing appears, screen does not turn on	Batteries are depleted	Replace the batteries
	Batteries are incorrectly installed	Reinstall the batteries, ensuring correct polarity
ERROR 1 (code E1)	Cuff is improperly positioned	Reposition the cuff and ensure it is placed at heart level
ERROR 2 (code E2)	You moved during the measurement	Remain calm and still during the measurement
ERROR 3 (code E3)	Cuff is not properly secured or is too loose	Reattach the cuff correctly around the wrist
ERROR 4 (code E4)	Measurement could not be completed	If you are weak or have arrhythmia, the device may have difficulty measuring
	Measurement error	Sit calmly for a moment, then reposition the cuff correctly
ERROR 5 (code E5)	Cuff pressure is too high	Measurement exceeds 300 mmHg. Consult a doctor as soon as possible
ERROR 6 (code E6)	Low battery	Replace the batteries
Monitor keeps re-inflating	Circuit issue	Remove and reinsert the batteries, then restart the measurement

CARACTERISTIQUES TECHNIQUES

Display	LCD display
Measurement range	Systolic pressure: 50–250 mmHg Diastolic pressure: 30–200 mmHg Pulse: 40–180 beats/minute
Accuracy	Pressure: ± 3 mmHg Pulse: $\pm 5\%$
Resolution	Pressure: 1 mmHg Pulse: 1 beat/minute
Measurement method	Oscillometric method, non-invasive
Power supply	2 × 1.5V AAA batteries
Operating temperature & humidity	+5°C to +40°C, 15–90% humidity
Storage temperature & humidity	–25°C to +70°C, up to 90% humidity
Dimensions	102 × 78 × 30 mm
Wrist cuff circumference	13.5 to 21.5 cm
Accessories	Wrist cuff, User manual, Storage case

APPENDICES

Manufacturer Declarations on Electromagnetic Immunity			
This equipment is intended for use in the electromagnetic environment specified below. The customer or user of the equipment must ensure that it is used in such an environment.			
IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency magnetic field IEC 61000-4-8	50 Hz: 30 A/m 60 Hz: 30 A/m	50 Hz: 30 A/m 60 Hz: 30 A/m	Magnetic fields should be at typical levels found in the intended environment.
Conducted RF disturbances IEC 61000-4-6	3 V _{rms} ^a 0.15 MHz – 80 MHz	3 V _{rms} ^a 0.15 MHz – 80 MHz	Portable and mobile RF communication equipment should not be used close to this device. The recommended minimum distance is calculated according to the applicable frequency transmission equation. Recommended separation distance $d = \left[\frac{6}{E} \right] \sqrt{P}$
	6 V _{rms} ^a in ISM and amateur radio bands between 0.15 MHz and 80 MHz ^b 80% AM at 1 kHz	6 V _{rms} ^a in ISM and amateur radio bands between 0.15 MHz and 80 MHz ^b 80% AM at 1 kHz	
Radiated RF EM fields IEC 61000-4-3	10 V/m ^c 80 MHz – 2.7 GHz	10 V/m ^c 80 MHz – 2.7 GHz	Where P is the maximum output power in watts (W) of the transmitter, d is the recommended minimum separation distance in meters (m), and E is the immunity test field strength in V/m. The field strength from fixed RF transmitters should be less than the compliance level in each frequency range, according to an electromagnetic site survey.
	80% AM at 1 kHz	80% AM at 1 kHz	

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*Grandway Technology (Shenzhen) Limited
MedNet EU-REP GmbH