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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary. User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result. Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

CONTENTS

1 rollator
1 basket
2 adjustment knobs
1 backrest
1 front wheel set
2 rear wheels with handles and brake cable
1 user instructions

UNPACKING

Check all parts carefully before assembling the product. Do not assemble the rollator if any of the parts are damaged. Do not assemble the rollator if any of the items on the contents list are missing. When unpacking, ensure that the handles and brake

cables are not bent as this may affect the operation of the brakes when assembled.

PRODUCT USE

The Neo Fashion rollator is a walking aid for people with reduced mobility. This product is intended for use indoors and outdoors and can be folded up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Do not exceed the maximum load of 5 kg in the basket. Loads must only be transported in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Do not use the rollator on stairs or escalators.
- Use only accessories and parts supplied by the manufacturer.
- Make sure that the knobs are tightened to the same level and the handles are securely fastened.
- The rollator is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the rollator itself. The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Always sit on the seat when the rollator is stationary and the brakes are locked. Serious injury to the user and/or damage to the rollator frame and/or wheels can result from inappropriate use.
- Make sure all parts (wheels, cables, handles, brakes) are secure and in good working order.
- Do not make any adjustments to the rollator while it is in use. All wheels must always be in contact with the ground.
- Check that the brakes function properly before use.
- Make sure that the parking brakes are securely attached.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position. Make sure the seat is clipped flat before use.
- Make sure the seat tube caps are present.
- During use, lock the parking brakes for all actions.
- Do not attempt to push the rollator while someone is sitting in the seat.
- Do not use the seat to carry objects.
- Do not use the seat without the backrest.
- Do not sit in the seat when the rollator is on a slope.
- The brakes must be in the locked position before sitting in the seat.
- Do not attempt to reach objects while sitting down by leaning forward. Do not lean backwards as this can cause the rollator to tip over.
- Be careful when walking up pavements or on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the rollator on a wet, frozen, or snow-covered slopes. If any parts are damaged, broken or worn, contact the dealer.

ASSEMBLING

Before assembly consult a suitably qualified healthcare professional regarding the correct height adjustment for maximum support, and correct brake activation.

1. Check all wheels are in good working order and roll freely.
2. Check the basket is securely fixed and all fixing buttons are tightly fastened
3. To assemble; insert the front & rear wheels, using the button clips on the rear wheels and the "E" Clips on the front to position.
4. Fit the back rest into the frame and secure with snap clips.
5. To set the seat height, with the rollator frame open, push down on the seat until all four wheels are extended and the cross member is in line. The seat height can be adjusted by sliding the wheel assemblies in the frame tubes and positioning using the button & "C" clips. Make sure that all the clips "click" into position and all four wheels make contact with the floor and the rollator is stable.
6. Set the correct height of the handlebars, as follows:

- Stand upright behind the rollator with your shoulders relaxed and your arms by your side, looking straight ahead.
- In this position, the handgrips of the rollator should be adjusted to a height level with your wrists.
- This will position your arms at an approximate 20° to 30° bend when using the rollator.
- Check that the holes in the handle bars and the frame are aligned.
- Place the supplied Hand bolts through the aligned holes in the handlebar and frame, and hand tighten firmly - but do not over tighten.

7. Check the brakes thoroughly before use.

NOTE: The brakes are factory set, and if assembled correctly do not need adjusting prior to use. To check; Apply and release the parking brake on each side 10 times. (Push brake lever down to lock, lift to release) The brake should grip the wheel and prevent any movement when applied. When released, the gap between the brake moulding and the wheel should be between 3 to 4 mm. If either of the above conditions is not correct adjust the brakes as detailed in the maintenance section, and re-check.

8. Check all other nuts, bolts and screws are tightly fastened.

USE

Opening the rollator

Hold one handlebar and push down on the seat until all four wheels are extended out and the seat is fully flat. Before using the rollator make sure that all four wheels make contact with the floor and the rollator is stable in the open position

Folding the rollator

Lift the seat and pull up the seat support bar until the rollator closes. For more compact storage, pull up the backrest.

Walking

This unit is to be used as a mobility aid, creating a natural gait as the user walks securely with their primary weight distribution between the handlebars and with the unit in front of the user.

Sitting

When using the rollator as a seat, ensure the safety precautions detailed on page one are adhered to, in particular;

- Ensure the parking brakes are in the locked position.
- Do not use as a seat without the backrest being in position
- Do not use as a seat on sloping or uneven ground
- Do not exceed the maximum user weight
- Do not use the rollator as a wheelchair

CHECKING THE BRAKES

Before using the rollator always check the brakes are working correctly.

Check the normal brake function by pulling up the brake handles, the rollator should be held firm with no movement.

To apply the parking brakes, push down the handles until they lock down. A distinct click

indicates that the brake is in the correct position, the rollator should be held firm with no movement. (See Diagram 3b) It is important that both brakes are used for parking. Lift up the handles to release the parking brakes..

OPERATING THE BRAKES

To slow down, pull up both of the loop handles. (See Diagram 3a)

To apply the parking brakes, push down both handles until they lock down. A distinct click indicates that the brakes are in the correct position. (See Diagram 3b) It is important that both brakes are put in the parking position when they are used.

Lift up both handles to release the parking brakes.

If you apply the parking brakes, or pull up the brakes to stop whilst moving, and the rear wheels on the rollator still move, stop use immediately. Do not use again until the brakes have been checked and correctly adjusted. Refer to the maintenance section

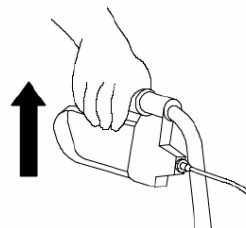
Schéma 3a

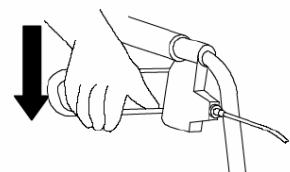
Schéma 3b


BRAKE ADJUSTEMENT

There are two brake adjusters on each side of the Rollator, one in front of the brake handle and the second is positioned on the

rear legs behind the wheels

To adjust the handle brake, loosen the lock nut, you may need a tool to carry out this operation, turn the adjuster anticlockwise to tighten the brake then retighten the lock nut. (See Diagram 4)

To adjust the rear legs brake, loosen the lock nut, you may need a tool to carry out this operation, turn the adjuster anticlockwise to tighten the brake then retighten the lock nut. For correct adjustment the brake mechanism should not touch the wheel, but the gap should be no greater than 3 to 4mm.

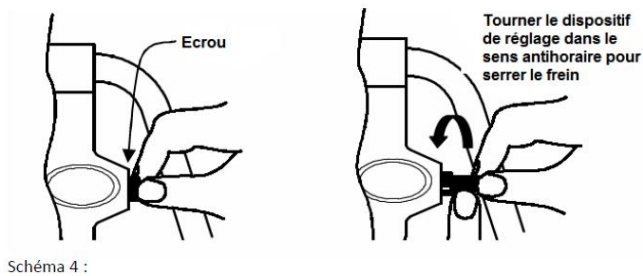Caution: After making any adjustments to the brakes, you must test them before the rollator is used. If adjustment does not tighten the brakes, do not use the rollator. Contact your dealer ;

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts.
- Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If the brakes do not completely block the wheels, the brakes must be adjusted.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGED OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your rollators :
 - In a dry and temperate place
 - Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that

TECHNICAL SPECIFICATIONS

Overall dimensions	Length.63 x width. 70 to 77 x height. 85 to 102 cm in 5 indexed positions (in relation to the seat 31 to 40 cm)
Passage width	45 cm
Seat dimensions	Length.37 x width.25 x height.53 à 61 cm
Weight	6.4 kg
Maximum basket load	5 kg
Maximum load	136 kg

PERIOD OF USE

The expected life span of this product is 3 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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