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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

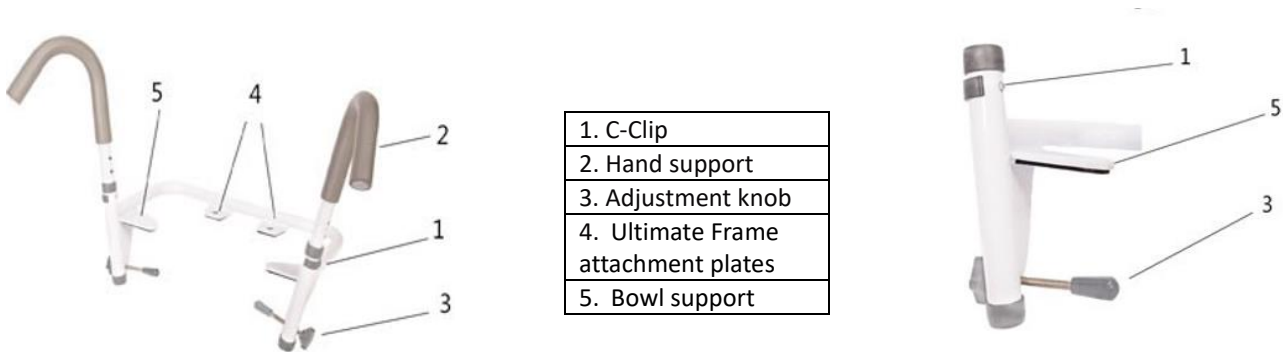
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION

The toilet armrests Malata hanging is a medical device. It is a technical toilet aid for the elderly, disabled, handicapped, injured or those who can easily lose their balance. It allows you to sit down and get up with complete peace of mind by resting on the stable and safe armrests. It can be installed under your existing eyewear or under your toilet seat. Adjustable in width thanks to two positions to adapt to all morphologies.

## PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 180 kg
- Make sure that all parts are supplied and in good working order.
- Make sure the product is properly installed and locked on the toilet before using it.
- If any parts are damaged, broken, worn or missing, contact the dealer.

## MOUNTING & USE

1. Remove existing toilet seat
2. Make sure tension knobs (3) are fully unscrewed.
3. Place Ultimate Toilet Safety attachment plate (4) over the existing holes on back of toilet.
4. If not using toilet seat riser, place toilet seat cover over attachment plate (4) and secure with new hardware.
5. Tighten tension knobs (3) equally on both sides of the toilet bow.
6. Remove C-clips (1) and place hand supports (2) (making sure they are installed on correct sides (left and right) to desired height and replace C-Clips (1) back.

**Note** : Inspect the C-clips (1) and tension knobs (3) periodically to ensure they have not loosened.

## CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Before each use, check that the spring clips are fully extended and the fixing plate tightened at the back of the toilet.
- Regularly check the condition of the armrests and the tightness of the fixings.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## FEATURES TECHNICAL

Overall Dimensions	Width 48 cm x Depth 24 cm x Height 26/38 cm
Armrests height	In relation to the bowl 26 to 38 cm in 6 positions.
Centre distance	15 cm
Weight	3 kg
Maximum supported weight	180 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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