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EN

### IMPORTANT

Read the instructions carefully before using this product and keep them for future reference.

User: If you are unable to read or understand the warnings, precautions, or instructions, please contact your healthcare professional or distributor before using this product to avoid risks and damages.

Consult your doctor or healthcare specialist before using an assistive device. They will advise and inform you on the most appropriate use for your situation.

Distributor: This manual must be provided to users of this product.

### PREPARATION

- Remove the insulating fabric cover and ensure that the rubber hot water bottle and the stopper are clean.
- Check for cracks or weak spots by gently stretching the rubber surface. Do not use if damaged.

### FILLING

- Fill with hot (not boiling) water:

Ideal temperature: 60–80°C

Never use boiling water directly.

- Fill vertically:

Keep the bottle upright.

Fill up to 2/3 of its capacity (leave air space).

Slowly tilt to 45° to release air bubbles.

Final fill level should not exceed 3/4.

- Screw the stopper tightly:

Wipe the rim dry before closing.

Ensure the stopper is securely tightened.

### APPLYING THE COVER

- Slide the filled bottle into the fabric cover:

Ensure full coverage.

Align the spout with the cover opening.

- Fasten all closures (zippers/buttons).
- The cover must remain dry – never submerge.

## USAGE

- Apply to the desired area for periods of up to 20 minutes.
- Never sit or lie directly on the hot water bottle.
- Keep away from infants and sleeping individuals.
- Remove the cover before reheating.

## EMPTYING AND STORAGE

- Empty completely after each use.
  - Air-dry inside out.
  - Store :
- In a cool, dark place, away from sharp objects, without twisting or kinking

## SAFETY WARNINGS

### ⚠ BURN HAZARD

- Test the temperature on the inner wrist before use.
- Never use without the fabric cover.
- Avoid direct skin contact for more than 5 minutes.

### ⚠ BURST PREVENTION

- Never fill beyond 3/4 of its capacity.
- Do not heat in a microwave or oven.
- Replace annually or sooner if the material becomes stiff or discolored

### ⚠ CHILD SAFETY

- Keep out of reach of unsupervised children.
- This is not a toy – contains hot liquid.

## FIRST AID

In case of leakage:

- Immediately remove from the skin.
- Cool burns under running water for at least 10 minutes.
- Seek medical attention for blisters or redness.

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