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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Nosy Be toilet frame is technical aid device. It is intended for people with reduced mobility who require assistance to access the toilet. The product can be used independently with the potty provided, or over an existing toilet. It has retractable padded armrests for easy lateral transfer (retractable with a simple push button).

PRODUCT DESCRIPTION

1. Foam armrest
2. Lid
3. Seat
4. Tip
5. Bucket
6. Foottube
7. Backrest

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Inspect all parts before assembly to ensure that they are not damaged during shipping. If there is shipping damage - DO NOT USE. Contact your dealer for further instructions.
- Check the stability of the chair before use.
- Use only on level ground.
- Never bend over while in the chair.
- If any parts are damaged, broken, worn or missing, contact your dealer.

ASSEMBLY AND USE

Setup Instructions

1. Remove frame, seat lid, pail and lid from carton. Allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat and lid clamps from cracking while being assembled onto commode back-frame rail.
2. Align the toilet seat clamps onto the back-frame rail. Push down on seat with both hands until clamps snap firmly into place.
3. To remove the seat, grasp seat and lift firmly.
4. From the front of the frame, slide the bucket (without lid) into the pail backrest under the seat.
5. Attach the back support by depressing the brass push pin and pushing until both are locked securely in place, An audible click will be heard.
6. The commode is now ready for use.

Adjust commode height

1. Depress the height adjustment button and slide the lower leg upward or downward until the height adjustment button is fully engaged in the desired hole.
 2. Repeat for other three leg attachments.
- *Caution: Ensure that all four commode legs are adjusted to the same height, and that all height adjustment buttons protrude fully through adjustment holes before use.

Use as a bedside commode

1. Set commode to desired height.
 2. Slide commode pail under platform seat into pail support backrest. Ensure that pail is properly positioned below commode opening.
 3. When user is finished using commode, place commode pail lid over commode pail to contain odors.
 4. Remove pail and lid simultaneously to empty pail. Clean seat and pail with disinfectant after each use.
- *Caution: It is NOT recommended to remove the backrest when using as a bedside commode. Serious injury may result.

Drop-Arm usage

1. To lower drop-arms, pull drop-arm mechanism outward and gently move arm downward to rest alongside the frame.
2. To raise drop-arms, gently pull up arms until they are fully seated in latches. An audible click will be heard.

Beware of the risk of pinching when using

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry after each use.
- Periodically clean the frame with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant can be used.
- Check that all 4 legs touch the floor at the same time without any wobble.
- Check the product for cracks and damage, especially the seat and plastic tube.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer. Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check the condition of the seat		X
Check the height adjustment and that the clips are fully extended	X	
Check the folding function of the armrests	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 66,5 x depth 55 cm x height 52 cm
Height-adjustable from 45 to 57 cm	
Seat Dimensions	Width 28 cm x depth 20,5 cm
Structure	Steel
Weight	7 kg
Maximum weight	136 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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