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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Tasmania 4 plates bed tables are medical devices. They allow the user to read or take a meal in bed or in an armchair. They are adjustable in height and tilt to adapt to the needs of the user during the day.

PRODUCT DESCRIPTION

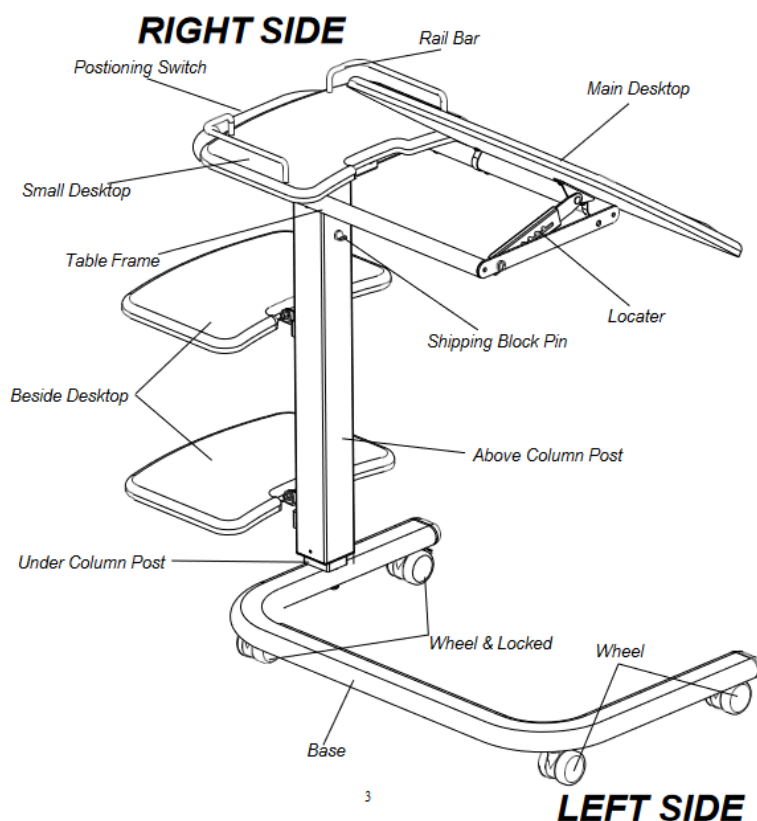SAFETY PRECAUTIONS


Failure to observe the precautions for use may result in the risk of serious injury.

- Use the bedside table only as described in this manual..
- Install the bed table on a flat surface.
- Store the bedside table in an indoor environment, away from moisture and dust.
- This bedside table uses a high-pressure pneumatic cylinder to adjust the height of the bed top. Do not remove the locking pin from the column assembly before use.
- Before use, make sure that all parts are present and in good condition. In case of damage, please contact your dealer.
- Carefully follow the assembly instructions and ensure that all components are properly secured before use.
- Before use, tighten the brakes.
- If any parts are damaged, broken or worn, contact your dealer.

MOUNTING

INSTALLATION

Place the bedside table on a clear surface and remove the packaging. Do not throw away the packaging before the assembly is completed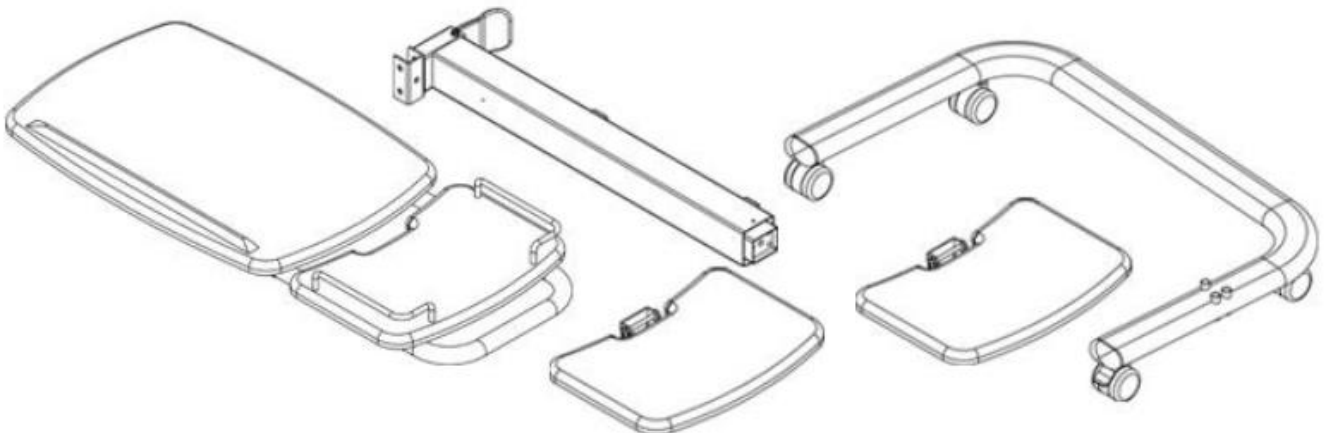

Set the Overbed Table in a cleared area and remove all packing materials. Do not dispose of the packing materials until assembly is completed.


Unpacking your **Overbed Table** where will be using it. Being careful

Not to damage . Place your **Overbed Table** on a level flat surface.

It is recommended that you place a protective covering on your floor.

STEP 1 Place table top upside down on a level surface as show at right.


Position column bracket above table top bracket as shown at right, with holes matched, and lower column onto table top. Insert socket head cap bolts through column bracket into table top bracket as shown. Use M8 hex key to tighten the bolts.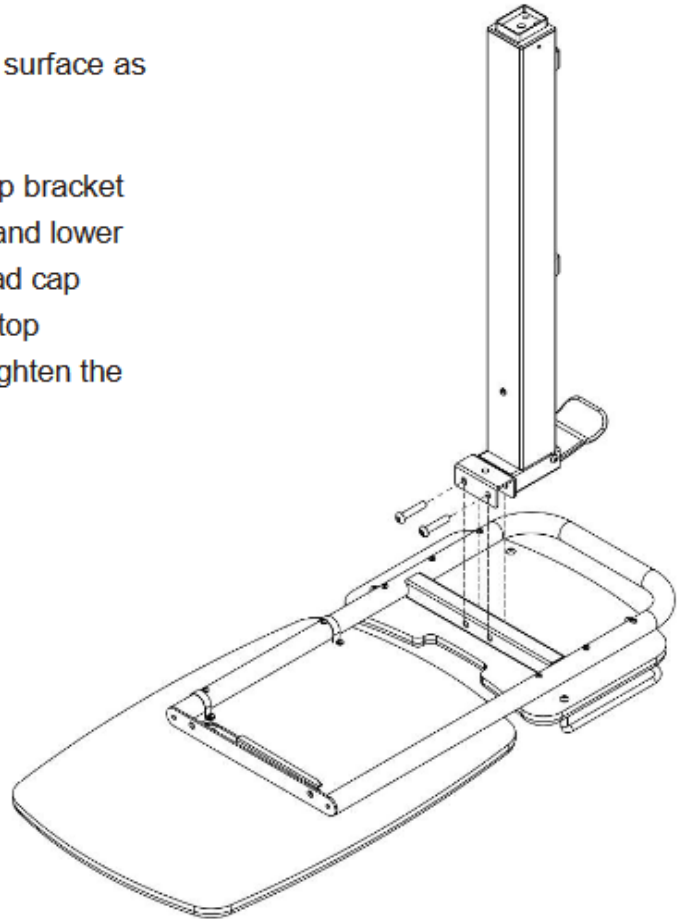


WARNING:

Do not remove shipping Block Pin from column assembly prior to assembly.

STEP 2 Place the base upside down above the column as shown at left. Install the 3 bolts through the bolt plate, base, and into column end as shown.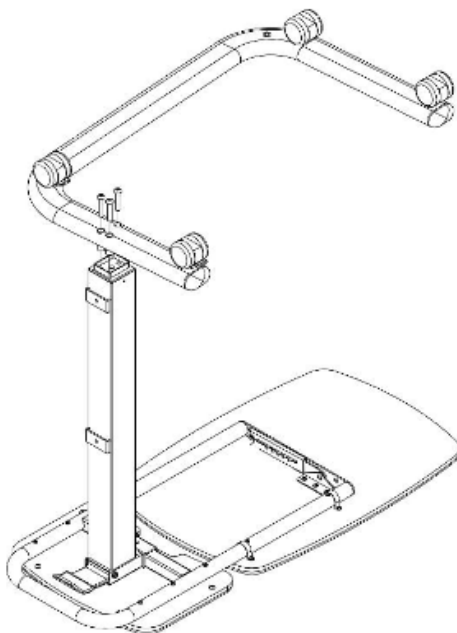



STEP 3 Install the Beside desktop. Insert bracket of beside desktop unit to position column bracket as shown. Use Cross hex key to tighten the bolt.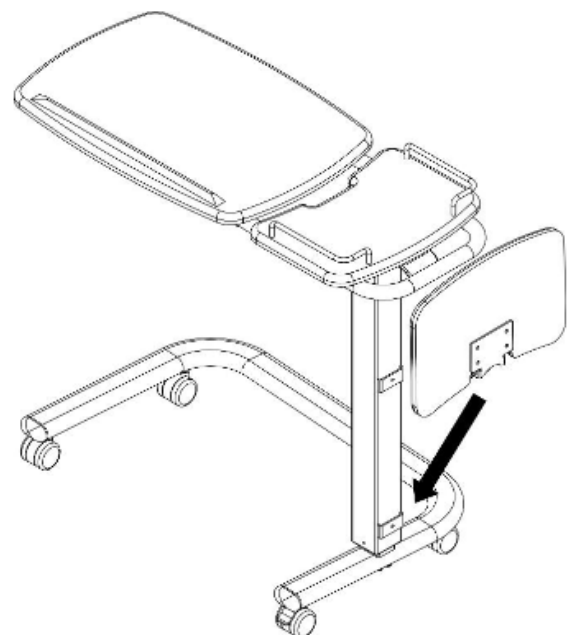

STEP 4 Turn the Table UPRIGHT and place it on the floor.

Remove shipping Block Pin from column assembly .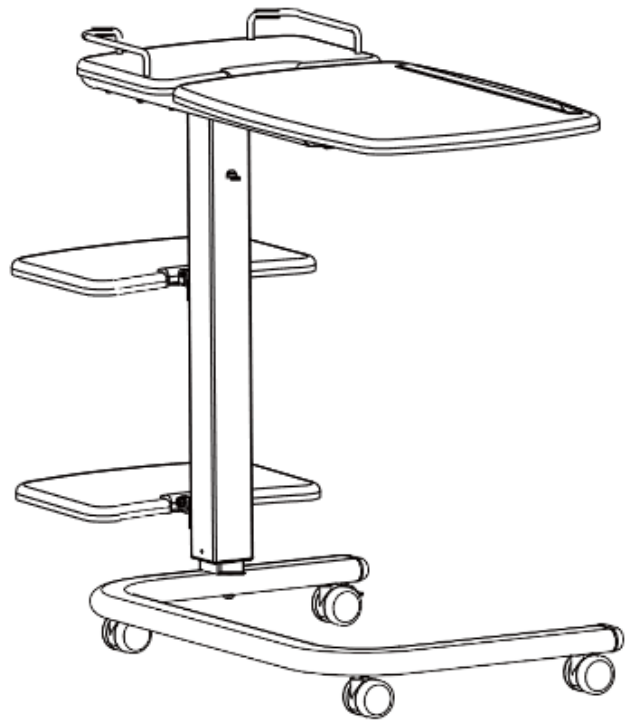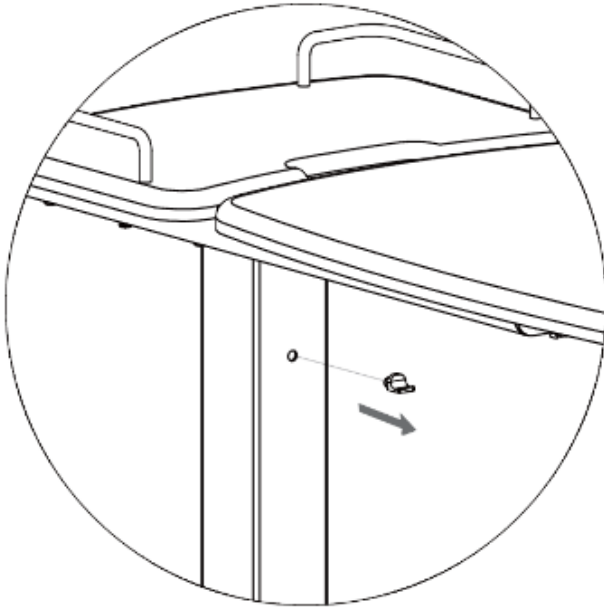


Make sure that all parts are tightened before you use the Overbed Table.

Titling the Overbed Table

Note : The table MUST be in the upright position BEFORE titling.

1. Pull up the edge of top table to activate table top tilting, there are 6-position-ratchet is reversible.
2. Tilt the table top to the desired angle.
3. Tilt the table top to the highest angle, and put down the table top softly , when you want to return to the level position.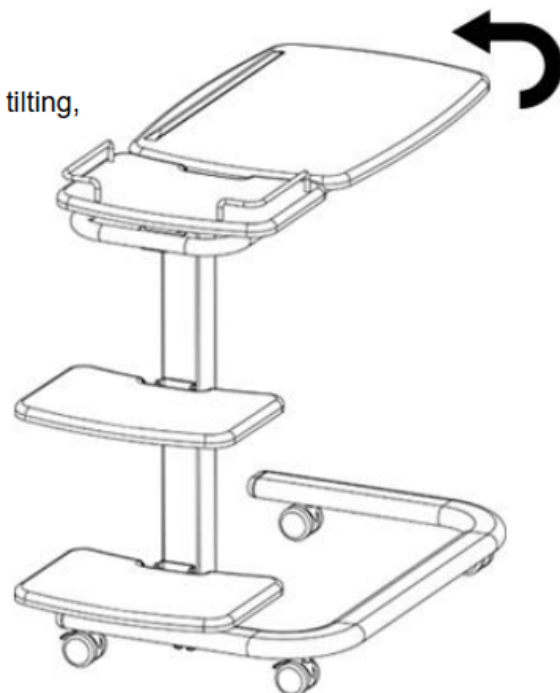

Adjusting the Hi -Low of Overbed Table

Note : The Overbed table MUST be in the upright position before Hi-Low adjusting.

Table top up - Pull the table top up.


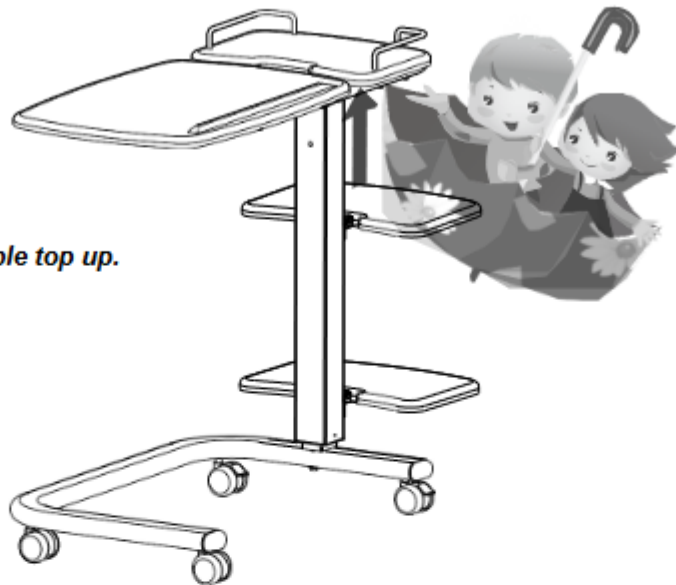
Table top down -lift up the switch and push the table top down

CARE AND MAINTENANCE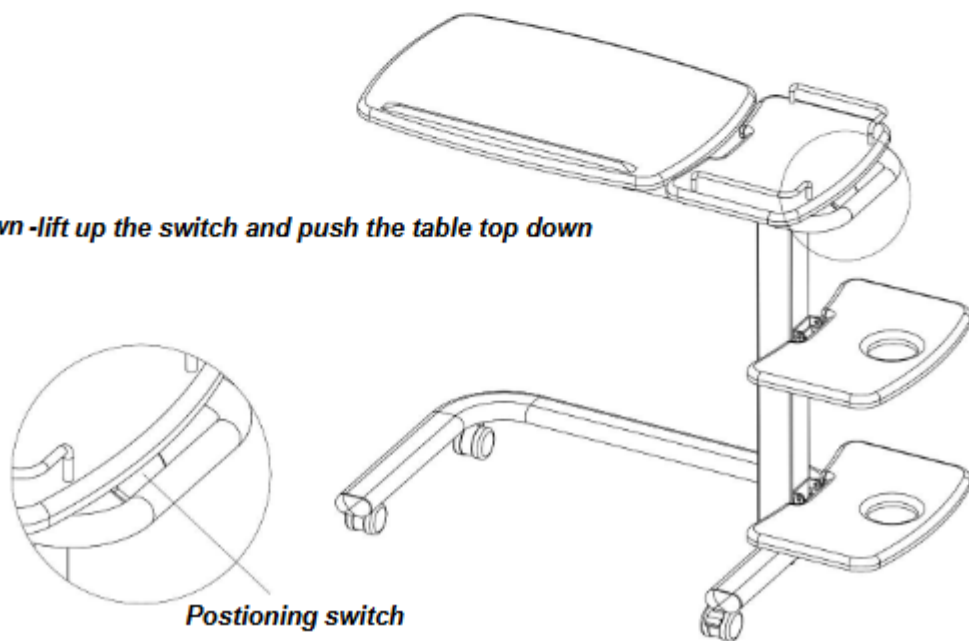

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water, a soft cloth and a neutral detergent.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Periodically check all screws and bolts for wear and strength. Replace all broken, damaged or worn parts immediately.
- Periodically check the cylinders.
- In the event of a malfunction, do not repair the table yourself. Consult an authorised dealer.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL SPECIFICATIONS

Base dimensions	Width. 71 cm x Depth. 45 cm
Height adjustable	79 à 114 cm
Tray dimensions	39 cm à 80 cm
Dimensions of auxiliary plates	25 x 35 cm
Weight	18 kg
Maximum weight supported	25 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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