FAUTEUIL RELEVEUR ZERO G

REFS. 821031 - 821031.G

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FR

INTRODUCTION

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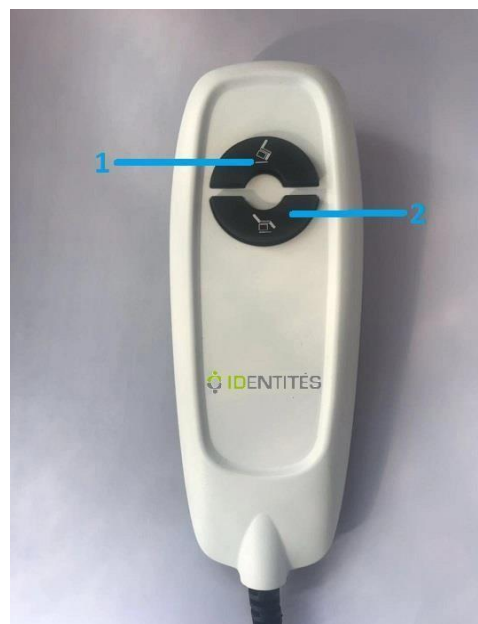
EN

INTRODUCTION

The Zero G lift chair is designed to help people get out of the chair easily, using electronically controlled motors. Please read the instructions carefully to ensure correct use of the chair, particularly the notes on safety.

SPECIFICATIONS

| | |
|--------------|-----------------------------------|
| Seat width | 50 cm |
| Seat depth | 50 cm |
| Seat height | 53 cm |
| Back height | 71 cm |
| Chair Weight | 58 kg |
| Maximum Load | 155 kg |
| Power supply | 100 - 240V // 50Hz - 40W |
| | Motor OKIN |
| Materials | PU and microfibre covering |
| | Wood and steel frame |
| | PU foam & polyester fibre filling |

SECURITY

1. Assemble the chair correctly in accordance with the enclosed assembly instructions. The chair manufacturers, their agents and distributors are not responsible for damage caused by incorrect assembling.

2. Place the chair on a level and flat floor.
3. Do not use the chair in an excessively damp or dusty environment to avoid the risk of electric shock and mechanical problems. The chair should be placed in a dry room. Do not use the chair when the room temperature exceeds 40 degrees Celsius.
4. Do not use the chair outdoors.
5. If the chair is not to be used for a long period of time, unplug the power cord from the wall socket to avoid overheating.
6. Do not place heavy objects on the armrests, footrest or backrest.
7. Do not jump or play on the chair. Children using the chair must be supervised by an adult. The chair is not designed for children.
8. Ensure that the voltage corresponds to the specifications indicated for the chair. The electrical connection must be made in accordance with local regulations. Do not disconnect the plug from the wall socket suddenly and do not disconnect by pulling on the cord.
9. Do not connect or disconnect the plug from the wall socket with wet hands. There could be a risk of an electrical shock or mechanical failure.
10. Do not use the chair if the adaptor cord or plug is damaged or if the electrical socket is excessively loose.
11. Be careful when operating the elevation function when the backrest of the chair is reclined. The backrest must be in the upright position when the user is sitting in the chair.

In the event of a malfunction, please contact your dealer.

Note: All repairs must be carried out by qualified technicians.

Warning: A sudden rise in temperature may cause condensation on electrical or mechanical parts. It is recommended that the chair be left at room temperature for at least one hour before use to ensure proper operation and reduce the risk of mechanical failure.

Do not use the chair in the following situations:

- a) Accidental spillage of water on the appliance. This could cause an electric shock.
- b) Tearing of the cover, exposing the internal mechanics.
- c) Failure or any other abnormal condition during operation.

Be careful in the event of a power failure. There is a risk of injury if the power is unexpectedly restored and the operating buttons are activated.

Avoid using the chair during violent thunderstorms.

Switch off the power control immediately and unplug the power cord from the wall socket in the event of visible damage or malfunction.

STEP 1

STEP 2

STEP 3

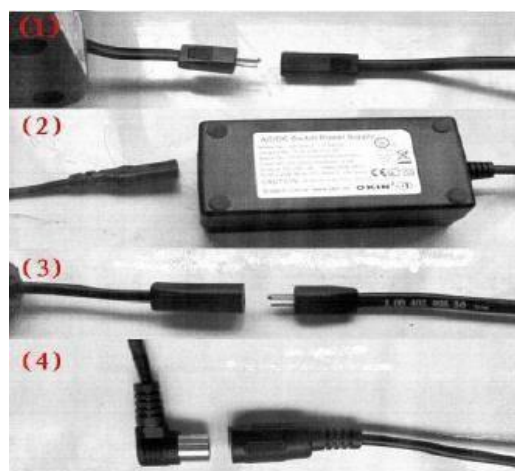
Note: The two rear supports must be fitted onto the vertical bars at the same time to prevent them from jamming.

Note: Do not assemble the chair if the covering or any part of the chair is damaged.

ELECTRICAL CONNECTIONS

Note: The wires must be connected correctly for proper operation.

1. Motor to wire connector
2. Wire to adapter connector
3. Adapter connector to plug
4. Remote control connection

POSITIONING THE WHEELCHAIR

Leave enough space at the front and rear of the chair to allow it to recline.
To allow the chair to recline freely, leave a minimum space of 40 cm behind the chair.
Leave at least 65 cm in front of the chair.
Place the chair on a level surface or floor in a dry room.

Warning

Remove any obstruction from behind or in front of the chair to allow safe reclining and elevation during operation.

The area between the seat and the footrest must be free of obstructions to avoid the risk of pinching when the chair is in use.

MAINTENANCE AND STORAGE

1. Storage

Wipe away dust and dirt. Keep the chair clean and out of areas of high humidity.

If the chair is not to be used for a long period, cover it with a cloth to protect it from dust. Store in a dry place. Do not store the chair in direct sunlight or in areas with high temperatures. Exposure could cause the fabric to fade.

2. Care

Before cleaning, make sure the switch is off and the power plug is disconnected from the socket.
Do not use wet wipes to clean chairs with imitation leather (polyurethane) upholstery unless it is our incontinence-resistant imitation leather. Leather cleaning agents may be used.

3. Chair body

Use very mild soap to clean the plastic, structure and upholstery of the chair. Gently wipe away any traces of soap with a dry cloth. To clean a dry stain, soak a piece of soft cloth in warm soapy water, wring out well and clean the stain.

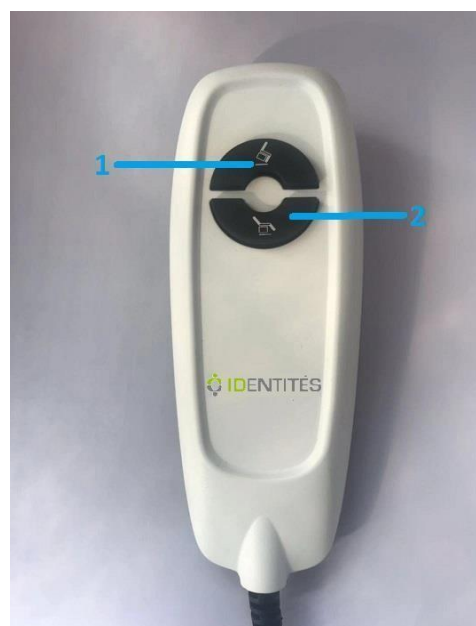
Do not use cleaning agents such as benzene or thinners, or any other cleaning agent that could discolor the fabric or scratch the covering. Do not remove the upholstery for cleaning. It is strictly forbidden to iron the upholstery.

USING THE REMOTE CONTROL

The ZERO G chair is equipped with a vertical raising and lowering function. This function raises or lowers the chair when necessary to enable the user to transfer safely from the chair to bed or another surface.

Use only a dry cloth to clean the remote control. Excessive moisture may damage the remote control.

1. Raise the chair
2. Tilt the backrest, lower the footrest and Zero Gravity function

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