TABLE DE LIT CORFOU

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

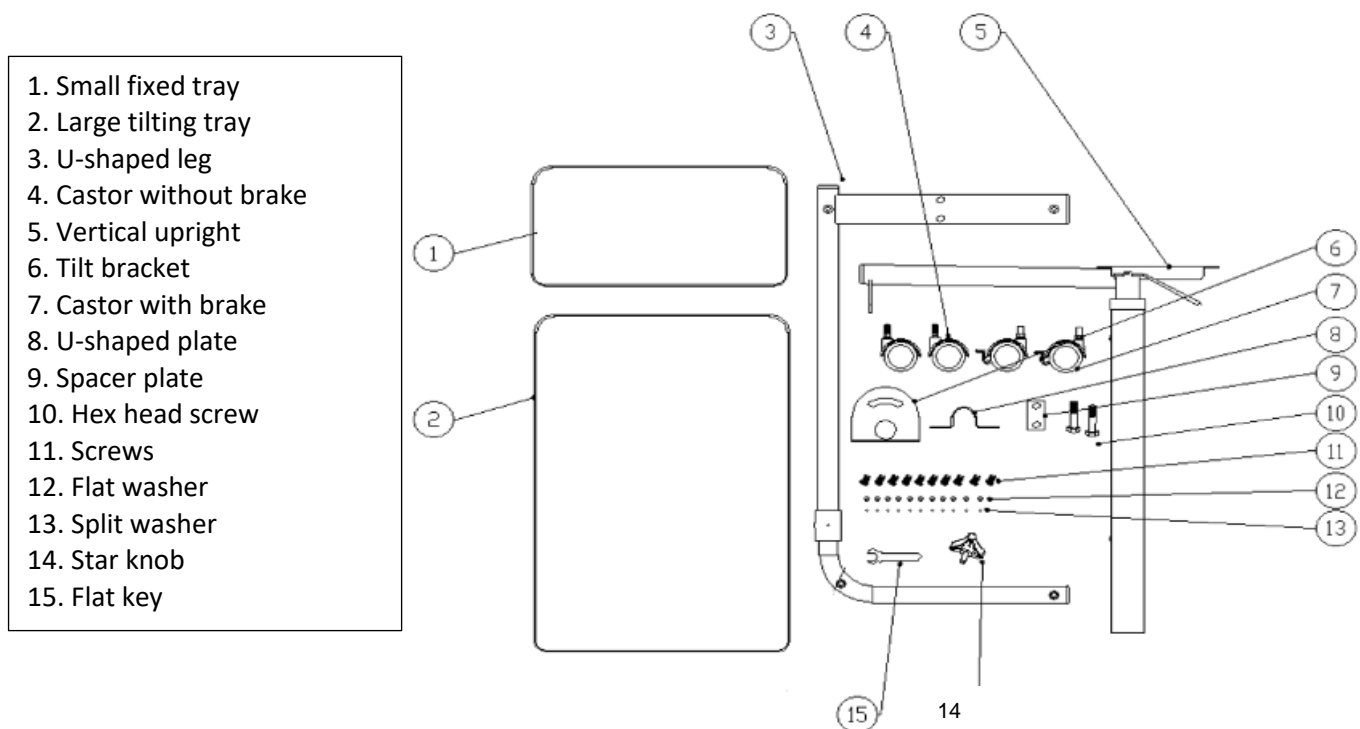
Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Corfou bed table is a medical device. This product allows you to read and eat in bed in complete comfort. The height and tilt of the table top can be adjusted to suit different times of the day. The height can be easily adjusted using the jack.


PRODUCT DESCRIPTION

823125 (no tray no. 1) - 823126 :

823127 :

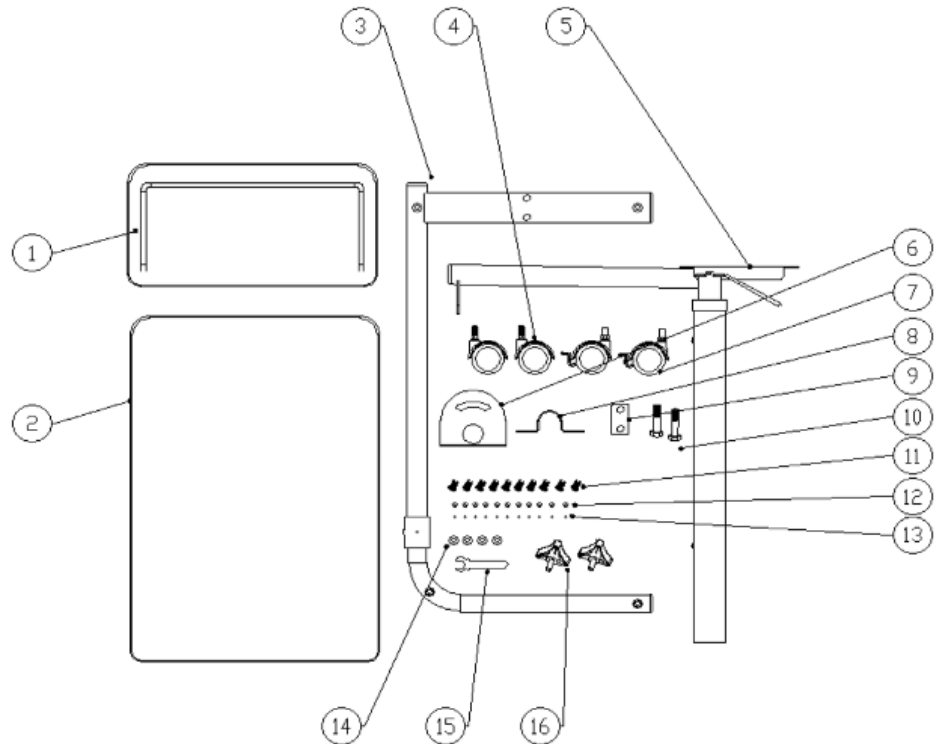1. Fixed tray
2. Tilting tray
3. U-shaped leg
4. Castor without brake
5. Vertical upright
6. Tilt bracket
7. Castors with brake
8. U-shaped piece
9. Washer plate
10. hex head screw
11. screw
12. washers
13. spring washers
14. Nuts
15. Flat key
16. Star knob


MOUNTING AND USE

823125 (no small fixed plate no. 1) - 823126 :

Step 1 :

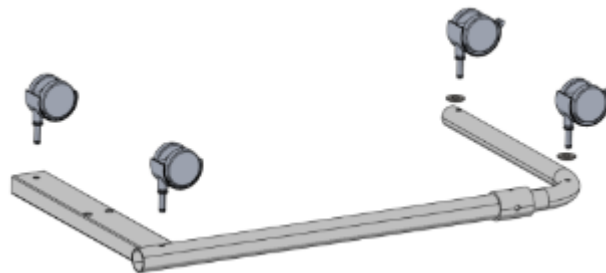
Screw the wheels ④ and ⑦ under the U-shaped foot ③ using the open-ended spanner.

Step 2 :


Place the ① and ② trays (holes facing up) on a flat surface. Align the holes in the vertical ⑤ with the holes in the small tray ①. Prepare the screws ⑩ by first threading a split washer ⑬, then a flat washer ⑫ on each screw. Screw the upright plate ⑤ onto the small plate ①.

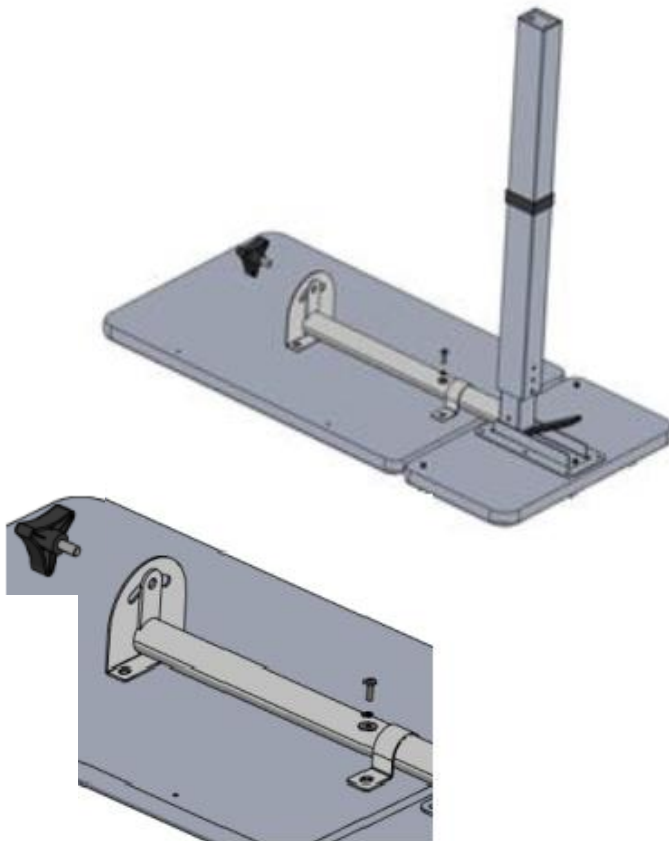
Place the U-shaped plate ⑧ around the tube and screw it onto the large plate ②. Place the tilt bracket ⑥ and screw it onto the large plate ②. Screw and tighten the large plate with the star knob 14.

Step 3 :

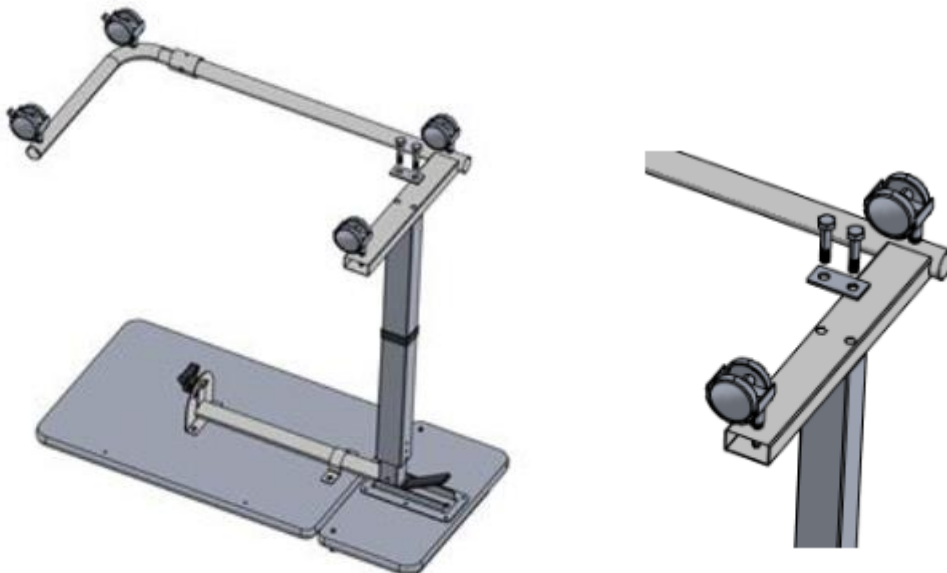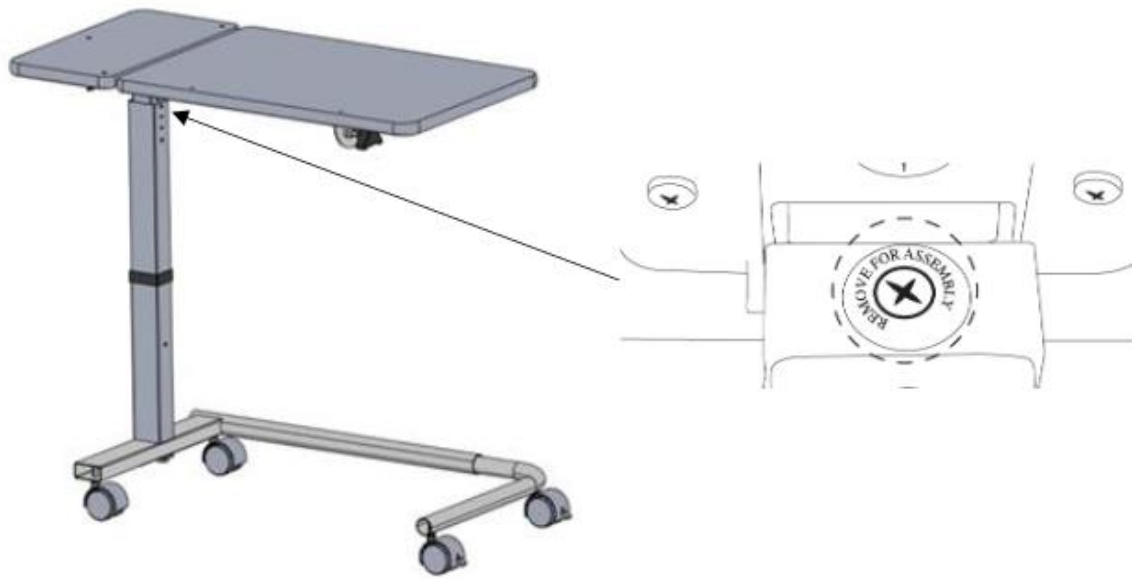
Install the U-leg ③ on the upright ⑤ and place the spacer plate ⑨, aligning the holes. Assemble with the hex head screws ⑩. Tighten with spanner.

Use and precautions : Before using for the first time, loosen the black screw located at the top of the vertical upright ⑤, just below the tray ①. This screw is used to lock the upright jack. It should be retightened if the bed table is to be transported.

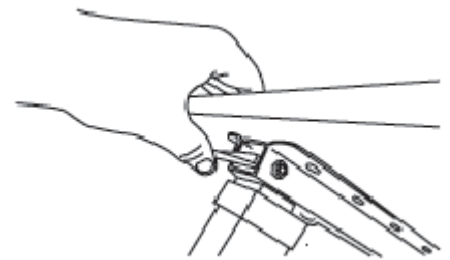Adjust the height of the Corfou bed table to the correct height by pulling on the vertical post adjustment lever while raising or lowering the tray, then release the lever. Locking and unlocking the brakes on the castors ⑦ is done using the small lever placed on each castor.

To tilt the large platter, slightly loosen the star knob 14, tilt the platter and then tighten the star knob again.


Caution: Check that the table is correctly assembled and tighten the screws regularly. Always use on a flat surface. Always keep the brakes locked when the table is not in use. Do not overload the table.

823127 :

Etape 1 : Screw the wheels ④ and ⑦ under the U-shaped foot ③ using the open-end spanner.


Etape 2 : Place the ① and ② trays (holes facing up) on a flat surface. Align the holes in the vertical upright plate ⑤ with the holes in the small plate ①. Prepare the screws ⑪ by threading in first a split washer ⑬, then a flat washer ⑫ on each screw. Screw the upright plate ⑤ onto the small plate ①.

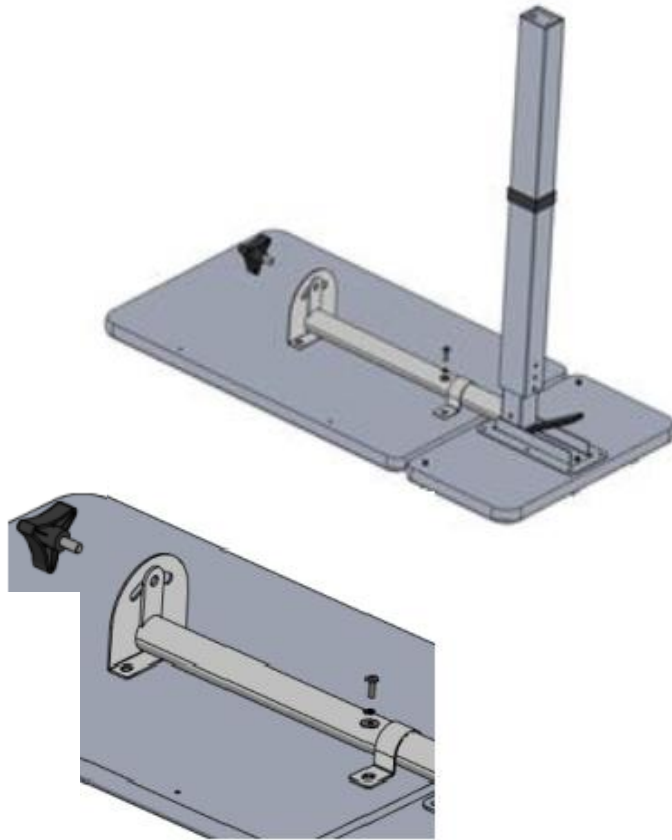Place the U-plate ⑧ around the tube and screw it onto the large plate ②. Place the tilt bracket ⑥ and screw it onto the screw it onto the large plate ②. Screw and tighten the large plate with the star knob ⑭



Etape 3 :

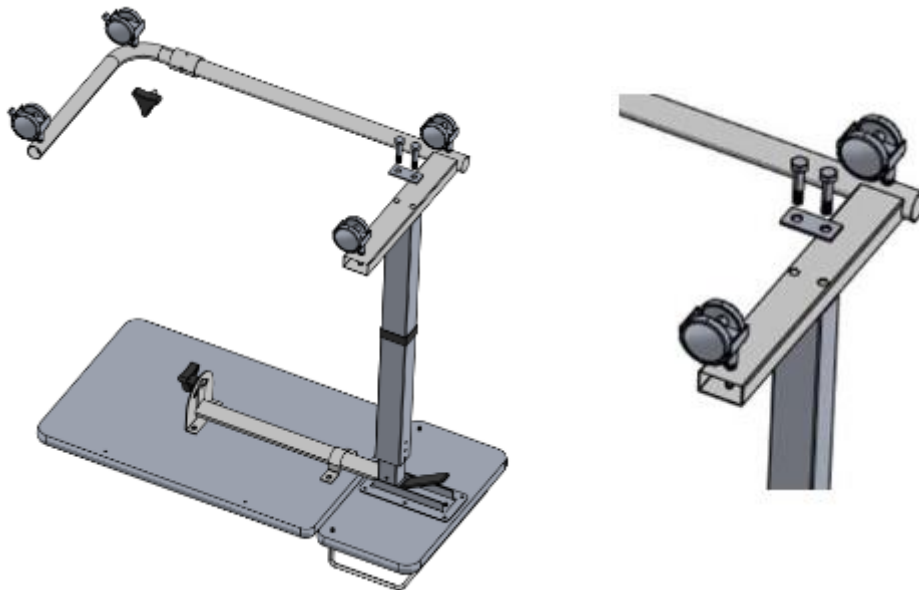Install the U-foot ③ on the upright ⑤ and place the spacer plate ⑨, aligning the holes. Assemble with the screws ⑩. Tighten with the open-ended spanner.

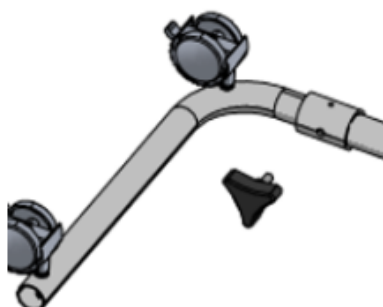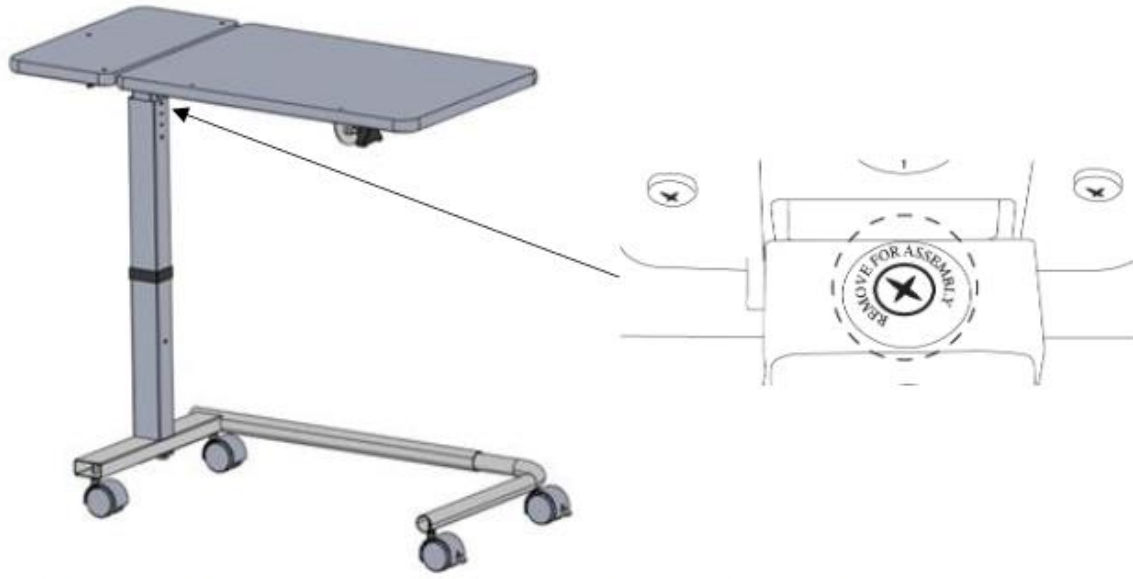
The passage width of the U-shaped foot ③ can vary from 60 to 78 cm. To increase or reduce this passage width, unscrew the knob ⑬, adjust the width, then screw the knob back in.

Use and precautions : Before using for the first time, loosen the black screw located at the top of the vertical upright (5), just below the tray (1). This screw is used to lock the upright jack. It should be retightened if the bed table is to be transported.

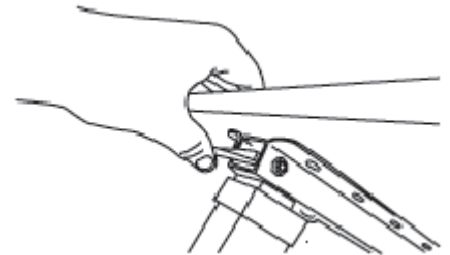Adjust the height of the Corfou bed table to the correct height by pulling on the vertical post adjustment lever while raising or lowering the tray, then release the lever. Locking and unlocking the brakes on the castors (7) is done using the small lever placed on each castor.

To tilt the large platter, slightly loosen the star knob 16, tilt the platter and then tighten the star knob again.

Caution: Check that the table is correctly assembled and tighten the screws regularly. Always use on a flat surface. Always keep the brakes locked when the table is not in use. Do not overload the table.

SAFETY PRECAUTIONS

Failure to observe the precautions for use may result in the risk of serious injury.

- Use the bed table only as described in this manual.
- Do not exceed the maximum weight of 15 kg
- Install the bed table on a flat surface.
- Store the bed table indoors, away from mould and dust.
- Check that the adjustment knobs work properly before use.
- Before use, check that all parts are present and in good condition. In the event of damage, please contact your dealer.
- Carefully follow the assembly instructions and ensure that all components are correctly secured before use.
- Before use, apply the brakes to the castors.
- If any parts are damaged, broken or worn, please contact your dealer.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly using lukewarm water, a soft cloth and a neutral detergent.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Dry with a soft, clean, dry cloth.
- Make sure there is no accumulation of dirt on the product.
- Periodically check all screws for wear and strength. Replace all broken, damaged or worn parts immediately.
- Regularly check the operation of the brakes on the castors and the operation of the jack.
- In the event of a malfunction, do not repair the table yourself. Consult an authorised dealer.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL SPECIFICATIONS

Dimensions	823125 : Dimensions of top: width 76 x depth 38 x height 72 to 99.5 cm. Leg dimensions: 69 x 37 cm. Clearance width 60.5 cm. 823126 : Dimensions large tray/small tray width 56/20 x depth 38 x height 72 to 99.5 cm. Leg dimensions: overall 69 x 37 cm. Clearance width 60.5 cm. 823127 : Dimensions large tray/small tray: width 56/20 x depth 38 x height 72 to 99.5 cm. Clearance width: 60 to 78 cm
Weight	823125 : 9,5 kg 823126 : 9,8 kg 823127 : 11,3 kg
Maximum weight supported	15 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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