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EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise on the most appropriate use for you.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT DESCRIPTION

The wheelchair mouvlight are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

## PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum supported weight of 135 Kg.

Make sure that all parts are well fixed and in good working order.

Do not use the product on stairs or escalators.

Only use the accessories and parts supplied by the manufacturer.

Check the correct operation of the brakes before use.

This wheelchair has been designed for use inside vehicles of all kinds. Always transfer the wheelchair to the user in a seat approved by the vehicle manufacturer and use all available safety restraints.

Do not use this wheelchair in roads, streets or other surface traffic.

Be sure to fully activate the wheel brakes when getting out of the wheelchair, or when you are in a lift or wheelchair lift. The wheelchair must be on a flat, stable and level surface.

When getting in or out of the wheelchair, do not climb or put any weight on your foot. When you try to extend your arms over your back, do so only until your arm can be extended without changing your sitting position.

Do not stand in the wheelchair

If you have to lean or lean on the chair, make sure that the front wheels are pointing forward, so that the base of the wheel extends to stabilise the chair.

Seek professional advice when going up or down slopes of more than 10 cm. If you have to stop on a slope, avoid abrupt and rapid movements when moving. Movement can cause instability due to the angle of the chair. The use of brakes can lead to accidental braking, uncontrolled movement or sudden blockage. Try not to change direction on a ramp, as this may cause instability and cause the chair to tip over.

During use, lock the brakes for all actions.

All wheels must always be in contact with the ground.

When the product is in a stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.

Make sure that the footrests are properly engaged.

Make sure the removal pin is securely engaged in the rear wheels and frame of the chair.

Be careful when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the product on a wet, icy or snowy slope. Beware of the risk of pinching during use.

If any parts are damaged, broken, worn, or missing, contact the dealer.

## **MOUNTING**

### **SEAT HEIGHT**

In this chair, the seat height is adjusted with the rear wheel support bracket where the axle is located. Once the seat height is adjusted at the rear, adjust the front part correctly so that the seat is not tilted forward or too far back.

Change the fork or change the position of the front wheel by choosing another hole in the fork. The purpose of adjusting the seat height is to maximise the user's efficiency by adapting the seat to the environment according to the user's physical and mechanical needs.

### **ADJUSTING FOOTRESTS ADJUSTMENT**

The Footrests are folding inward for easy transfer. The footrest of the footrest is adjustable in degrees of inclination.

**Note:** To remove the footrest, squeeze the trigger toward the chair, rotate the footrests outward or inward and raise them. To install the footrests, place the plastic swivel on the tubing in front of the seat, turn the footrests inward, until you grasp the locking bolt.

### **FIXING SYSTEM-ADJUSTING REAR WHEEL**

The fixing-adjustment system is the most important adjustment for the wheelchair, because it determines the center of gravity, the rotation of the chair, the degrees of the wheels and the height of the seat. The center of gravity is determined by the user's position on the chair. You can change its Handle Attendant brake vertical coordinate by changing the height of the seat, but the higher the seat, the more unstable the seat.

The position of the rear axle with respect to the horizontal coordinate of the center of gravity determines the ease of propulsion and handling of the chair, but also the subsequent stability.

### **ARMRESTS**

To allow easy access from each side of the chair, the armrests are folded down. Check that they are locked when they are replaced.

### **OPTIONS ET ACCESSOIRES**

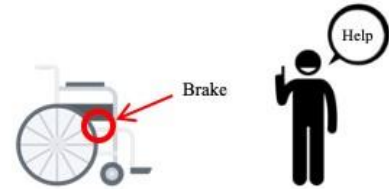
Small transit wheel
Quick release
Seat cushion

# USE


## 1 - WHEELCHAIR USE

Make sure that the front wheels are in the front and the rear wheels braked when sitting on the wheelchair you sit in the wheelchair and transfer to another location (chair, bed, etc.).

Brake the wheels every time you raise the wheelchair, and if necessary, ask for help from an assistant or family members for he

## 2 - HOW TO GET OUT OF THE WHEELCHAIR ALONE

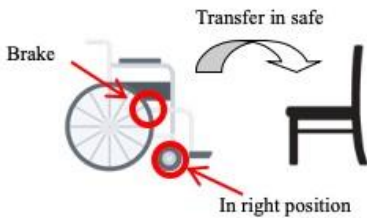Avoid bending too far down, or sideways, or backward, to catch or SOLIT objects. There is a risk of overturning or losing balance especially if the chair has a reduced seat and / or a high seat height (eg cushion).

Make sure that the front wheels are in the front and the rear

wheels braked when sitting on the wheelchair transferring to another location (chair, bed, etc.). Don't do this and keep in balance!!!

## 3 - OVERCOMING OBSTACLES :

Ask for an assistant, as long as you have to overcome obstacles larger than 5 cm.

Unassisted: the wheelchair can tip over and causing damage to the person and the chair; the chair may be blocked and the rear wheels may be suspended in the air.

## 4 - UP AND DOWN :

### GOING UP

Do not hang anything above the backrest or above the cuffs. Dispose the chair to tip over.

To go up hills, you only install the anti-tippers. To this end insert the antitippers into the rear tub and take care that the safety clip comes out of the hole in the back tube.

### GOING DOWN

Please help me when you are going down a slope

Be careful that the wheels are not blocked in the ground when going down a slope

### 5 – GOING UP AND DOWN THE STAIRS :

To climb a ladder, once the chair is tilted backwards by a person holding the chair by the handles, the rear wheels rest on the steps. The second person must hold a fixed part of the front area of the frame, lift the seat and hold it until the first person stands on the next step and so on.

Do not release the chair until you have reached the last step and moved away from the ladder.

### 6 -USING AN ELEVATOR LIFT :

Automatic door closing can cause hand injuries !

**Practical :** For the safety of your wheelchair, in and out of the house, you need to be confident in the main manoeuvres, such as pushing, braking, going back and forth on flat surfaces (can be done with the help of an assistant). Avoid slopes, uneven surfaces and similar situations in the first place.

Crossing narrow steps: When driving through narrow spaces, such as building entrances, doors, etc... Watch out for elbows. - Danger of injury!

### 7 – TRANSPORT OF THE CHAIR

1. Remove the footrests.
2. Remove the armrests.
3. Remove the wheels in case they take a quick release.
4. Hold the chair by pulling the handles on the sides of the seat until the chair is completely folded.
5. Put it in a car.

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use. Check the brakes by pushing the handles into the locked position, then trying to push the product.
- Check the upholstery has no cuts, is not torn or heavily used. Replace if necessary.
- If there is a malfunction, do not repair the product yourself. Consult an authorized dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the product	Anytime	Every week	Every month
Screws and fasteners		X	
Stability of frame and backrest			X

Condition of the seat and wheels			X
Functioning of brakes and anti-tilt castors	X		
Function of armrests		X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## TECHNICAL SPECIFICATIONS

DESCRIPTION	876001.40	876001.42	876001.45	876001.48	876001.50
Overall Dimensions	Width 62 x height 90 x depth 102 cm	Width 64,5 x height 90 x depth 102 cm	Width 67 x height 90 cm x depth 102 cm	Width 69,5 x height 90 cm x depth 102 cm	Width 72 x height 90 cm x depth 102 cm
Seat Dimensions	Width 40 x height 47 x depth 42 cm	Width 42,5 x height 47 x depth 42 cm	Width 45 x height 47 x depth 42 cm	Width 47.5 x height 47 x depth 42 cm	Width 50 x height 47 x depth 42 cm
Folded Dimensions	36 cm				
Front caster size	20 cm	20 cm	20 cm	20 cm	20 cm
Rear caster size	61 cm	61 cm	61 cm	61 cm	61 cm
Metal Grey Color					
Weight	15 kg	15,4 kg	15,8 kg	16.1 kg	16,5 kg
Maximum Weight authorized	135 kg	135 kg	135 kg	135 kg	135 kg

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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