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EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION


The RS1 Transfert Disc helps a carer or attendant to assist a patient or user to transfer to another seat, bed, wheelchair, toilet or commode by standing and turning.

The device is only suitable as a standing and turning aid. The device should not be used to move the patient other than as stated above.

It is designed for individual patient use up to 200Kg and should only be propelled when there is nobody stood on the device.

## PRODUCT DESCRIPTION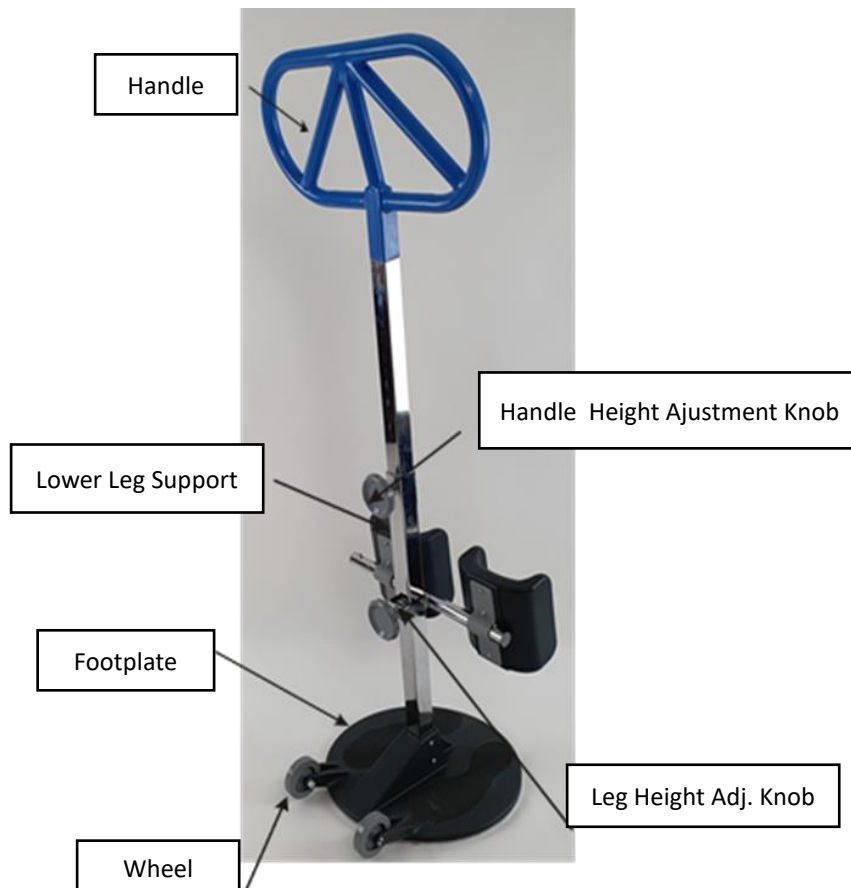


## SAFETY PRECAUTIONS


Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 200 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Do NOT tilt or lever the Transfer Disc during use. The whole footplate should remain in contact with the floor at all times.
- Only use the product on a flat surface.
- The carer / attendant must always apply counter-pressure when the user is standing up, sitting down or transferring.
- The carer must always support and counterbalance the device when used by a user. If the patient uses the device to move to or from a wheelchair, the wheelchair brakes should always be applied and the legrests removed or swung out.
- Ensure both the carer and patient wear suitable footwear.
- Do not leave a patient unattended on the device.
- In case of damaged, broken, worn or missing parts, contact the dealer.

## MOUNTING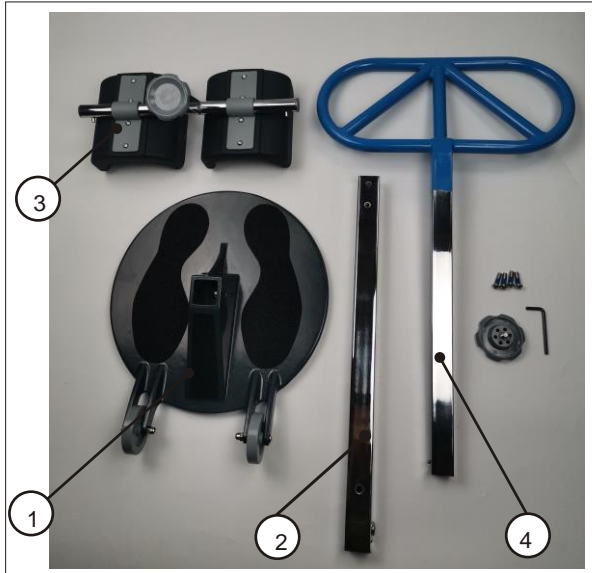

1. Foot Base
2. Handle Receiver Tube
3. Lower Leg Support
4. Handle

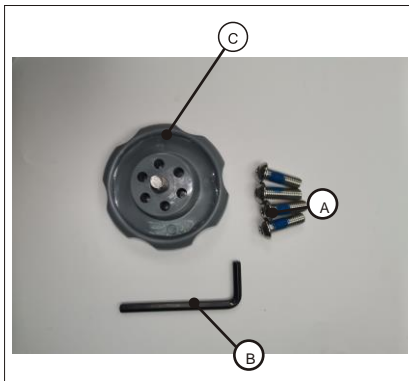
- A. Fasteners  
Screws (4No.)  
a. Hex. Allen  
Key  
C: Height  
Adjustment Knob

**Step 4**  
Slacken the knob on the Lower Leg Support (Item 3) by turning the Height Adjustment Knob (Item C) anticlockwise until it is possible to slide the unit over the top of the handle receiver (Item 2). Once the Lower Leg Support is attached, lower the support to the required height and then tighten the Height Adjustment Knob by turning it clockwise until it locks onto the tube.

Carefully unpack all parts of the RS1 Patient turner and place the base on a flat surface

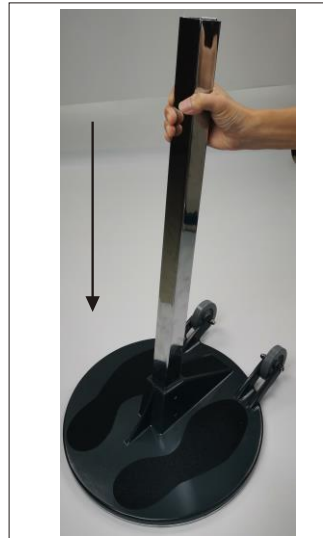**Step 1**

Position the handle Receiver tube (Item 2) into the centre section of the Foot base (Item 1) as shown.

**Step 2**

Position the screws each of the four holes in the upright of the base and tighten them one at a time with the Hex. Allen key supplied

**Step 3**

Slide the spring steel onto the handle receiver (Item 2) if it is not already fitted to the top of the receiver.

### **Step 5**

Insert the Handle (Item 4) into the Handle Receiver tube (Item 2) until the Handle small spring button reaches the top of the Handle Receiver tube. Press the spring button in until it allows the Handle to drop down to the point where the spring button clicks into the hole in the side of the Handle Receiver tube.


### **Step 6**

Adjust the handle height by pressing the button as shown and then slowly push down on the handle until the desired height is reached.

### **Step 7**

Insert the handle Height Adjustment knob (Item C) into the rear of the Handle Receiver tube (Item 2). Turn the knob clockwise until it locks the handle at the required height.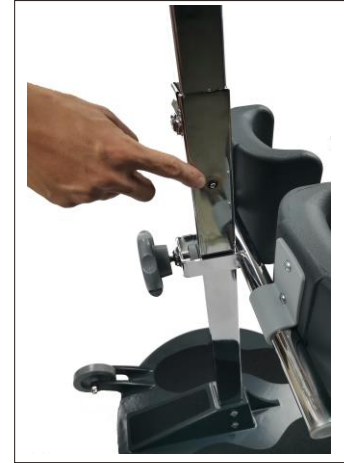

### **Step 8**

Your RS1 Patient Turner device is now complete and ready for use. Please **DO NOT** attempt to transport the unit while the patient is standing on the unit.

**ALWAYS** ensure that the Lower Leg Support (Item 3) and Handle (Item 4) are positioned at the correct level to supply optimum comfort and support to each patient that is using the device.

## **USE**


### **HEIGHT ADJUSTEMENT**

The height of the Handle and Lower Leg Support can be adjusted to suit the user and/or carer.

Turn the Handle Adjustment Knob anticlockwise to loosen the Handle. This is shown by direction A in the photo below.


Then, reposition the Handle to the required height as shown by the direction B in the photo below.

Finally, turn the knob clockwise until the Handle is secured. This is shown by direction C in the photo below.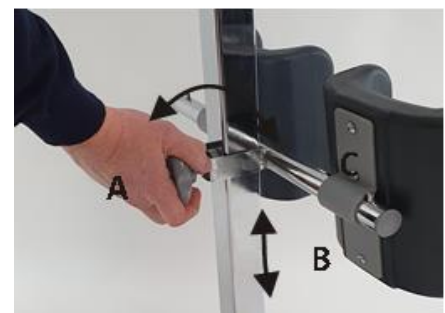

If the Handle is raised too high it will lock, which can be identified with an audible click. If this occurs, press the push button (shown) whilst pushing down on the Handle to release the Handle.

After performing this, adjust the Handle height as previously describes.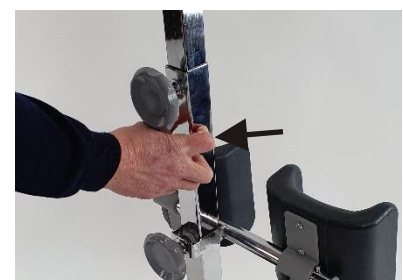


### **To Adjust the Height of the Lower Leg Support :**

Turn the Lower Leg Adjustment knob anticlockwise to loosen the Support. This is shown by direction (A) in the photo above.

Then, reposition the Lower Leg Support to the required height by moving the bracket up and down, as shown by (B) in the photo above.

Finally, turn the knob clockwise until the Lower Leg Support is secured. This is shown by direction (C) in the photo above.

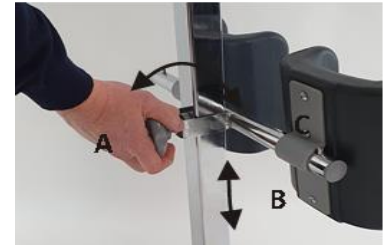### **TRANSPORTING THE TRANSFER DISC**

The transfer Disc can be transported to between locations using the two wheels are at the back of the device.

Firstly, the carer should place their foot on the back of the Base Plate and tilt the handle back so the device is supported on its two wheels.

The Handle should be tilted at least 45° so that the Base Plate does not touch the ground when the device is moved.

The device can now be pushed or pulled by the carer so it can be transported.

**CAUTION:** Under no circumstances should the device be transported with a patient. Failure to follow this instruction may result in damage and/or injury.

### **TRANSFERRING THE PATIENT**

The details below are only suggestions. Moving and Handling Instructions should only be given by your health professional. The patient must be able to bear weight and a risk assessment of the user's ability should be done prior to operation. Please read all the information herein before use.

The Transfert Disc helps a carer or attendant to assist a patient or user stand and/or transfer to another seat, bed or wheelchair by standing and turning.

1. Position the user so they are sat in a chair, wheelchair or on the edge of the bed.

If the user is sat on a wheelchair, ensure the brakes are applied on the wheelchair and the legrests removed or swung out prior to transfer.

2. The carer should position the device so that the user or carer can place the user's feet on the left and right foot symbols on the footplate.

3. The carer should position the height of the Lower Leg Support so that the tops of the pad are just below the knees of the user. (See section on Height Adjustment).

4. The carer should stand behind the device and support/ counterbalance it whilst the user uses the handle to support themselves whilst they stand on the footplate, and move from a sitting position to a standing position

*If necessary the carer can apply a foot to the base plate and hold the handle (as shown below) to prevent the base plate from rotating whilst the user is embarking on to the device.*

5. Once the user has securely reached a standing Position, they will be stood as shown in the photo (right) ; Then the carer should slowly turn the Patient Turner using the handle. The Patient Turner can turn 360°.

**Note: Do NOT tilt or lever the Transfert Disc. The whole footplate should remain in contact with the floor at all times.**

6. Once near the required seat, the user should have their back to the seat and be able to feel the edge of the seat near the back of their legs.

7. The carer should steady the device as previously described. The user can then use the handle Tubing as a support whilst lowering into the new seat. Once near the required seat,

## CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The product can be cleaned in an aqueous decontaminant machine at 60°C.
- Alternatively, the device can be chemically decontaminated in an autoclave machine at 60°C.
- Periodically check the following:
  1. Castors are securely fitted, the wheels run smoothly, and there is no sign of corrosion, damage or debris in the mechanisms.
  2. Check the Patient Turner revolves smoothly and easily. Ensure there is not excessive wear or play in the nylon discs and Base Plate.
  3. Check the Lower Leg Support bracket adjusts smoothly and securely, and look for signs of wear to the two polyurethane pads.
  4. Ensure the handle and all fasteners are secure.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## FEATURES TECHNICAL

General dimensions	W 47.5cm x D 40.5cm x H. 91-127 cm
Base dimensions	40.5 x 47.2 cm
Weight	8.1kg
Maximum authorised weight	200 kg (440 lbs)

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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