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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The RS2 static transfer handlebar is a medical device. It is a sit-to-stand patient transfer aid for indoor use on flat surfaces.

PRODUCT DESCRIPTION

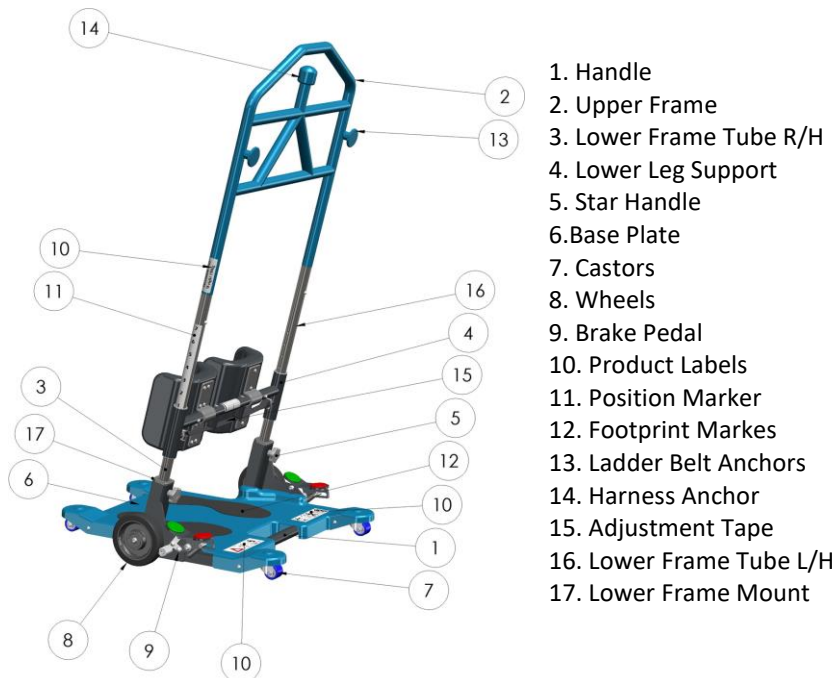SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.


- Don't exceed the maximum supported weight of 160 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Do NOT tilt or lever the product during use. The whole footplate should remain in contact with the floor at all times.
- Only use the product on a flat surface.
- The carer / attendant must always apply counter- pressure when the user is standing up, sitting down or transferring.
- The carer must always support and counterbalance the device when used by a user. If the patient uses the device to move to or from a wheelchair, the wheelchair brakes should always be applied and the legrests removed or swung out.
- Ensure both the carer and patient wear suitable footwear..
- Do not leave a patient unattended on the device.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING


1. Carefully remove all of the parts from the packaging

2. Put the base on a level surface, activate the locking brake to prevent the base from moving during assembly.


3. Place each lower frame tube into the lower frame mounts, making sure that the numbered side of the tube is facing outwards..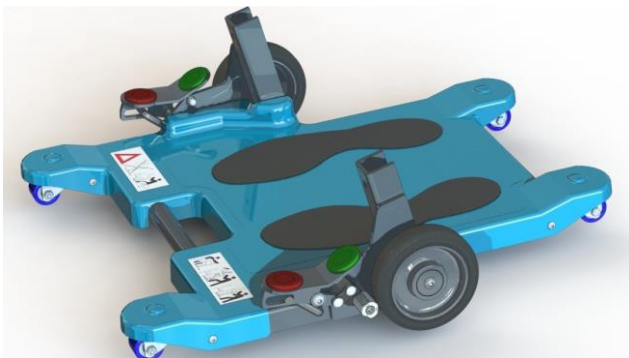




4. Position the lower frame and the lower frame into the baseplate mount.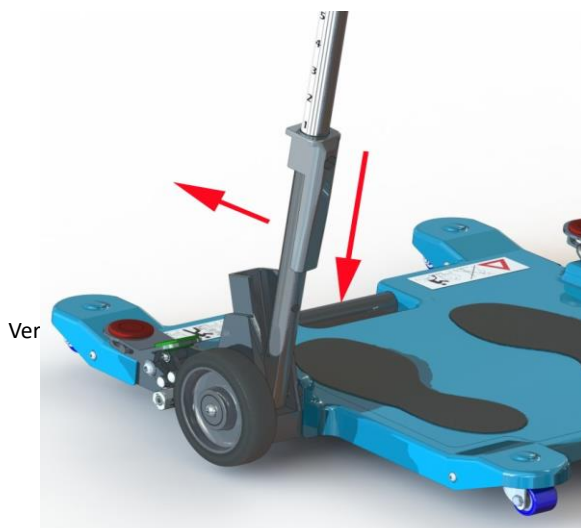


5. Place a nyloc nut into the hexagonal space in the front of each lower frame mount.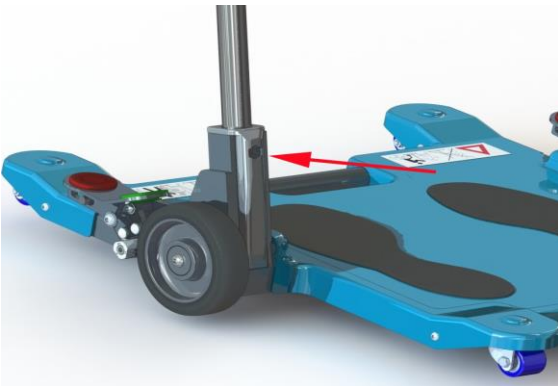

6. Firmly screw a star handle into the nyloc nut until the left/right lower frame is secure. Repeat for the opposite side. (See picture on next page)

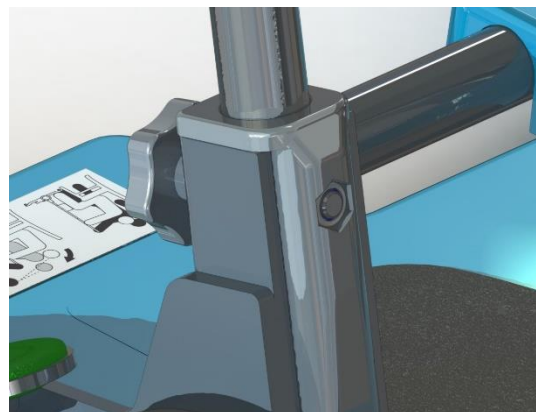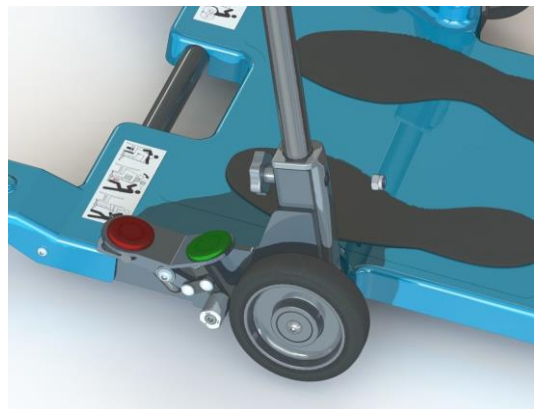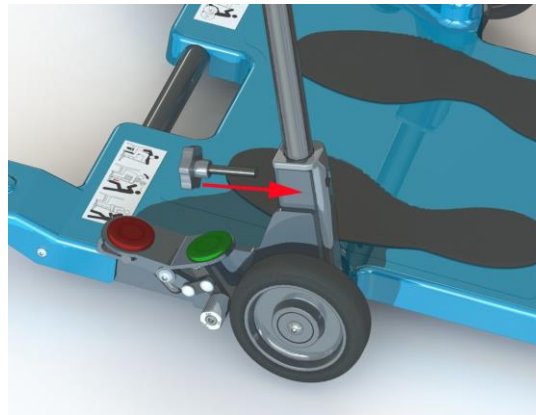
7. When both sides of the lower leg frame are securely fastened to the base, position the lower leg support assembly over the top of the tubes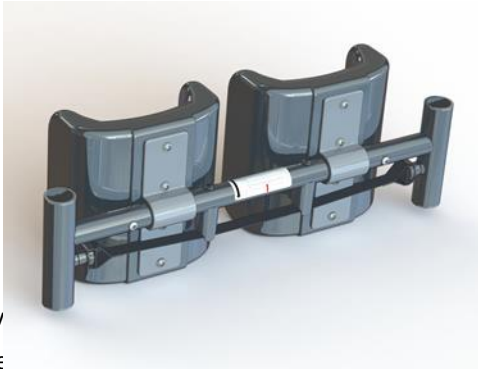

9. Move the level rest, pulling the adjustment tape upwards until the lower leg support reaches the correct position, as shown by the numbers on the outside of the tube.

8. Put the adjustment tape upwards to release the internal pins. This will allow the lower leg support to move down the tube until the pins reach the first positioning hole.

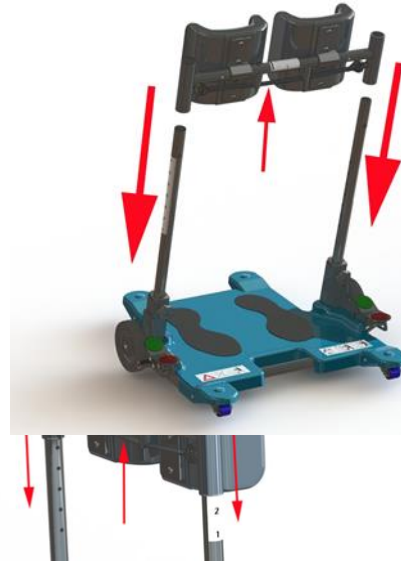10. Attach the upper frame to the lower frame ensuring the spring pin clicks into place to secure the upper and lower sections.

11. Unlock the brakes on the base plate. The Unit is now ready for use.

USE

1. Ensure that the environment is safe for the transfer to take place. When satisfied, place the patient transfer assistant in front of the service user.

2. Ensure that the service user's feet are placed on the base plate, using the foot markers as a guide. The legs should be in a comfortable position, cushioned by the leg supports. To facilitate a comfortable position, adjust the height of the lower leg supports so that the top is approximately two inches below the service user's knees.

3. Once you are sure that the patient transfer assistant is correctly positioned, lock both wheels by depressing a brake pedal. Both wheels will lock regardless of which side you are walking on.

NB: The wheels must always be locked when the user stands up and sits down!

4. If the user needs help to stand up using the ladder belt or harness, place it safely around the user.

a. Place the ladder sling behind the user, with the non-slip material against the body and the lower edge resting on the seat cushion.

b. If using the harness, place the lower section just below the user and the upper section against the lower back. Then secure with the clasp.

Your method of applying a counterweight should be reviewed and approved as part of your risk assessment.

5. If you are using the ladder belt to aid transfer, secure it to one side of the frame by placing the desired ladder section over the ladder belt anchor.

6. Ask the user to grasp the frame in a comfortable position. The user may hold any part of the handles. Service users who use only one arm or hand can hold onto the central handles.

7. If you are helping the user to stand using the ladder belt, prepare yourself in a stable and safe starting position. Place your nearest hand on top of the patient transfer assistant and hold the ladder belt handle with your other hand. Maintain good posture with a straight back.

NB: Use your risk assessment and / or consult your lead / back manual care advisor if you are unsure of the correct manual handling techniques.

8. When you and the service user are both ready, encourage them to stand. Gradually transfer your weight in unison, maintain tension on the ladder strap and help the service user to stand. Maintain eye contact and communicate clearly.

NB: When using the patient transfer aid, a counterweight must be applied at each stage of the sit-to-stand/stand-to-sit transfer. Your method of applying a counterbalance may vary, for example using the ladder strap, placing a foot on the footrest and reinforcing the frame, or supporting the service user in a traditional method. Your method of applying a counterweight will need to be

This patient transfer aid has been designed for use with one or more carers. However, it is important that a risk assessment is carried out prior to use. In each individual case, assess the risks and make notes. Assess the person's ability to use the product and, if necessary, use appropriate accessories or another product.

If using with two carers, follow the steps above. The second person can then assist the service user as required.

A safe transfer distance (from point A to point B) should be determined during your risk assessment.

We recommend that you do not transfer a person over a raised threshold or on uneven surfaces.

It is recommended that a ladder belt or harness is used for all transfers between point A and point B.

9. When the user is standing, attach the ladder strap to the strap anchor on the other side or to the centre upright when using the harness.

10. Release the brake pedal with one foot.

11. Check the environment and safely manoeuvre the patient transfer aid to point B. The transfer from point A to point B is best achieved when the service user is facing in the direction of travel.

12. Apply the brake

13. Check the environment again and prepare for a safe transfer, standing up to sit down.

14. With the maintenance user clinging to the frame, unhook the belt or harness from the ladder.

15. Ask the service user to sit down safely. When facilitating a transfer using the ladder belt, ensure that you are in a stable and secure starting position, with your near hand firmly gripping the frame. Encourage the service user to start sitting up with gradual assistance. It may be helpful for the carer to step forward in unison as the service user lowers themselves into a sitting position. Continue to provide the necessary level of support through the ladder belt during the transfer.

NB: Remember that a counterweight must be applied at each stage of the transfer. Your method of applying counterbalance may vary, for example using the ladder belt, placing a foot on the footrest and reinforcing the frame, or supporting the

service user in a traditional method. Your method of applying a counterweight should be reviewed and approved during your risk assessment.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- Before each use, check :
 - Shin guard height adjustment
 - Brakes and castors are in good working order
 - General condition of foam parts
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 6 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL CHARACTERISTICS

General dimensions	W 63 cm x D 60.7 cm x H 118.5 cm
Base dimensions	63 cm x 60.7 cm
Shin pad dimensions	29.2 cm x 46.7 cm
Weight	14.9kg
Maximum authorised weight	160kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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