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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

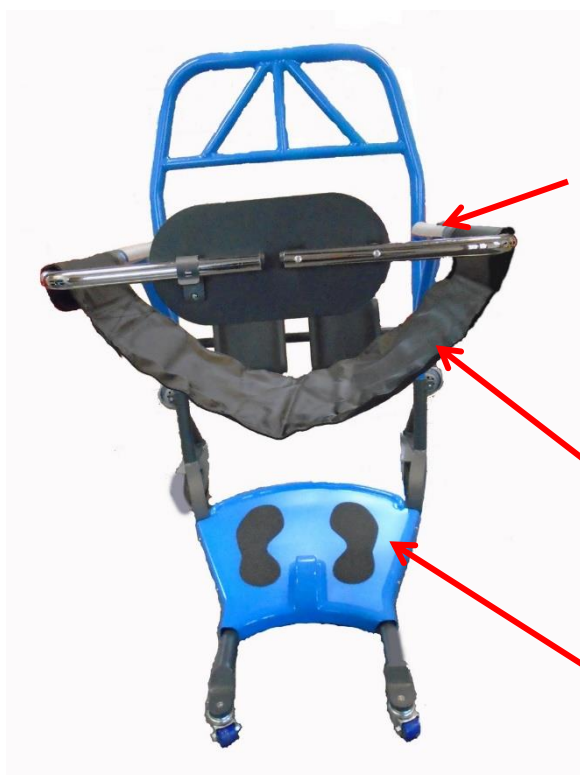
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The RS3 is designed to provide a quick and easy transfer from a seat to any other location around the home, school, care home or hospital setting for clients that are able to bear weight for a short period of time. The RS3 transfer device typically only requires one carer to use.

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Seat Pad hooks to be attached with the hook facing outwards.

Seat Pad

Footplate

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 150 kg.
- Only use the appliance in accordance with the above instructions. Please ensure that all users are familiar with these instructions and with the operation of the appliance before using the product.
- Do not use the device if any part of it is damaged.
- Do not use the RS3 unit to transfer users outdoors or in vehicles.
- Ensure that the product is in good working order.
- Visually inspect the product
- Do not tilt or pry with the Transfer Disc during use. The footrest assembly must remain in contact with the ground at all times.
- Only use the product on a flat surface.
- The carer should always apply counter pressure when the user is standing, sitting or transferring.
- The healthcare professional should always support and counterbalance the device when the patient is on it. If the patient is using the device to move to or from a wheelchair, the wheelchair brakes should always be applied and the leg rests removed or swung outwards.
- Ensure that the healthcare professional and the patient wear appropriate footwear.
- Do not leave a patient unattended on the equipment.
- If any parts are damaged, broken, worn or missing, contact your dealer.
- Only use the machine on flat surfaces.
- Ensure that the brakes on both rear castors are in the locked position before transferring a patient/user onto or off the RS3 unit.
- DO NOT lift the handle section higher than the top hole for the spring pin lock.
- Do not attempt to lift the handle while the unit is in use with a patient.

MOUNTING

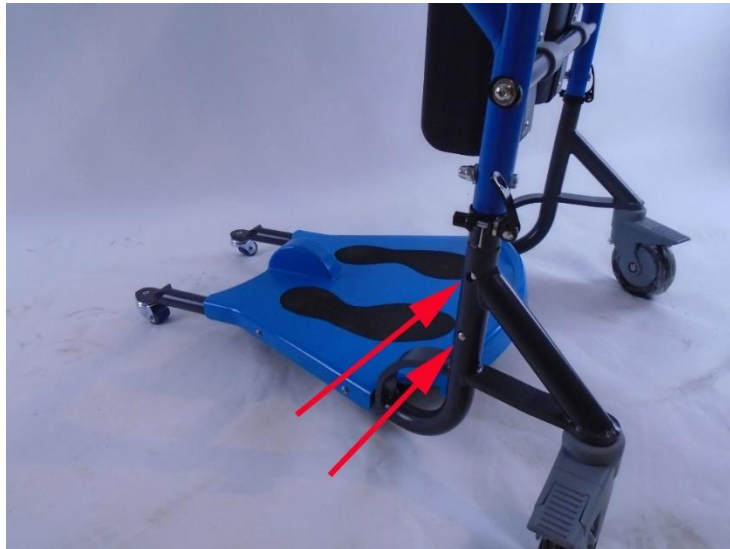
HEIGHT ADJUSTMENT

The height of the RS3 Transfer device should be adjusted to suit each user. The user's kneecaps **must** be in the centre of the kneepad(s).

1. Release both clamping levers (See Photo below).

2. Raise or lower the upper half of the unit according to the height required. There are two stop points with spring pins setting the lowest or highest points. The height of the upper half can be set anywhere between the lowest & the highest point. (See photo below).


If the spring pins are engaged into the holes provided, press the spring pins in on each side and push down or pull up the handle section to the correct height. (This may require two persons, one to press the pins and one to move the handle)

Note: DO NOT set the RS3 handle section higher than the maximum position indicated by the top hole position in the base section where the spring pin locates.

3. Once the correct height has been reached, lock the clamping levers to ensure that the handle section is securely tightened in place. If required, the clamping lever handles can be rotated clockwise to increase the tension on the levers.

SEPARATING THE HANDLE SECTION FROM THE BASE

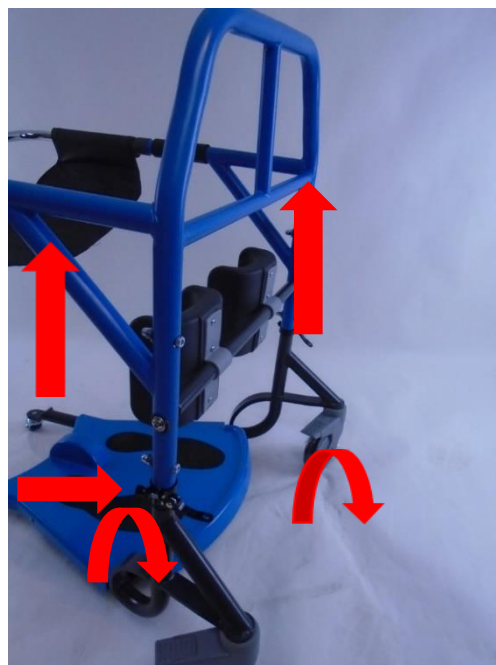The RS3 Handle section can be removed from the base to allow for a more compact method of storage or ease of transportation to a separate location. The method of separating the two parts is as follows :

1. Release both clamping levers.

2. If the spring pins are located in the positioning holes, push the pins in and pull the handle section upwards until it is released from the lower section. (This may require pressing the pins in twice).
3. Pull the handle section upwards until it is released from the lower section. (See photo below)

USE

Daily checklist to be checked before use

- Clamping Levers are locked tightly
- Castor Brakes function correctly
- Seat is free from cuts, damage or any loose stitches.
- Seat correctly fitted (See image below)
- Seat hooks fit securely to arms.
- Backrest pad secure and undamaged
- Knee Pad(s) secure.
- Footplate secure and clean.

1. Start with the rotating arms turned outwards and the seat hooked onto just one of the arms to allow the greatest amount of room for the user to access the device. (See Photo below).

2. Move the RS3 device as far under the seat the user is positioned in as possible. Ask the user to place their feet onto the footplate where the black footprints are located. (See photo below).

3. Apply brakes on the rear castors (See arrows below). With the user holding on to either the side or front bars, support the user as they move into a standing position on the device footplate. (See photos below).

4. While the user is standing & continuing to keep a firm hold onto the bars, hook the seat across onto the opposite arm and ensure it is securely fitted. Rotate the arms into a closed position ensuring that the back pad is securely locked to provide a secure back rest for the user. Once the user is in a comfortable position, release the brakes on the rear castors.

5. Move the unit with user safely positioned, to the new desired location, pushing from either the front or the rear.

Note: It is easier to cross thresholds if the large castors are pushed over the threshold first.

6. In order to transfer the user from the device to a new position when the new location has been reached, follow the above instructions in reverse order.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- In order to prevent cross infection, the RS3 unit should be thoroughly cleaned between use by different patients / users.
- The seat is removable and can be cleaned in a washing machine at a temperature of 40 degrees Celsius using a biological detergent.
- The frame should be cleaned using a diluted disinfectant or a general purpose cleaning wipe that **DOES NOT** contain chlorine.
- After cleaning, please ensure all parts of the device are completely dry before re-use.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

General dimensions	W 70 cm x D 82.9 cm x H 94 to 101 cm
Seat height	51.5 to 58.5 cm
Shin guard dimensions	29.2 to 46.7 cm
Weight	25,7kg
Maximum authorised weight	150 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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