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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary. User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result. Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

## PRODUCT DESTINATION

The Delta Evolution rollator is a walking aid for people with reduced mobility. This product is intended for use indoors and outdoors and can be folded up for easy storage or transport.

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 114 Kg.
- Do not exceed the maximum load of 5 kg in the basket. Loads must only be transported in the basket. Do not hang heavy objects on the rollator as this could affect its stability.
- Do not use the rollator on stairs or escalators.
- Use only accessories and parts supplied by the manufacturer.
- Make sure that the knobs are tightened to the same level and the handles are securely fastened.
- The rollator is a walking aid. **It is not a wheelchair.** It is not designed to carry a person sitting on it. **Such use may damage the wheels and the structure of the rollator itself.** The rollator could be weakened and no longer meet its mechanical characteristics in future use. Only use the rollator as a walking support.
- Do not use the rollator to walk backwards.
- Always sit on the seat when the rollator is stationary and the brakes are locked. Serious injury to the user and/or damage to the rollator frame and/or wheels can result from inappropriate use.
- Make sure all parts (wheels, cables, handles, brakes) are secure and in good working order.
- Do not make any adjustments to the rollator while it is in use. All wheels must always be in contact with the ground.
- Check that the brakes function properly before use.
- Make sure that the parking brakes are securely attached.
- When the rollator is in the stationary position, the brakes must be locked. Do not attempt to move the rollator if the brakes are locked.
- Be sure to put your hands on each handle so that the rollator can be in the parallel force position. Make sure the seat is clipped flat before use.
- Make sure the seat tube caps are present.
- During use, lock the parking brakes for all actions.
- Do not attempt to push the rollator while someone is sitting in the seat.
- Do not use the seat to carry objects.
- Do not use the seat without the backrest.
- Do not sit in the seat when the rollator is on a slope.
- The brakes must be in the locked position before sitting in the seat.
- Do not attempt to reach objects while sitting down by leaning forward. Do not lean backwards as this can cause the rollator to tip over.
- Be careful when walking up pavements or on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the rollator on a wet, frozen, or snow-covered slopes. If any parts are damaged, broken or worn, contact the dealer.

## ASSEMBLING & USE

Check all parts carefully before assembling the product. Do not assemble the product if any parts are missing or are damaged.

### ASSEMBLING :

1. Open the 3 wheels rollator into its operating position.
2. Insert the handlebars into the frame tubes and set to the correct height as follows:
  - Stand upright behind the rollator with your shoulders relaxed and your arms by your side, looking straight ahead.
  - When you are in this position, the handgrips should be adjusted to height that brings them level with your wrists.
  - Your arms should now be able to be bent at an angle of 20° to 30° when using the rollator.
  - Check that the holes in both the Handlebars and the frame are correctly aligned.
  - Place the supplied bolt through the aligned holes in the handlebar and frame. The holes in the frame are hexagonal in shape so that they accommodate the head of the bolt.
  - Place the washer over the threaded section of bolt that is protruding from the outer side of the frame and then screw the hand wheel onto the threaded section. Hand-tighten firmly, but do not overtighten.
  - Follow the same procedure for the second handlebar.
3. Check the brakes grip the wheels thoroughly before using the rollator.  
NOTE: The brakes are factory set and if assembled correctly, should not need adjusting.  
To check: Apply and release the parking brake on each side 10 times. (See diagram 2b) The brake should grip the wheel and prevent any movement when they are applied. When released, the gap between the brake and the wheel should be between 1mm and 3mm. If either of the above conditions are not correct, adjust the brakes as detailed in the maintenance section and re-check the brakes.
4. Fit the bag to the frame using the poppers supplied.

### MOUNTING :

1. **Opening:** Fold out the rollator and push down the seat fully. (See Diagram 1a) – It is very important that the cross bar be pushed down fully to prevent the Walker from closing when it is being used.
2. **Folding:** Pull up the Seat Strap. (See Diagram 1b) The rollator will then close inwards to make storing the unit as easy as possible. Take care to keep fingers outside of the frame when folding the rollator. The bag should be empty when folding the rollator.
3. **Walking:** This product is designed to be a walking aid, creating a natural gait as the user walks securely and safely, with their primary weight distribution between the handlebars and with the rollator in front of the user.
4. **Braking:** The locking brakes are intended to assist the user to control the rollator. To apply the brakes when in motion, pull up both of the top handles. (See Diagram 2a) To apply the parking brakes, push down both of the loop handles until they lock onto the wheels and lock into position. A distinct audible click will be heard to show that the brakes are in the correct position. (See diagram 2b)

It is important that both Parking brakes are applied when the unit is not being pushed. Lift up the loop handles when the user is ready to start moving again.

**Diagram 1a**  
Push down the seat fully  
Lock the Walker in the open position.

**Diagramme 1b**  
Pull the up the seat strap to fold the  
Walker. Take care to keep fingers  
Away from the frame when folding.

**Diagram 2a**

Pull loop handles up while in motion to stop the Walker from moving while user is walking..

**Diagram 2b**

Push loop handles fully down to lock the Walker in the Parking position. Use this position for parking the Walker when not in use or for implementing the bag.

## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts.
- Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If the brakes do not completely block the wheels, the brakes must be adjusted.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

## CHANGED OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Observe the conditions for storing and stowing your rollators :
  - In a dry and temperate place
  - Protect your rollator by packaging it from dust, corrosion (e.g.abrasive elements, sand, sea water, salty air).
  - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
  - All components must be stored without any load (do not place any objects that

## TECHNICAL SPECIFICATIONS

Overall dimensions	Length. 63.5 cm x Width 58 cm x Height 82 – 99 cm
Seat dimensions	Length 44 cm x Width 28 cm x Height 53 cm
Weight	6 kg
Maximum basket load	5 kg
Maximum load	114 kg

## PERIOD OF USE

The expected life span of this product is 3 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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