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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a mobility aid.

Distributor: This manual must be given to users of this product.

PRODUCT DESTINATION

Folding lift walking frames and articulated walking frames are walking aids designed for people with reduced mobility. These walking frames are designed for both indoor and outdoor use. Folding frames fold up for easy storage and transportation.

PARTS DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 120 Kg.

Do not use the product on stairs or escalators.

Only use accessories and parts supplied by the manufacturer.

When adjusting the height of the leg extension, make sure that each is set into the same height adjustment hole and that the walker is level. Snap buttons must fully protrude through the adjustment hole of each leg extension to ensure a

positive lock. Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance.

The product is a walking aid. It is not a wheelchair. It is not designed to carry a person sitting on it. Such use may damage the wheels and the structure of the product itself. The product could be weakened and no longer meet its mechanical characteristics in future use. Only use the product as a walking support.

Do not use the product to walk backwards.

Do not make any adjustments to the product while it is in use.

Check that the brakes work properly before use.

During use, lock the parking brakes for all actions.

All wheels must always be in contact with the ground.

When the product is in the stationary position, the brakes must be locked. Do not attempt to move the product if the brakes are locked.

Be sure to put your hands on each handle so that the product can be in the parallel force position.

Use caution when climbing pavements or walking on sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

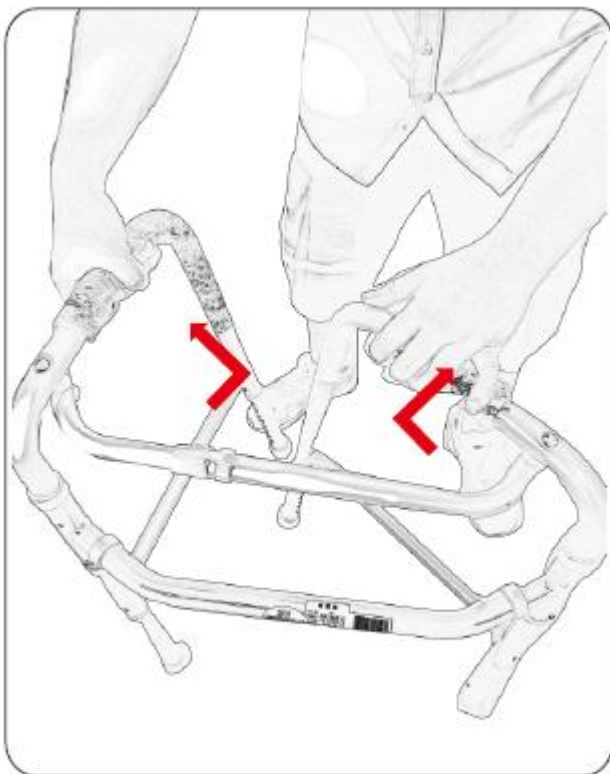
Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken, worn or missing, contact the dealer.

Beware of the risk of pinching when folding.

MOUNTING & USE

1 Setting the walker


- Take out the walker from the carton.
- Swing both side frames out away from the front cross brace until an audible "click" is heard.

2 Folding the walker (826085)

- Find the plastic piece on the top of the front bar.
- Press down on the locking mechanism and fold the side frames in toward the front the front cross brace.

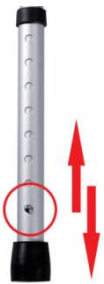3 Adjusting Leg Extensions

- Press the snap button, extend or retract the leg to raise or lower the height.
- Adjust the leg extension to the desired height and allow the snap button to snap into the height adjustment hole. (An audible "click" will be heard)
- Make sure that the walker is level.

WARNING:

The walker **MUST** be level before use. Pinch points exist when installing and adjusted leg extensions. Make sure hands and body are clear of all pinch points.

2.4 Fixing / Inactive adjustment

Fixed

Interactive

- Press the folding button and fold the side frames slightly.
- Fixed movement mode : press the snap button into upper hole (An audible "click" will be heard). It to be moved forward with lifting walker.

- Interactive movement mode: press the snap button into lower hole (An audible "click" will be heard). It to be moved forward one side at a time, in step with the user.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Mon- thly	Yearly
Control of the pin in a secure position during folding	X			
Checking the condition of feet and handles		X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in direct sunlight (e.g. behind a window or near a radiator) or near a cold source.

Keep away from open flames and sparks.

Respect the conditions for storing your product.

- In a dry and temperate place
- Protect your walking frame by packing it away from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air)
- Store all removed parts together, in the same place (or mark them if necessary) to avoid mixing them with other products during reassembly.
- All components should be stored without any load (do not place heavy objects on the product parts, do not jam anything).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

FEATURES TECHNICAL

Overall Dimensions	826085 - Width 56 x Depth 47 x Height 71 to 89 cm 826084 - W 56.5 x D 47 x H 74 to 92 cm
Width between armrests	44 .5 cm
Weight	826085 – 2,3 kg 826084 – 2,42 kg
Maximum load	120 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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