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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise you on the most appropriate use for you. The distributor of this product is responsible for its installation.

Distributor: this leaflet must be given to the users of this product.

SYMBOL APPEARING ON THE LABEL OR IN THE INSTRUCTION MANUAL

	Manufacturer's adress		Invitation to consult the user manual
	Date of production		Follow the safety instructions
	Batch number		CE Conformity
	Product referance		Does not tolerate dampness
	Maximum user weight		Medical Device

PRODUCT DESCRIPTION

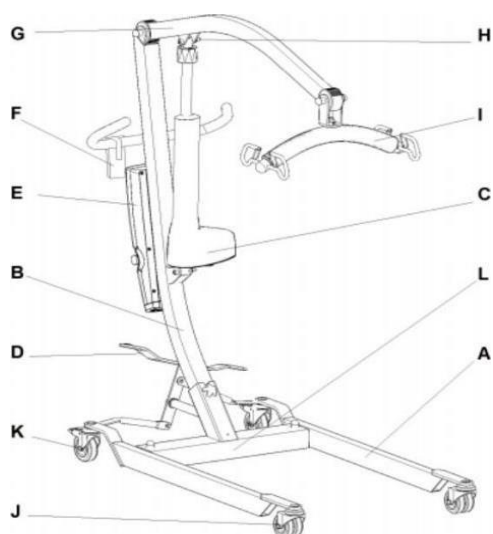
Majorque dosmetic patient lifting device are technical home support devices for people with reduced mobility. They are designed to carry only one patient at a time.

The lifting device is designed to use indoors only on hard and even surfaces.

They are designed to carry only one patient at a time. For your safety, during lifting and transport, the patient must be under the supervision of a qualified caregiver. Affected patients are :

- Patients with paraplegia - including the lower limbs.
- Patients with paralysis.
- Patients with limb loss.
- Patients with cardiovascular failure.
- Patients with reduced joint mobility and muscle weakness.
- Elderly people requiring specialized home or professional care.

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- A - base
- B - frame
- C - electric actuator
- D - width adjusting pedal
- E - control box with battery,
- F - remote control
- G - lifting arm
- H - adjusting screw
- I - hanger
- J - front castor without brake
- K - rear castor with brake
- L - base

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 150 Kg.

Do not use a device to transport patients over long distances.

Choose the sling recommended by the doctor for the patient. When moving the patient while he is suspended from a carrier attached to the lift, do not move the base wheels through items such as raised carpet frames, door sills, or any uneven surfaces that may cause the lift to shake. If any undesirable effects occur, the patient's transport should be stopped immediately.

Check that the product contains all the parts

Make sure that all components do not have visible mechanical damage. In case of doubt, contact the manufacturer.

Do not use the device until all components are properly connected.

Please do not place the hoist on an inclined surface as it may fall down

Do not attempt to exceed any steps, curbs, or other obstacles that could cause injury or injury when the product falls over.

Do not attempt to repair the battery yourself

Using the lifting device on an unstable surface (covered with ice, wet, greasy, sandy) is unacceptable- it causes risk of falling,

You can only use the lifting device on surfaces where all four wheels are touching the ground.

Do not leave the lifting device with the user close to open sources of fire.

Be aware that the surface temperature of the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of lifting device. It may cause burns or frostbite parts of the body,

In order to transfer lifting device, grab the frame. Never transfer the lifting device by grabbing movable elements or control unit.

Lifting device must be easy to fold and to unfold. In case of problems with folding, you should re-read the instruction or contact the manufacturer.

Brakes cannot slow down the lifting device during ride, you should use them only to prevent the unintended movements.

Oil, grease, water or other chemicals, may cause that brakes may have reduced effectiveness

It is extremely important to keep all lifting devices components clean in order to maintain the product parameters.

Do not use the lifting device after discovering any visible or detectable damages or when the wear of some parts may cause an accident. All the repairs need to be performed by skilled personnel and during the warranty period only by the manufacturer's service.

During transport (e.g. in the trunk), do not burden lifting device elements, it may weaken or damage its structure.

Keep out of the reach of children under five years of age.
 Should not insert fingers between the lifting device components - risk of injury or entrapment.
 Keep extreme caution while unfolding / folding the lifting device. Lack of attention may result in finger entrapment.
 Always maneuvers the lifting device with the handle provided.
 Do not use a sling that is not recommended for the lift.
 Only trained and qualified caregivers should transfer a person.
 Lifting device is designed for transferring patients only. Do not use the lifting device for any other purpose.
 Before each patient lifting, make sure that base legs were blocked in maximum wide position.
 While moving the patient, should bypass obstacles at safe distance, which might cause the lifting device to tip over.
 Before each lifting device usage, make sure that slings are in good condition. Usage slings showing signs of damage is forbidden.
 Before using lifting device, should charge the battery for about 6-8 hours. Make sure, the red warning lights is off.
 If any parts are damaged, broken, worn or missing, contact the dealer.

MOUNTING

I. Assembly guide

1. Install the column

Fig. 2. Scheme of mounting the lifting column.

Remove the device and spare parts from the package. Loosen the jam nut and bolt, then raise the frame and align the holes in parallel. Tighten the screws and tighten the nut properly (make sure there is no movement at the frame).

ATTENTION: When mounting the lifting column, avoid standing under it, failure to do so may result in accidental fall Of the column and risk of injury.

2. Hoist Arm and Powered Actuator

Fig. 3. Installation of lift arm

Align the bracket with the mast mounting hole, screw in the bolt and washer, install the bottom of the actuator and column in place, and finally install the electric actuator and bracket, tighten all nuts.

3. Install the Controller

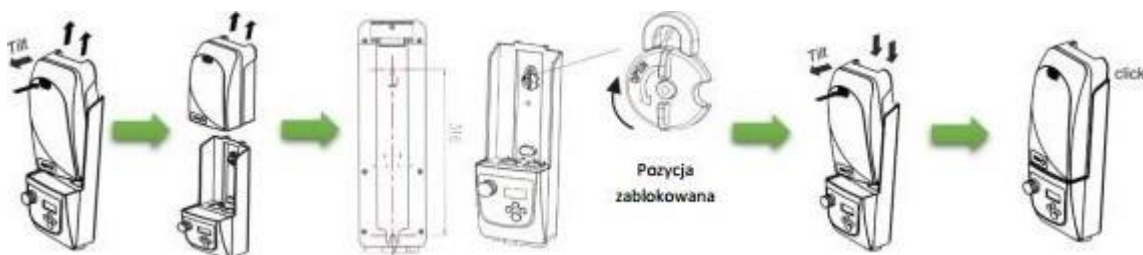
Fig. 4. Controller installation scheme

Releasing: Press the release button and pull the battery out in the direction. Installation: Place the battery in place and press the switch in the direction shown in the scheme (Fig. 4). Ensure that the battery cannot be separated without pressing the switch.

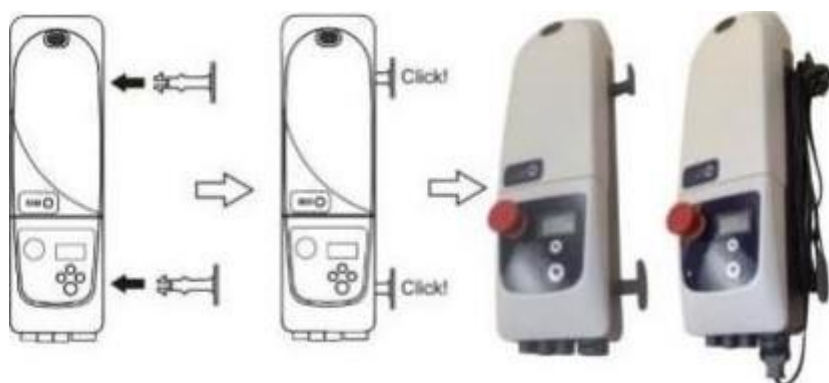
Fig. 5. Installation scheme Of hooks for the power cord.

II. Operation guide

Electrical controller (remote control):

1. Emergency Stop Button (Emergency Stop)

The emergency stop button interrupts and turns the power on, causing the engine to stop immediately in an emergency.

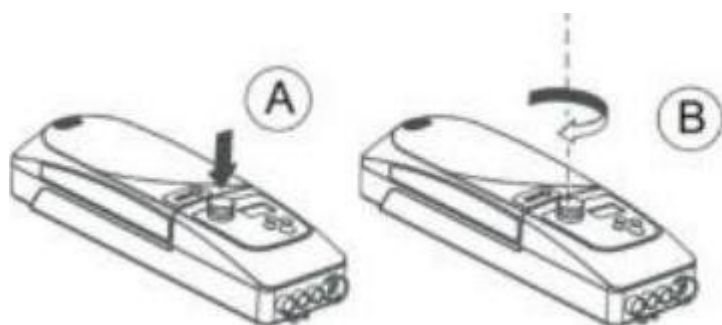
Fig. 6. Emergency Stop Button operation scheme, positions A and B.

Pressing the Emergency Stop button (red button) (Fig. 6A) will stop the engine. To restore the engine to normal operation, turn the button (Fig. 6B) clockwise to unlock the device. Lifting and lowering the boom are simple and is done using the above pilot / controller. Suspend the remote control after using or adjusting the boom height in a safe and easily accessible place near the driving handle.

NOTE: When the device is not used for more than 3 days, press the Emergency Stop Button to maintain longer battery life.

2. Electrical failure - 'up' and 'down' buttons:

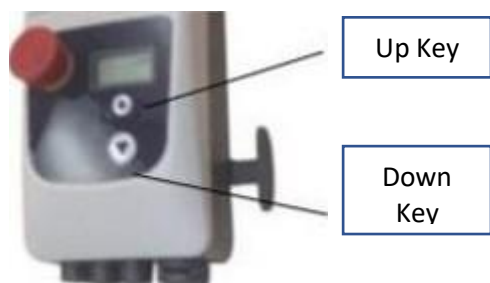
Fig. 7. Button location, up and down

Use the 'up' or 'down' arrow keys (Fig. 7) to bring the motor to its default position if the controller fails or there is no controller handle.

3. Controller support:

Use the "up" and "down" buttons on the controller handle to control the lift and lowering the lifting arm, after releasing the buttons it will stop moving. If the remote control does not work properly, press the red Emergency Stop Button.

4. LCD display

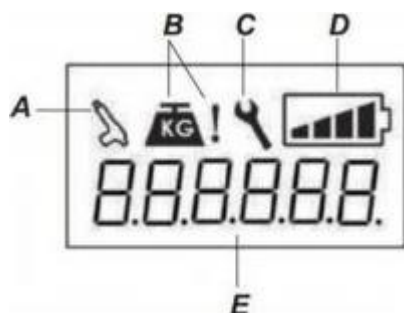
Fig. 8. Scheme of controller information displayed, designations:

A — Over duty on main lifting actuator,	D — Battery lever bars,
B— Overload on actuator/s	E— Error code display.
C— Remind to replace main actuator,	

The display can show tips and information on transport lifting device malfunctions. Battery charge indicator - from the left: 100%, 75%, 50% and 25-0%.

Display	Information	Disposal
	Low battery	charge the battery immediately
	The electric push rod is running under overload (continuous operation for 4 minutes). Stop running and wait for it to rest fully before using it	The electric push rod is running under overload (continuous operation for 4 minutes). Stop running and wait for it to rest fully before using it
	Electric push rod overload	Remove part of the load
	Electric push rod transmission part overload	Move backward for 2 seconds
	Electric control box failure	Contact supplier for repair or replacement

5. Control device connection :

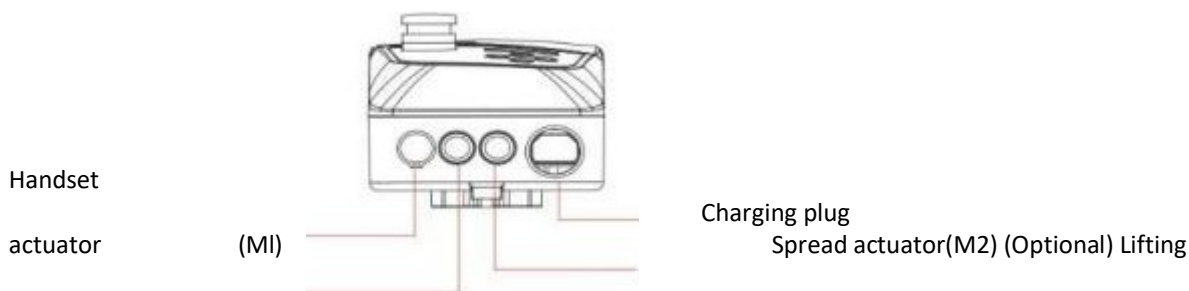
Fig. 9. Port diagram of the electrical box controller.

The device has 4 ports (Fig. 9), from left to right: Handset, lifting actuator (MI), Spread actuator (Optional), Charging plug. Installation: Insert 4 cables into four different ports one after the other, respectively.

6. Base width adjustment

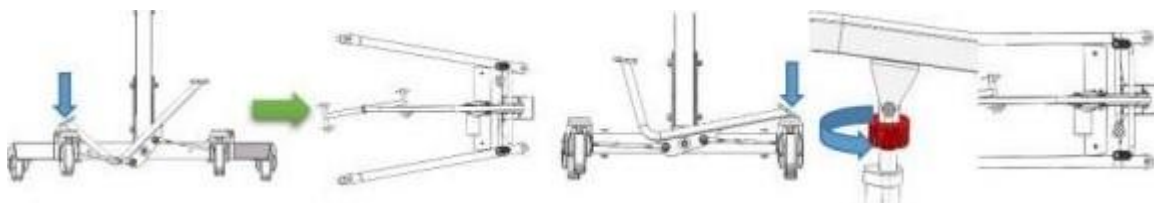
Fig. 10. Scheme width adjustment of the base.

The width of the lift base is manually adjusted with a foot pedal. Press the left pedal to extend the base. Press the right pedal to make the base narrower. It is possible to manually lock the base width adjustment by turning the red sleeve (Fig. 10) counter clockwise.

Warning! Locking the wheel during hoist movement will increase the risk of falling down the hoist.

7. Rear wheel opening and braking

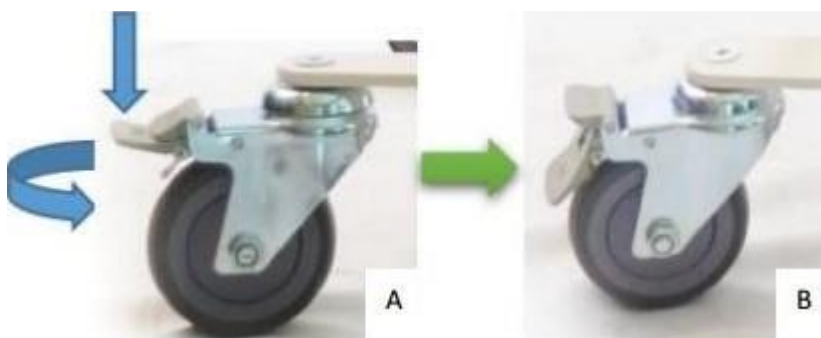
Fig. 11. Rear attachment with rear lifting brake

The rear wheels can be locked to prevent movement. Lock or unlock the rear wheels with your feet. Use your foot to control the brake position. Press the longitudinal plastic element with the foot to block the wheel (Fig. 10A), then press the shorter plastic element to resume movement (Fig. 10B).

WARNING! Locking the rear wheels of the jack increases the risk of the machine tipping over when lifting and lowering. Always keep the support perpendicular to the patient's center Of gravity when working.

Charging (power supply)

When the battery has only one line left, the control box emits 4 beeps and remind you to immediately charge. The engine will not run without charging.

Battery charging procedure

When you hear a beep, turn Off the emergency stop button and connect the control box. When the battery is fully charged, the battery charge status shows 4 dashes (100%) and the LED on the control panel does not flash.

USE

1. Lifting the patient

- First, set the ride base in the maximum enlarged position. To do this, press the pedal Of the head adjusting the width of the transport lifting device.
- For easy and comfortable lifting Of the patient should be prepared a transporter lifting device, a sling with a leg separator, a sling with head stabilization and a wheelchair.
- The patient should lie in the middle of the bed, then turn he/she to the side, in the opposite direction to the caregiver.
- The sling should be placed under the patient. Examples Of slings are shown in the figures 12 & 13.
- Turn the patient towards the caregiver and place he/she in the middle Of the sling so that the back loops are in the upper back of the patient and the thigh loops be placed within the patient's thighs.
- Hanging loops should be hung on a four-point cradle.
- Make sure hanging loops are tight and make sure the patient is in a comfortable position.
- Lock the rear wheel brakes and lift the patient by pressing the corresponding button on the remote control.
- Should lift the patient until his feet are not touching the bed or the floor.
- Then unlock the rear wheel brake and carry the patient over the wheelchair.
- The brakes should be locked both in the transport lifting device and in the wheelchair.
- The next step is to press the "down" button on the remote control to gradually lower the patient.
- When descending, help the patient to take a correct and a comfortable sitting position.
- After setting down the patient in a wheelchair, remove the sling loops from the fourpoint cradle.
- If using a sling with a leg separator, make sure that the patient is properly dressed before moving the lift towards the movable toilet chair.
- Make sure that the hanging loops are tight and that the patient is in a comfortable position. The patient can be placed on the sling in a sitting position, e.g. when transported from a wheelchair or bed as follows.
- Place the sling around the back of the patient.
- Donning is easier if the patient bends slightly forward.
- Guide the sling carefully at the patient up on the height of the tailbone.
- Move the thigh belts forward at patient's thighs until they are on both sides at the same height.
- Be careful not to bend the material.
- Wear the seat belts under the patient's slightly parted legs.
- Donning is easier if the patient slightly raises thighs.
- In order for the patient to sit comfortably on a wheelchair, it should be planted as deep as possible.
- During the process, lowering the patient, lightly press the patient's knee so that his/her back is closest to the backrest of the wheelchair.
- Loops have different colors to make it easier to put on slings at the same height.


Fig. 12. Sling with a leg separator

Fig. 13. Sling with head stabilization

2. Foldable and portable frame:

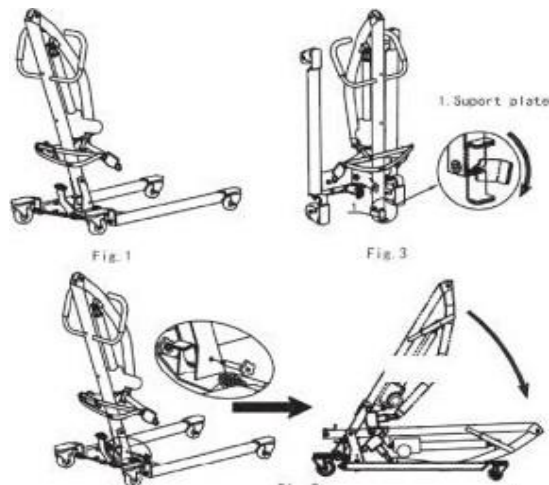- Lower the control unit and then fully attach the lift frame.
- Remove the screw before folding the frame.
- Release and support the plate before raising the frame
- The scheme of folding the frame is shown in figure 14.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

Care

Clean the seats with a damp cloth. Stains can be removed with a cloth or a soft brush. Wash with a damp cloth (in clean water) and leave to dry. Before washing, follow the instructions on the headgear label. Do not use solvents, toilet cleaners, sharp brushes and hard objects, corrosive cleaners. The straps can be machine washed at 30 - C. Do not use chlorine bleach. Do not tumble dry. Do not iron.

Maintenance

Routine inspection listed below shall be performed every day for the sake of safety.

1. Check if there is any damage or wear by visual inspection.
2. Make sure screws and nuts on lifting arm are secured.
3. Check if hook on lifting arm works properly.
4. Controller, actuator and base width adjustment verification.
5. Emergency descend function verification (both manually and electrically)

If needed, to clean the patient lift with warm water or alcohol. Make sure there is no dust and hair on the wheels.

Attention: to clean the patient lift with chemicals with Phenol and chlorine is prohibited, which may cause damage on plastic powder or plastic material.

Inspection

Make a full inspection at least once a year. Pay close attention on those worn part such as connection and screw bolt.

Troubleshooting

Issue description	Troubleshooting
-------------------	-----------------

Patient lift cannot work (ascend and descend)	<ol style="list-style-type: none"> 1. Check if the emergency stop button is pressed down. 2. Verify the controller and cables from actuator is properly connected. 3. Check the battery level from display screen on the control box. 4. Verify if there is any problem or damage on battery contact panel. 5. Ask for after-sale service if patient lift still cannot work.
Charger cannot charge the battery	<ol style="list-style-type: none"> 1. Check if the charger fully plugs into socket. 2. Verify if there is any problem or damage on battery contact panel. 3. Ask for after-sale service if patient lift still cannot work.
Patient lift stops when ascending	<ol style="list-style-type: none"> 1. Check if the emergency stop button is pressed down. 2. Employ the emergency descend device. 3. Ask for after-sale service if patient lift still cannot work.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

Storing the lifting device in areas with very high humidity (baths, laundry, etc.) or in the open air, also the lack of hygiene causes rapid loss of operating parameters and aesthetic of the product.

Press the emergency stop button for transportation or not using the patient life for a certain Of time. Tem-perature for storage Of patient life shall be higher than 0°C and relative moistures60%.

Inspection:

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Technical parameter	Parameter value
Frame	Steel, powder coating
Chassis length	950 mm
Chassis width	520-580 mm
Lifting height	1230 mm
Handle height	780-1870 mm
Castor height	115 mm
Motor	DC 24V 8000N
Battery	DC 24V 5AH, interchangeable battery Approx. 60 lifts per full charge
Charger	24V-1A
Charging time	5 hours per time
Front castor	3" solid tires
Rear castor	4" solid tires with locking
Push handle	Push handle with rounded corner
Max. user weight	150 kg
Addons	Standard sling in Oxford fabric
	Standard Mode / Stand Aid Mode Interchangeable
G/N.W	46/40 KGS

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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