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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Evolution 2 Wheeled Rollators are walking aids for people with reduced mobility. These products are intended for use indoors and outdoors and can be folded up for easy storage or transport.

PRODUCT DESCRIPTION

1. Handgrip
2. Interlock button for handgrips
3. Backrest
4. Seat
5. Pedal for open / fold
6. Rear slide
7. Front wheel

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 136 Kg.

Make sure that all parts are properly secured and in good working order.

Regularly check the screws and fasteners of the rollator and retighten them if necessary.

Make sure that the handle locking latch is properly engaged.

Make sure that the rollator is fully opened, seat is attached in place and the pedal is in the down position.

Be sure that both handgrips are fully tightened and are at equal heights. The walker for stability before applying full weight to the handgrips.

Do not use seat when unit is on an incline or uneven ground.

Replace wheels when significantly worn or damaged.

DO NOT MOVE WALKER WHILE SEATED. This is a walking aid only and is not to be used as a transportation device.

Do not use the rollator on stairs or escalators.

Only use the accessories and parts supplied by the manufacturer.

Do not use the rollator for walking backwards.

Do not make any adjustments to the rollator while it is in use.

All wheels must always be in contact with the ground.

Use caution when climbing pavements or walking on inclined, uneven, or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.

Do not use the walker on a wet, icy, or snowy slope.

If any parts are damaged, broken or worn, contact the distributor.

MOUNTING & USE

To unfold the walker, push the pedal down to the end and then flip over the seat. (see figure 2)

Make sure the seat is attached in place before using the walker.

To fold the walker, likewise stated as above but in reverse order (see figure 2)

Handgrip Height Adjustment

Lift the catch and push or pull the handgrip to the correct height. Press down the catch, the handgrip will now be locked at the correct height. (see figure 4)

Position the walker on an even and solid surface. The walker must stand and must not slip. Sit straight up and do not lean backwards. **DO NOT MOVE WALKER WHILE SEATED.**

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment	X			
Structural weld at wheels		X		
Cleanliness				X
Screws and fasteners		X		
Pedal Operation	X			
Stability of frame and backrest			X	
Condition of seat			X	
Condition of the wheels and pads		X		

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the rollator, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your rollator for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your rollators.

- In a dry and temperate place
- Protect your rollator by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the rollator elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 60 cm x Depth 60 cm x Height 86 to 94 cm
Folded dimensions	Width 20 cm x Depth 58 cm x Height 91 cm
Seat	Width 35 cm x Depth 28 cm x Height 54 cm
Dimensions between handgrips	45 cm
Gross Weight	3.6 kg
Maximum weight	136 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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