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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Iceland folding toilet chair is a medical device. It is a technical aid for toileting that is designed for people who are elderly, have reduced mobility, are disabled or injured, or who can easily lose their balance.

PRODUCT DESCRIPTION

Material delivered in the box :

- ~ Frame
- ~ Backrest cushion
- ~ Seat cushion
- ~ Soft seat cover
- ~ Bucket + Lid
- ~ Feet + Rubber tips (x4)

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 125 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Check the stability of the chair before use.
- Only use on level ground.
- Before each use, check that the 4 legs are at the same height and that the clips are fully extended.
- Never bend over when using the chair.
- Beware of the risk of pinching when using the chair.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

INSTALLATION :

Open the toilet chair and make sure the chair open completely in balance state

Put on the soft seat cover on the seat cushion

Holding the toilet chair and pushing it up to fold the chair

The chair is folded

Remark : The soft seat cover is not recommended for bathing.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- We recommend servicing the product at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it is serviced more frequently.
- Before each use, check the condition of the rubber feet and seat. In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 61,5 cm x Depth 56,5 cm x Height 90,5 – 98 cm
Seat dimensions	Width 43 cm x Depth 43 cm
Seat height	46 – 54 cm
Dimensions of the seat opening	Width 22 cm x Depth 30 cm
Weight	5 kg
Maximum user weight	125 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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