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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

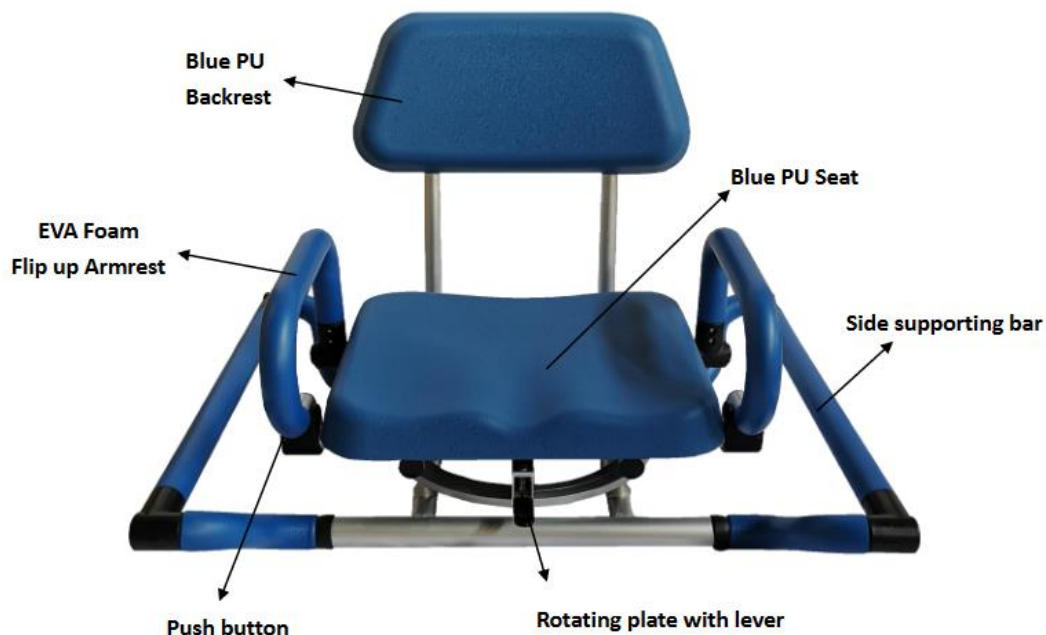
Consult your doctor or health care professional before using an assistive device. He or she will advise and inform you about the most appropriate use in your case.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Swivel bath chairs Soft are technical aids. They are intended to be used to facilitate access to the bathtub for people with reduced mobility. This rotating shower seat is designed to enable a user of limited mobility to rotate, whilst seated, into the bath to allow seated washing. It is suitable for most standard baths but is not suitable for corner baths, baths which have uneven/profiled top edges or some baths with deeply molded side panels. The frame can be adjusted to fit most common bath sizes.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Inspect all parts before assembly to ensure that they are not damaged during shipping. If there is shipping damage - DO NOT USE. Contact your dealer for further instructions.
- Always sit in the centre of the swivel bath seat and do not lean on the edge of the bath seat as this may cause instability. Do NOT stand on the bath seat.
- If any parts are damaged, broken, worn or missing, contact your dealer.
- Do not overtighten the adjustment screws
- Ensure that the side supporting bar is securely attached to the bath

Products are delivered half mounted and can be mounted without tools.

Step 1

Attach the side supports to the frame by depressing push buttons and sliding side supports into the tubes located on each side of the seat.

Make sure that push button “snap” correctly through holes and that the side supports are stable and adjusted the same way.

Step 2

Attach backrest to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located on each side and on the back of the seat.

Step 3

Place the bath seat over the bath. At least 25mm of each non-slip arm must rest on the flat sides of the bath. Check stability, the seat shall not be unstable or shaky.

USE

- (1) Lift up the lever and turn the seat to face the user, then lock the seat into position by lowering the lever and keep swiveling the seat until it locks in position.
 - (2) Transfer carefully the user into seat; beware of wet and slippery surfaces.
 - (3) Unlock the seat by lifting the lever.
 - (4) Lift the user's legs over the edge of the bath one at a time or both together while the bath seat is swiveled slowly.
 - (5) When users are into bath position lock the seat and ensure that it is locked into position and no longer rotates freely.
 - (6) To exit the bath, unlock the seat (by lifting the lever) and turn the seat slowly, while lifting the user's legs over the edge of the bath one at a time or both together.
 - (7) Lock the bath seat when the user faces the open side of the bath. The user can now stand up carefully and slowly. Using the armrests, press on the button and lift backwards the armrests.
- Rental: this chair can be rented under the condition that it was correctly cleaned.
This user's manual belongs to that product and must be delivered to each new user.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the product dry after each use.
- Periodically clean the bath seat with a damp cloth and a mild detergent or disinfectant. For example, a mild chloride-based disinfectant can be used.
- The bath seat can also be cleaned in an aqueous system up to 80°C.
- Check that the bath seat swivels easily, smoothly and locks into place. If the locking mechanism does not seem positive or if there is excessive movement in the unit, remove it from service and contact your dealer.
- Check the bather for cracks and damage, particularly to the seat and plastic tube.
- Do not use in environments above 60°C (e.g. saunas).
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check the operation of the spring loaded knob to allow 90° rotation of the product	X	
Check the condition and the fixation of the armrests		X
Check the function of the anti-slip screws	X	
Check that the brackets are fully attached to the edges of the bathtub	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

Fits standards baths with	Inside width max. 59 cm
Overall dimensions	Width 56-66,5 cm x Depth 55 cm x Height 48 cm
Seat Dimensions	Width 40 cm x depth 33,5 cm x height 42 cm
Structure	Aluminium
Weight	7,4 kg
Maximum weight	136 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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