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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Swingeasy cane is a walking aid for people with reduced mobility. It can be used indoors and outdoors.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 120 kg.
- Ensure that all parts are secure and in good working order.
- Place the cane as vertical as possible on the ground.
- Do not place the tip of the cane too far from the body.
- Check that the clips are fully extended when adjusting the height.
- Check that the anti-noise screw is tight before each use.
- Do not use the cane on a slope greater than 9 degrees or on a wet, icy or snowy slope.
- Do not make any modification to the item and do not repair it yourself! Incorrect repair can cause unforeseeable damage.
- If any parts are damaged, broken or worn, contact your dealer.

DESCRIPTION AND USE

The Swingeasy walking cane is composed of a handle, telescopic tube, fixed tube, height adjustment pin and articulated rubber tip. It provides walking support and is suitable for use on a firm surface. On soft ground the walking stick could sink and you could lose your footing. The cane has been specially designed for the comfort and safety of people with reduced mobility.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the cane regularly with warm water and a soft cloth. Do not use abrasive products or detergents.
- Dry with a clean, dry soft cloth.
- Ensure that there is no accumulation of dirt or debris around the tips.
- Do not use cleaning agents containing phenol or chlorine and all solvent-based products, as this may damage the aluminium and polyamide materials.
- Check the following parts for wear and cracks : rubber tips, handles anti-noise screw.
- In order to ensure safe and continuous use of the rods, worn or cracked parts (rubber tips, handles) must be replaced immediately with those recommended by the manufacturer. Replacement parts can be obtained from your dealer.
- If the rods are not used for a longer period of time, we recommend that they are checked by a specialist before being used.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Adjustable height	From 76 to 99 cm
Weight	310 g
Maximum supported weight	120 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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