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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Aruba toilet armrests are toilet aids for people with limited mobility. These products are intended for indoor use.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS



Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 100 kg.
- Use only the accessories and parts supplied by the manufacturer.
- Visually check the general condition of the frame.
- Make sure that the toilet armrest is correctly unfolded and that all legs are adjusted to the same height.
- Ensure that all parts are secure and in good working order.
- Do not make any adjustments to the toilet armrest while it is in use.
- Do not use the toilet armrest on stairs or escalators.
- Make sure you put your hands on each handle so that the toilet armrest can be in a parallel force position.
- Do not use the toilet armrest as a step support.
- Do not use the toilet armrest to walk backwards.


- Always lean on the toilet armrest.
- Install on a tiled floor for floor suction cups
- Ensure that the clips are fully inserted for height adjustment before each use
- In case of damaged, broken or worn parts, contact the dealer.

MOUNTING


1. Take all the items out of the carton and make sure that all the components are present (see picture below)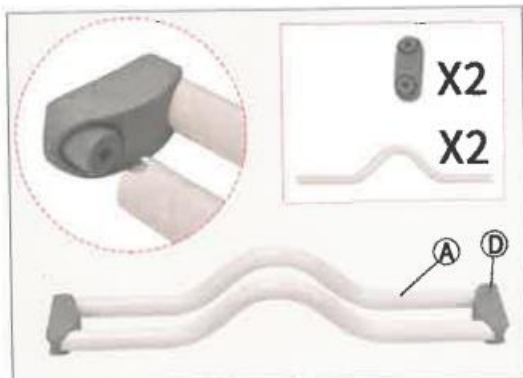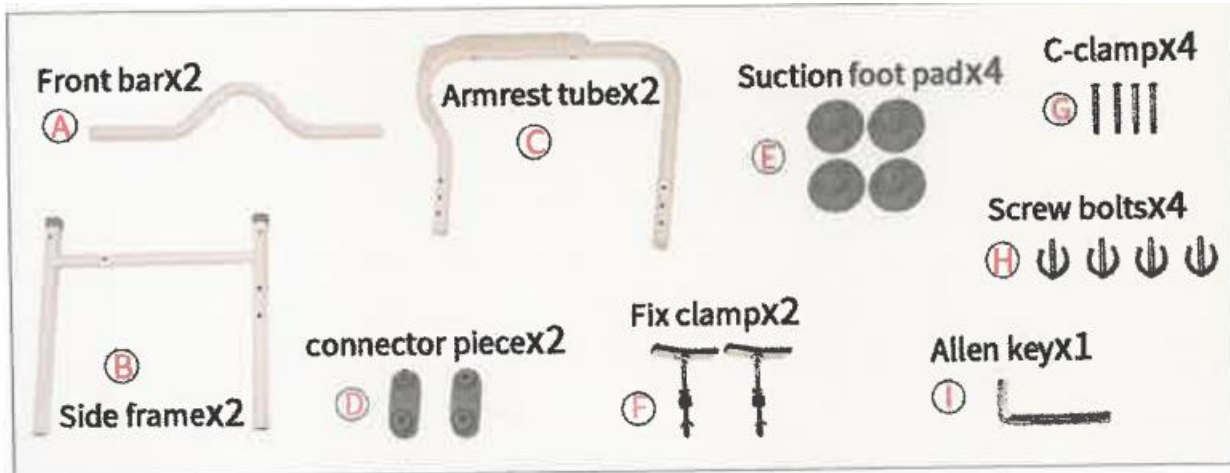


2. Insert the front bar (A) to the connector piece (D) along with its slots.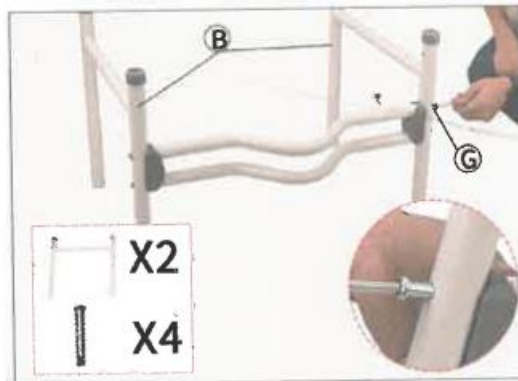


3. Insert the installed part (step 2) to the side racks (B) and insert the screw bolts (G) to the corresponding holes, fixed it with the Allen key (G).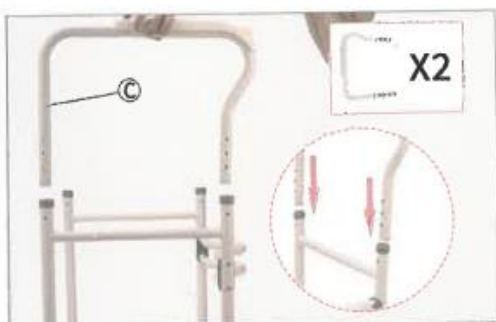


4. Insert the armrest tube (C) to the installed parts of step 3 according to the arrow shows direction.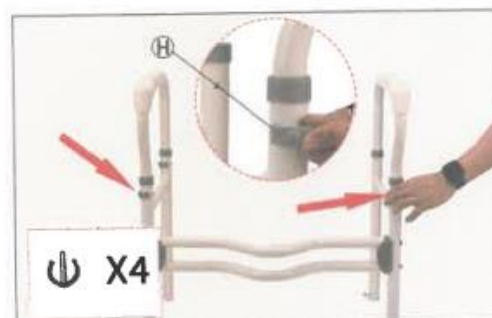


5. Insert the 4pcs C-clamp (H) to the corresponding holes.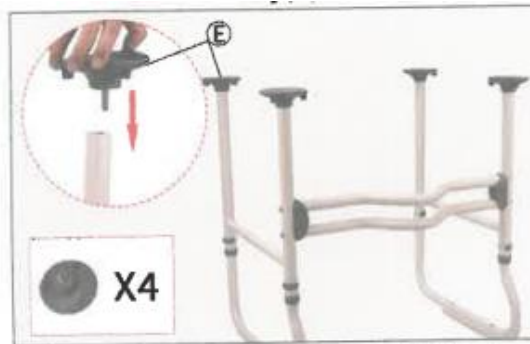

6. Twist to fix the suction foot pad(E) to the corresponding part in the installed frame (step 5).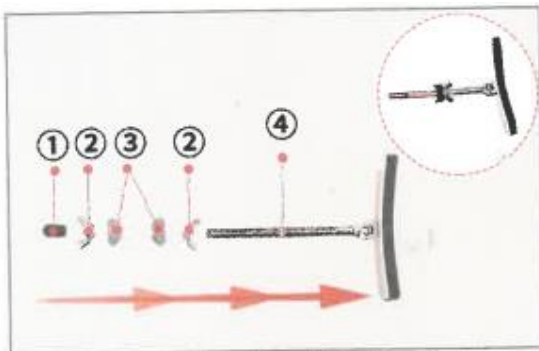

7. Separate the fix clamp(F) to be single parts as shows in the picture: ①, ②, ③, ④.

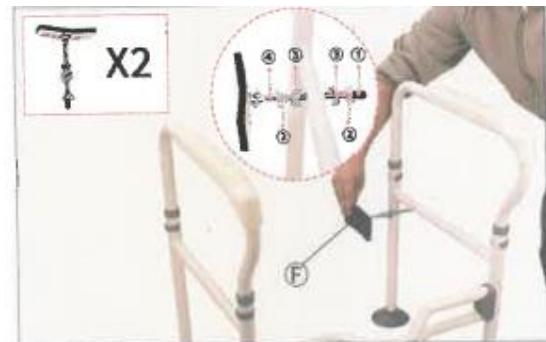8. Assemble the small parts that is separated in Step 7 from the inside out in the order ④, ②, ③, ③, ②, and ①.

9. First, loose the fixing screws, rotate the left and right fixing blocks according to the direction of the picture shows(make sure the left and right are in balanced), and then fix it with the fixing screws.

USE

To use the armrest, place your forearms on the armrests and your hands on the handles. Adjust the height if necessary. Move forward slowly.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth to avoid oxidation.
- Ensure that there is no accumulation of dirt or debris around the nozzles.
- Do not use cleaning agents containing phenol or chlorine, as this may damage the aluminium and polyamide materials.
- Periodically check all screws, nuts and other components for wear and strength.
- Before each use, check that the fixing plates adhere well to the toilet
- Regularly check the condition of the armrests and the tightness of the fixings
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

FEATURES TECHNICAL

| | |
|--------------------------|---|
| Overall dimensions | Width 52 cm x Depth 43 cm x Height 72,5/78,5 cm |
| Screw rods length | 17 cm |
| Maximum supported weight | 100 kg |

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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