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USER MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The children's toilet commode chair Ulysse is a medical device. It is an assistive toileting aid designed for children under 1.4 meters in height. It provides comfortable positioning and facilitates toilet use for children with disabilities or those who may easily lose balance. The chair is equipped with a removable potty.

PRODUCT DESCRIPTION

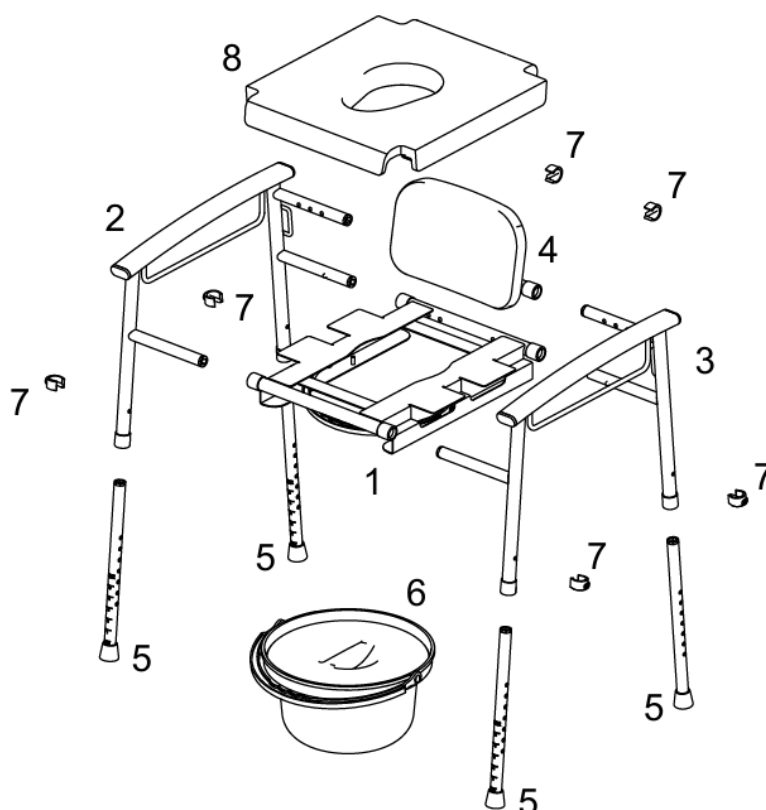Parts List :

1. Seat frame × 1
2. Side frame (right) × 1
3. Side frame (left) × 1
4. Backrest × 1
5. Legs × 4
6. Toilet bucket with lid × 1
7. E-clips × 6
8. PU padded seat × 1

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.


- Never exceed the maximum load capacity of 100 kg.
- Ensure that all components are securely fastened before use.
- To avoid the risk of falls, make sure the path leading to the chair is free of obstacles.
- Ensure that the product is in good working condition.
- Visually inspect the product (seat, backrest, tubes, armrests, etc.).
- Check the stability of the chair before each use.
- Before each use, ensure that the backrest is properly secured with the clips.
- Use only on a flat, level surface.
- Before each use, check that all four legs are set to the same height and that the clips are fully engaged.
- Never lean while seated on the chair. The risk of tipping increases with uneven or unilateral heavy loads.
- Do not stand on the chair – risk of slipping!
- Beware of pinch points during use.
- If any parts are damaged, broken, worn, or missing, contact the retailer.
- All serious incidents occurring in relation to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.
- Ensure that the height adjustment buttons fully protrude through the corresponding holes of each leg extension. This ensures that the leg extensions are securely locked in position and that the height is even.
- Check that the rubber tips on the leg extensions are not torn, worn, or missing. Replace any defective tip immediately.
- All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
- Regularly check that all E-clips are correctly in place.
- Ensure that the seat is cleaned with an antiseptic wipe after each use.

ASSEMBLY & USE

1. ASSEMBLY / INSTALLATION


1.1 Installation of the backrest assembly onto the left side frame

- Fit the backrest assembly onto the left side frame and secure it using the clip.
- The toilet chair features three width adjustment positions.
The backrest is shown installed in the widest setting.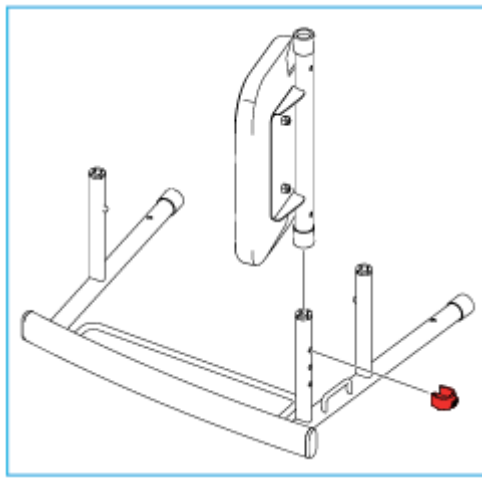

1.2 Installation of the seat frame assembly

- Fit the seat frame assembly onto the left side frame.
- Press and hold the clips, then push down on the seat frame assembly until the clips engage in the correct hole positions.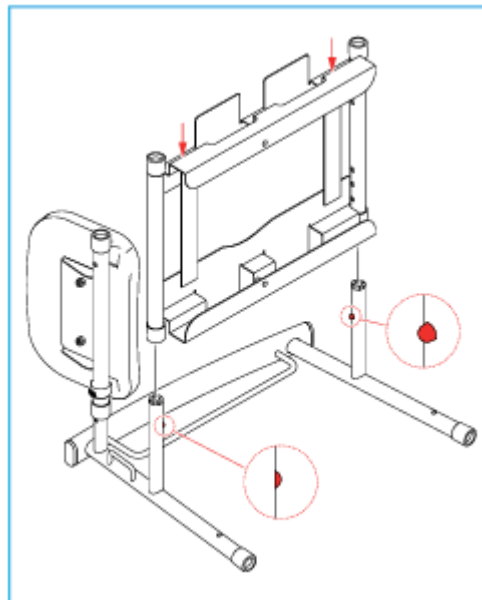


Warning : The position of the seat frame must be aligned with the position of the backrest; otherwise, the assembly of the right side frame will not be completed correctly.

1.3 Installation of the right side frame (RH) onto the backrest / seat assembly

If the side frames are installed in the narrowest position, it is recommended to install the padded seat moulding before fitting the right side frame.


- Fit the right side frame into the seat/backrest assemblies.
- Press and hold the clips, then push down on the right side frame until the clips engage in the correct hole positions.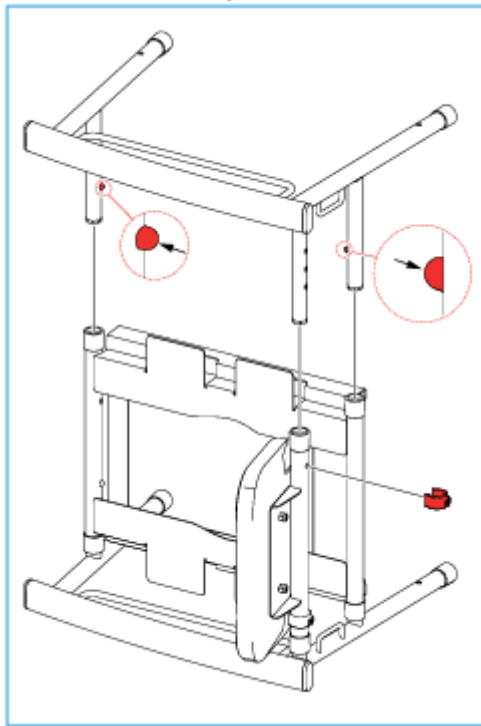

Warning: The right side frame must be set to the same width as the left side frame.


1.4 Installation of the legs

Insert the legs into the side frame assemblies and secure them using four X-clips. Ensure that all legs are adjusted to the same height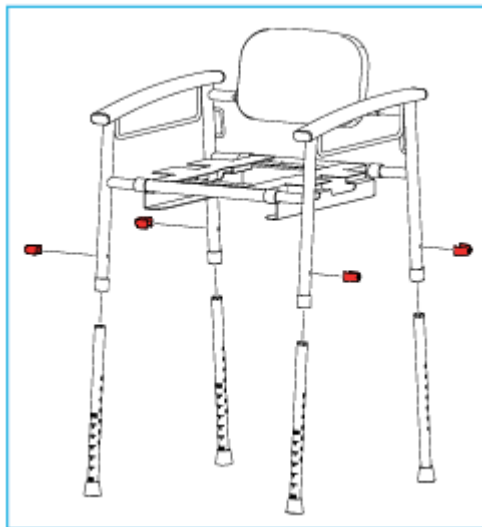

1.5 Installation of the padded seat

Install the padded seat onto the seat frame. Ensure that the commode chair is fully secured before use.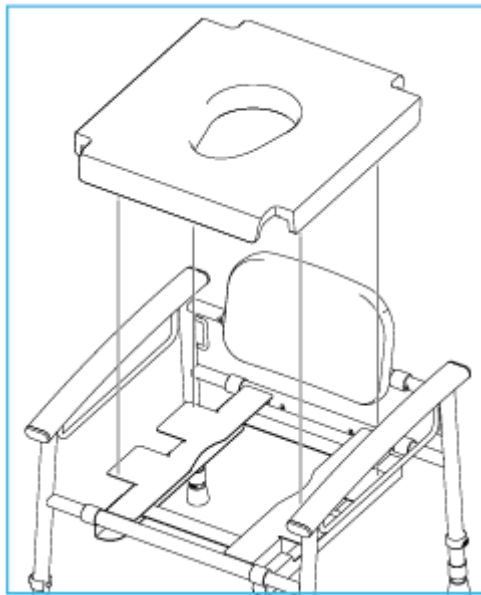


1.6 Installation of the bucket (pot)

Slide the bucket into the rails located under the seat.
The bucket is held in place by retaining pins; pull it forward to remove it.

2. COMMODE CHAIR ADJUSTMENT

2.1 Height adjustment

- Remove the four clips and adjust the legs to the desired height.
- Refit the four clips.
- Ensure that all legs are set to the same height.

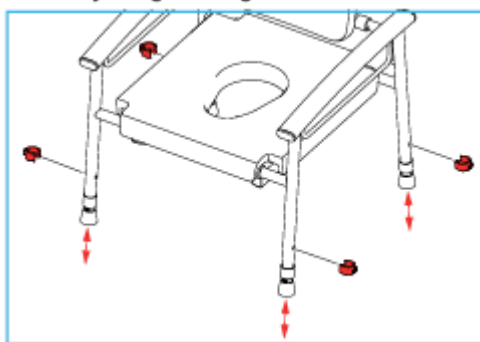2.2 Width adjustment

- Remove the two backrest clips.
- Press and hold the spring clips to release each section of the side frames, then adjust to the desired width.
- Align the backrest holes and refit the two backrest clips.

USE

- Do not allow children or unauthorized persons to play with or use the equipment without adequate supervision.
- Do not rock forwards, backwards, or sideways while using the product, as this may cause excessive stress and could result in damage or failure leading to injury.

- Do not use the product outdoors or on uneven surfaces (such as in a garden), as this may place stress on the legs and may cause damage.

CARE AND MAINTENANCE

Failure to properly maintain and service your product may result in a risk of serious injury.

- Clean the product regularly using a damp cloth and a disinfectant. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth.
- The chair must be fully inspected on a monthly basis.
- It is recommended to carry out product maintenance at least once per month. If the product is subject to intensive use and/or used in a multi-user environment, more frequent inspections are recommended.
- Before each use, check the condition and operation of the following components: seat, backrest, legs, adjustment knobs, feet, bucket.
- Check that all fixings and button clips are correctly fitted and that there is no excessive movement.
- In the event of a malfunction, do not attempt to repair the product yourself. Consult an authorized dealer.

Type of control	Method	Fréquency
Seat and backrest inspection	Adjustment / Visual inspection	Before each use
Leg inspection	Adjustment / Visual inspection	Before each use
Fastener inspection	Adjustment	Before each use

TROUBLESHOOTING

PROBLEM OBSERVED	POSSIBLE CAUSE	RECOMMENDED SOLUTION
Malfunction of the height adjustment and clips	Incorrect adjustment or incorrectly positioned clips	Check hole alignment, reposition the clips correctly, and ensure they are securely locked
The four legs are not the same height	Uneven leg adjustment	Adjust each leg to the same height and check stability before use
Defective seat/backrest	Manufacturing defect or incorrect assembly	Check the assembly of the seat and backrest; contact after-sales service if the issue persists

CLEANING

- Clean regularly using a neutral, general-purpose detergent and lukewarm water.
- Check the compatibility of the cleaning product before use.
- Do not use abrasive, solvent-based, or aggressive products, nor abrasive cloths, as this may damage the surfaces and void the warranty.
- Use only a soft cloth.
- For disinfection, wipe with a diluted bleach solution, then rinse thoroughly with clean water.
- This toilet chair is compatible with thermal decontamination: steam cleaning or autoclave cleaning is possible up to 80 °C.
- Allow the product to dry completely after cleaning or disinfection.
- If necessary, dry with a clean, soft, dry cloth before use or storage.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Overall dimensions	Width 37.2 to 47.2 × Depth 45.6 to 47.7 × Height 49.7 to 61.7 cm
Seat dimensions	Width 30 × Depth 35.1 × Height 27 to 39 cm
Width between armrests	30 to 40 cm
Height from the bottom of the backrest to the seat	22 cm
Armrest height	13 cm
Weight	5,2kg
Maximum allowed weight	100kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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