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IMPORTANT

Read the instructions carefully before use. Follow all guidelines and keep them in a safe place! If this aid is to be used by others, they must be properly informed of the following instructions prior to use.

PRECAUTIONS FOR USE

- Always consult your specialist dealer to install the device and to learn how to use it properly.
- Do not store outdoors or expose to direct sunlight (Be careful not to leave it inside a car in full sun!).
- Do not use in extreme temperatures (above approximately 38°C or 100°F or below 0°C or 32°F).
- Check the condition of each riser regularly before use.
- Do not use this product for any purpose other than that for which it is intended.
- ALWAYS respect the weight limits indicated on the label or in the user manual.
- Only qualified personnel are authorized to repair this item.
- If the product is not used for an extended period, have the risers checked by your specialist dealer.

USE

This product must only be set up by qualified personnel. Improper installation can lead to incorrect use and risk of accidents. It is therefore essential that the product be individually adjusted to each user by qualified personnel. If in doubt, do not hesitate to consult your dealer.

Indication

These riser blocks elevate beds and chairs to make it easier for users to sit down or stand up from their bed or chair. When placed under a bed, they create additional storage space.

The top of the blocks features a lip that helps secure the furniture leg inside the block.

Instructions for Use

Inspect the product upon unpacking; do not use a cutter or knife to avoid damaging it.

Lift the furniture and gently position each leg in the center of a block. Repeat this process for all four legs.

Ensure each leg is properly centered in its block and check the stability of the furniture before use.

MAINTENANCE

Clean the product regularly with a clean, dry cloth and a non-abrasive household cleaner.

TECHNICAL SPECIFICATIONS

Composition	Polypropylene
Inner Diameter	9 cm model (REF. 815208): 8.7 cm 14 cm model (REF. 815209): 7.6 cm
Base Diameter	9 cm model (REF. 815208): 16.5 cm 14 cm model (REF. 815209): 16.5 cm

RECYCLING

Dispose of this product in accordance with current regulations.

WARRANTY

This product is guaranteed for 2 years from the date of purchase against any manufacturing defects.

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