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IMPORTANT

Read the instructions carefully before using this product and keep them for future reference.

User: If you are unable to read or understand the warnings, precautions or instructions, please

contact your healthcare professional or distributor before using this product as this may cause risk and harm. Consult your physician or health care professional before using any assistive device. They will advise and inform you on the most appropriate use for you.

Distributor: These instructions should be given to users of this product.

USER PROFILE

The Spirit X3 Rollator is a walking aid for people with reduced mobility. This product is intended for indoor and outdoor use and folds up for easy storage or transportation.

SAFETY PRECAUTIONS

Failure to follow safety precautions can result in serious injury.

- Do not exceed the maximum weight limit of 130kg.
- Do not exceed the maximum load limit in the basket (5kg). Loads should only be carried in the basket. Do not hang heavy objects on the rollator as this may affect its stability.
- Ensure that all parts (wheels, cables and handles) are secure and in good working order.
- Make sure that the knobs are tight and at the same height.
- Do not use the rollator on stairs or escalators.
- Only use the accessories and parts supplied by the manufacturer.
- The rollator is a walking aid. **It is not a wheelchair.** It is not designed to carry a person sitting on it. **Such use may damage the wheels and the structure of the rollator itself.** This may cause the rollator to become brittle and not meet its mechanical requirements for future use. Only use the rollator as a walking aid.
- Do not use the rollator to walk backwards.
- Do not make any adjustments to the rollator while it is in use.
- Check that the brakes are working properly before use.
- All wheels should always be in contact with the ground.
- When the rollator is **in a stationary position**, the brakes should be locked. Do not attempt to move the rollator if the brakes are locked.
- Ensure that you put your hands on each handle so that the rollator can be in a parallel force position.
- Take care when climbing pavements or when using sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on wet, icy or snowy slopes.
- If any parts are damaged, broken or worn, contact your dealer.

ASSEMBLY & USE

Remove the rollator from its box and check that all components are present. Remove all packaging and protective covers. Ensure all components are in good condition before assembly.





ADJUSTMENT OF THE HANDLE HEIGHT :

- a. Put the rollator in park mode (see figure 1).
- b. Press and hold the adjustment button on the top of the frame (see figure 2).
- c. Hold the button down and adjust the height of the handle by raising or lowering it (see figure 3).
- d. Release the button to lock the height adjustment. Pull the handle up and push it down to check that it is locked (see figure 4).
- e. Adjust the other handle in the same way.
- f. Check that both handles are set to the same height mark.

Note : Do not remove the inner tube from the outer tube when adjusting the handle height.

To ensure the rollator is at the correct height, stand behind the rollator with your arms at your sides. Then adjust the handle height to the height of the user's wrists.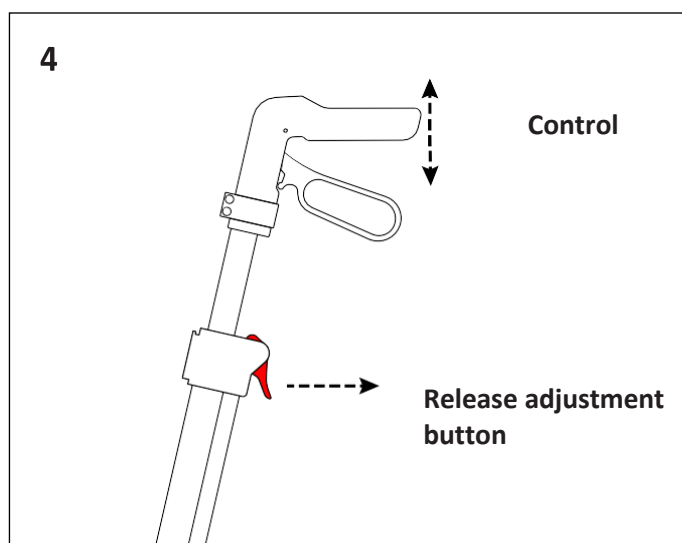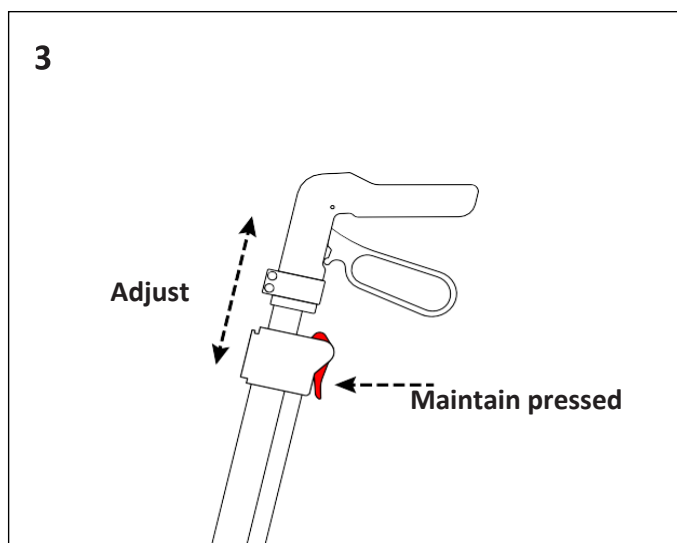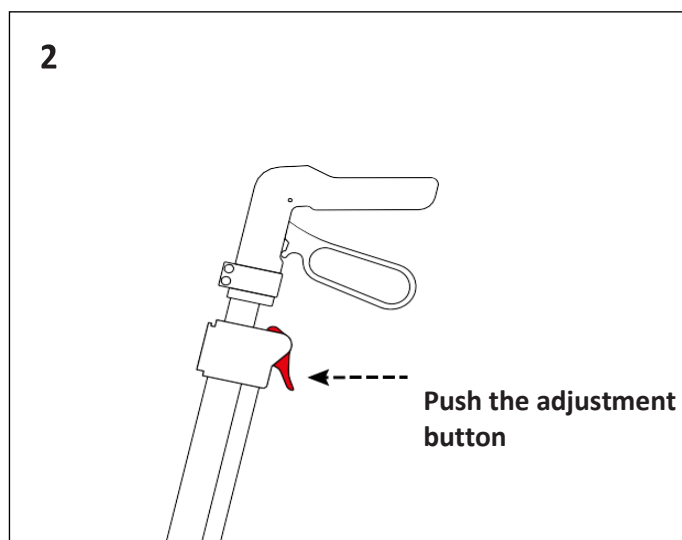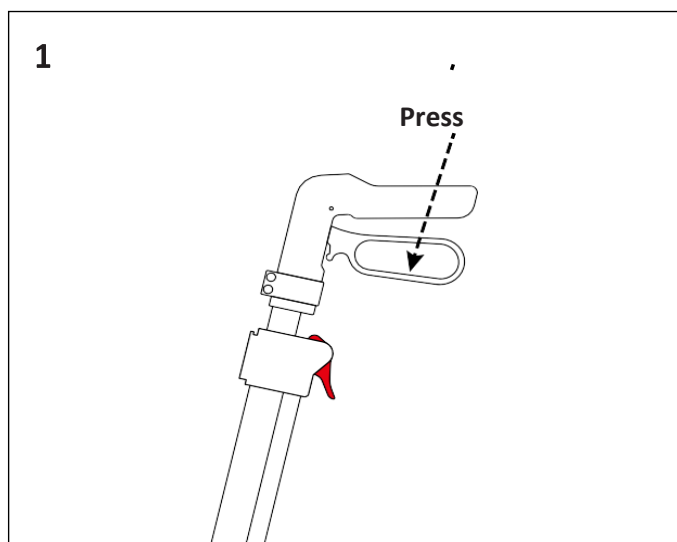

WALK WITH THE ROLLATOR :

- While in use, stand straight behind the rollator, do not bend your back.
- Adjust the handles height if the use position is not correct.

FOLD THE ROLLATOR :

- a. Lift the seat by the seat strap in the centre to release the locking mechanism.
- b. To ensure that the rollator is fully folded, push the handles together.

The Spirit X3 rollator is designed to stand on its own.

UNFOLD THE ROLLATOR

- a. Lift the seat by the seat strap in the centre to release the locking mechanism.
- b. Push down on the outer seat rails in one quick movement until you hear a significant click from the locking mechanism

USE OF THE BRAKES :

The rollator is equipped with brakes, operated by two brake levers. The brake levers can be used in two braking modes: walking brake and parking brake.

Check the operation of the brakes to ensure your safety. The condition of the brake system is influenced by wear, oil and other contamination of the rear wheels and brake mechanism.

- To use the travel brakes, pull both brake levers upwards simultaneously; to stop braking, release the brake levers.
- To use the parking brakes, push both brake levers down to the locked position [P]. Always use both parking brakes simultaneously. To end the parking mode, pull both brake levers upwards simultaneously.

USE OF THE SEAT

Before using the seat :

- Ensure that the backrest belt is securely attached to the frame.
- Check that all screws, bolts and nuts on the strap are tight.

PAVEMENT PASS

It is equipped with a pavement pass that makes it easier to negotiate small obstacles such as pavement and low steps.

1. Make sure you are in a stable position while holding both handles.
2. Use one foot and push on the tilt lever to lift the front wheels.
3. Hold the rollator in this position with the handles and slowly move forward until the front wheels are on the pavement.
4. Now lift the handles and move forward to place the rear wheels on the pavement.
5. When the rollator is stable, you can step onto the step or pavement yourself.

The stability of the rollator changes when it is tilted. Be careful and try to go around the obstacle rather than over it

GENERAL USE

Make sure both brake levers are in the park position before sitting down. Hold the brake levers in this position until you start walking again. Keep your feet between the rear wheels.

CLEANING - MAINTENANCE

Failure to service and maintain your product can result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasives, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Ensure that there is no build-up of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine as this may damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check all screws and bolts for wear and strength. Replace all broken, damaged or worn parts immediately.
- Check that the brakes work before each use.
- Check the parking brakes by pushing the handles into the locked position and then attempting to push the rollator.
- If a malfunction occurs, do not repair the rollator yourself. Consult an authorised dealer.

- **Checklist :**

To keep the rollator in good condition, check before each use, and check regularly as follows :

Check the item	Anytime	Weekly	Monthly	Yearly
Clip and knob for height adjustment		X		
Structural weld at wheels		X		
Cleanliness	X			
Screws and fasteners		X		
Stability of frame and backrest			X	
Condition of seat				X
Brakes	X			

CHANGE OF OWNER

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene regulations. When transferring the product, remember to hand over all the necessary technical documents to the new user. The product should be inspected by an authorised specialist beforehand.

RECYCLING

When the product has become unusable and you need to dispose of it: please contact your specialist dealer. If you wish to recycle the product yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

- Do not store your product for an extended period of time near a heat source or in direct sunlight (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from open flames and sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together, in the same place (or mark them if necessary) to avoid mixing them with other products during reassembly.
- All components should be stored without any load (do not place heavy objects on the product parts, do not jam anything...).

LIFETIME OF THE PRODUCT

The expected lifetime of this product is 5 years under normal user use, safety and maintenance.

user. Beyond this period, the product can be used for as long as it is still in good condition.

If the product is used in a multi-user environment (e.g. nursing home or hospital), the lifetime may be reduced.

TECHNICAL FEATURES

Overall dimensions	Width. 61 x Depth. 69 x Height. 77 / 96 cm
Handle width	45 cm
Wheel diameter	22 cm
Maximum load for the basket	5 kg
Weight	7.2 kg
Maximum load	130 kg

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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