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## INSTRUCTIONS MANUAL EN

### IMPORTANT

Read the instructions carefully before using this product and keep them in case of need.

User : If you are unable to read or understand the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause risks and damage. They will be able to advise and inform you on the most appropriate use.

Distributor: this leaflet must be given to users of this product.

### PRODUCT DESTINATION

Technical positioning aid with memory effect for lying patients. This product is designed for patients at risk of bedsores. It prevents bedsores in immobilised patients lying down. The product acts by reducing pressure points as well as friction points on the knees.

This device is intended for carers and must be prescribed by a doctor. Read each property of each cushion before use.

### PRECAUTIONS FOR USE

This product must be used on the advice of a healthcare professional following a medical prescription.

If the patient does not accept the product, do not use it. If the patient does not tolerate the product, do not use it.

If the patient experiences pain when using this product, do not use it.

Follow the instructions in the medical prescription and the planned treatment protocol.

Do not use a soiled or undecontaminated cushion.

Do not use the product if it is damaged - Check the seams before use.

Failure to observe the precautions for use may result in discomfort and injury to the hemiplegic patient.

### USE

It is advisable to position the patient every 2 hours in a 30° supine position (semi-fowler) and in a 30° left and right lateral position. Read each cushion specification and possible combinations. Ask a healthcare professional for advice.

Clinical monitoring involves ensuring that the patient is properly hydrated and eating properly. The product is available in grey and aubergine.

### CHANGE OF PATIENT

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions.

When changing patient, remember to give this leaflet to the new user.

### CARE AND MAINTENANCE

When the product is to be re-used on another patient: it must be decontaminated and used in perfectly hygienic conditions.

When changing patient, remember to give this leaflet to the new user.

The product should be machine-washed at 40°C.

Disinfect the cushion after each use. Dry with a soft, clean, dry cloth. Do not use abrasive products, detergents or chlorinated products. In the event of wear or tear, do not attempt to repair the product. Dispose of it or contact your retailer to find out if it is covered by the after-sales service.

### RECYCLING

When the product has become unusable and you need to dispose of it, contact a local waste disposal company.

If you wish to recycle the product, please enquire locally.

### STORAGE

Failure to comply with storage conditions may lead to deterioration of the product and therefore a risk of injury.

The product can be stored between 0°C and 50°C.

Keep away from sources of ignition.

Observe the following conditions when storing your products

- In a dry, temperate place
- Protect your product with dust-proof packaging
- Do not place heavy objects on the cushion: the foam could settle and deteriorate.

### TECHNICAL CHARACTERISTICS

**Assembly:** polyester thread stitching

**Filling:** 100% polyester with memory effect.

This product contains biocides in accordance with EU Standard 528/2012. Protection against microbiological deterioration.

**Active ingredients:** silverglass complex 398477-47-9 + zinc - pyrethione

**Cover :** 54% polyester – 46% polyurethane

<b>Overall dimensions</b>	35 x 22 cm
<b>Weight</b>	0,250 kg

## PERIOD OF USE

The expected life of this product is 2 years under normal use and maintenance. After this period, if the product has lost its memory properties, if the cover is damaged or if the Velcro fasteners are damaged: do not use this product. It can be used as long as it is still in good condition.

The lifespan of the product will vary according to the weight of the patients who use it and the number of times it is washed. The flattening of the product will be an indicator of its age.

## WARRANTY

This product is guaranteed for two years from the date of purchase by the user against defects in stitching and workmanship. Contact the distributor from whom you purchased the product should you need to make a claim. Failures resulting from accidental damage, misuse, improper storage, improper unpacking, modifications or wear and tear are not covered by this warranty.

Contact the retailer from whom you purchased the product to apply the guarantee.

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