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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Mobile Toilet Seat Sardinia is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, people with disabilities, people with injuries or people who can easily lose their balance.

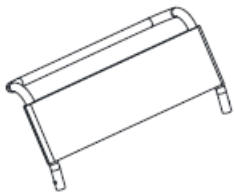PRODUCT DESCRIPTION

	x1 Backrest		x1 Pail
	x1 Cushion		x1 Pedal

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum weight capacity of 130 kg.
- This product is not suitable for patients with cognitive impairment or who are unable to remain seated.
- Ensure that all components are secure before use and that the product is in good working order.
- Visually check the product. Check that the chair is stable before use.
- Use only on level ground. Before each use, check that the 4 legs are at the same height and that the clips are fully extended.
- Never bend over when using the chair. Beware of the risk of pinching when using the chair.
- If any parts are damaged, broken, worn or missing, contact your retailer.
- Regularly check all the parts that make up this item (screws, tubes, frame, seat, seat cover, etc.). Be careful not to sit down without first installing the seat. Check that all screw connections are tightened correctly.
- Do not use for any purpose other than that for which the product is intended: do not stand on the chair, for example.
- ALWAYS comply with the weight limits shown on the label or in the instructions.
- Do not modify this product. If necessary, use only original parts to repair it. Only qualified personnel should repair this product.
- Once you have unfolded or assembled the product, make sure that it is locked in the OPEN position and that it is stable on the floor before using it. Do not hang anything from your chair. This could throw you off balance and cause you to fall.
- If you are not going to be using your appliance for a long time, have it checked by your specialist dealer.
- Do not put on or remove the bucket when the patient is sitting in the chair: there is a serious risk of injury to the genital area!
- If in doubt, do not use it and ask your specialist dealer for advice.
- Only use the chair on a level, stable floor. Do not sit on the chair's armrests. Do not bend over to avoid tipping over.


MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.


INSTALLATION :

This product should only be commissioned by qualified personnel. If in doubt, ask your dealer.


STEP 1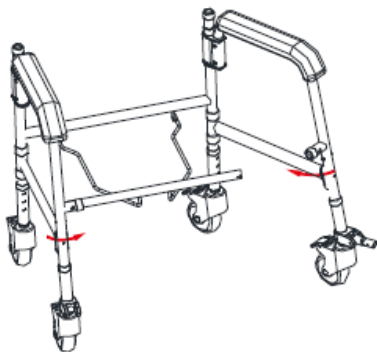

STEP 2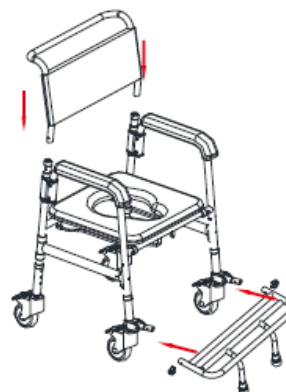

STEP 3

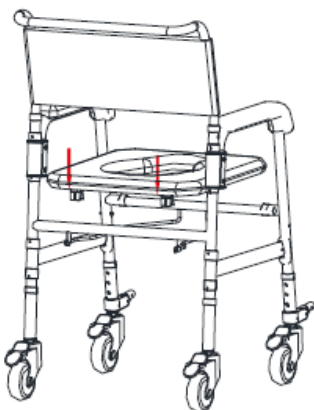STEP 4

USE :

- Do not allow children or unauthorized persons to play with or use the equipment without adequate supervision.
- Do not rock backwards and forwards or sideways when using the product, as this will cause excessive stress and may result in damage and failure which could cause injury.
- Do not use the product outside the house or on uneven surfaces (such as in the garden), as this will put stress on the feet and could damage them.
- Do not put your fingers or foreign objects into exposed tubular sections, as this could cause serious injury.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with a damp cloth and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Before cleaning the product, separate the components from the main frame.
- The chair must be completely overhauled every year, with compulsory lubrication.
- It is recommended that the product be serviced at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it be serviced more frequently.
- Before each use, check the condition of the wheels and seat. Check the brakes before each use.
- Regularly check that the seat and seat cover are securely fastened.
- Be careful, as urine is highly corrosive and attacks steel tubes very quickly. It is therefore important to clean and dry the chair daily.
- In the event of a malfunction, do not repair the product yourself. Consult an authorized retailer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorized specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).
- Do store outdoors, do not leave in the sun (Be careful not to leave inside a car in direct sunlight).
- The chair is intended for indoor use only and must not be stored outdoors.

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition. If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL SPECIFICATIONS

Overall dimensions	Width 78.5 x depth 55 x height 91.5 to 94.5 cm
Weight	5,87 kg
Maximum user weight	130 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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